



Assembly materials:


1 hygienic protective barrier (HPB)

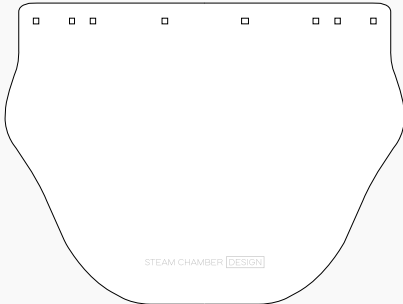

2A headband

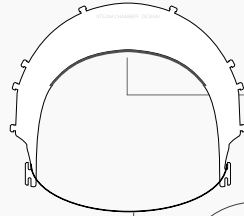

2B headrest

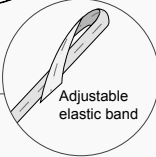

To Sanitize:

Simply cleanse the entire faceshield using non-abrasive¹ cleansers with any of the following techniques below:

- hand wash with soap and warm water
- OR suberge in diluted bleach solution
- OR swab with antibacterial wipes.

Avoid abrasive cleaners¹ or tools that can scratch the faceshield surface over time.

Remove elastic band and allow both components to fully air dry or blot dry with a towel.

1. Compatible cleaning products include 0.5% bleach, 1.4% hydrogen peroxide, alcohol, isopropyl-based wipes.



INSTRUCTIONS:

Before beginning, gently remove the protective plastic sheeting from all pieces using a razor and/or commercial degreaser as needed.

1. To attach the headrest (2B) to the headband (2A)

Lay the headrest on a flat surface (logo should appear backward, on the other side)

Form the headrest in an "s" shape and snap into slots on piece 2A (headband) from right to left

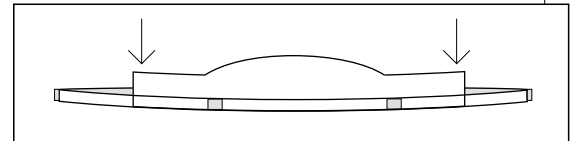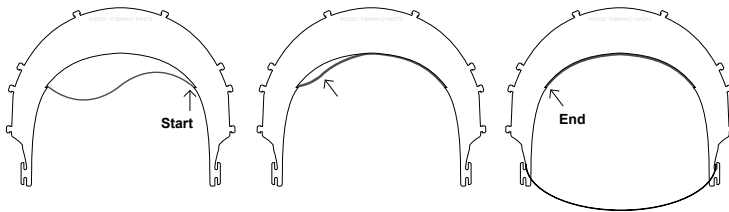


2. To attach the completed headband (2AB) to the hygienic protective barrier (1)

Position headband on the inside of the HPB (logo will appear right side up)


Insert each headrest attachment into its corresponding hole (right to left)

(the headrest should be facing down as show in the diagram below)

Attach buttonhole elastic to both side of the headrest, adjust as needed

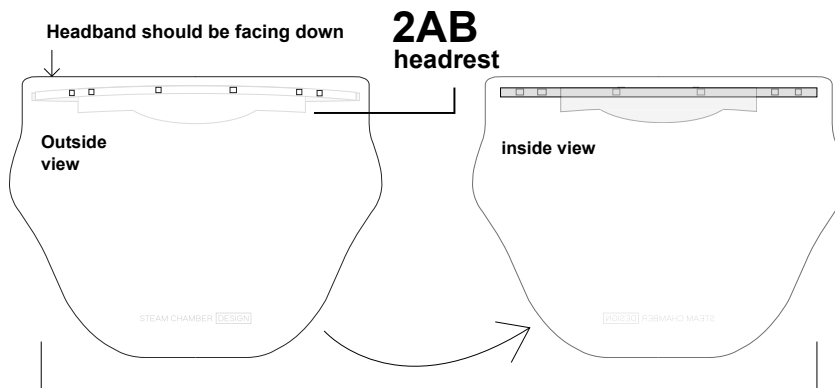

Set up

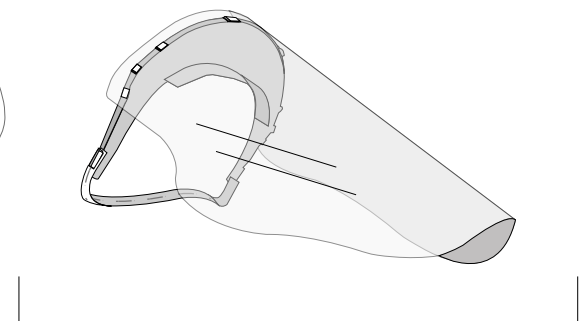

Final