



INSTRUCTIONS FOR THE BIPLANE[®] HANDLE WITH ADJUSTABLE LUER LOCK

The Biplane[®] liposuction handle is an ergonomic handle with an adjustable Luer Lock coupling to allow fixed positioning of the liposuction cannula. This allows the hole patterns to be positioned exactly as preferred by the physician.

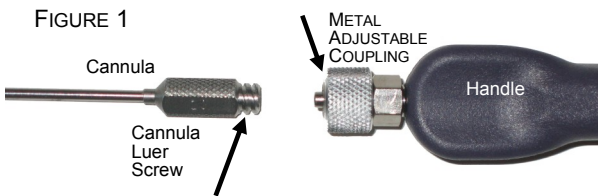

A. First familiarize yourself with all the parts of both the Adjustable Luer Lock Biplane[®] Handle and the Luer Lock Cannula. (See Figure 1)

FIGURE 2

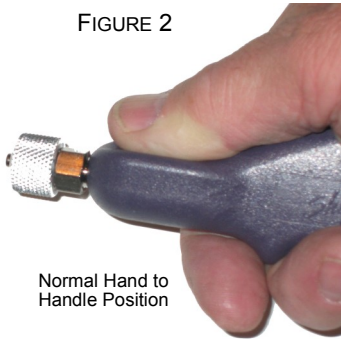

B. Place the Biplane[®] in your hand in normal working position. (See Figure 2)

Normal Hand to Handle Position


C. Move your thumb and index finger forward over the Luer Lock tip. (See Figure 3)

FIGURE 3



D. With your other hand, place the Luer Lock end of the cannula in contact with the Luer Lock end of the Biplane[®] Handle and position the hole pattern in the desired position. Usually this means positioning the holes downward.

FIGURE 4

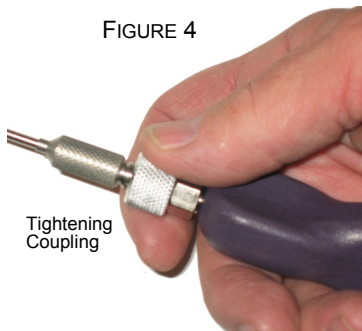

E. Using the thumb and index finger of your working hand, tighten the rotatable coupling in a clockwise direction (See Figure 4). This tightens and locks the Luer. Tighten this coupling as tight as you can manually. **Do not use pliers or other metal devices that may damage the threads.**

Thumb and Index Finger on Coupling

F. To loosen, rotate coupling counterclockwise and the cannula clockwise.

Cleaning: Clean and Autoclave the Biplane[®] handle and cannula after each patient.



CEpartner4U, ESDOORNLAAN 13
3951DB MAARN, THE NETHERLANDS
www.cepartner4u.eu

Phone: 303-754-0044 or 1-800-888-8663
Fax: 303-754-0318 or 1-800-284-0864
www.shippertmedical.com



Shippert Medical Technologies, Inc.

6248 South Troy Circle, Suite A

Centennial, CO 80111 USA

1-800-888-8663

Label PN
03L 1418 rev A