

Cannula Cleaning and Sterilization Instruction

Each Cannula and/or Handle must be:

- Immersed in hot, soapy water using a suitable detergent
- Carefully scrubbed internally and externally with a Cannula Brush to remove all debris
- Rinsed thoroughly to remove all debris and detergent
- Dried
- Packaged
- Follow the Autoclave manufacturer instructions, the processing guidelines of your practice, or the following general industry process:
Autoclave at 265 – 270 degrees with 15 psi for at least 15 minutes.

Assembly and Usage for Cannulas and Handles

Luer Lock Cannula:

Connect Luer Lock to suction hose and aspiration system or injection device.

Cannula with Biplane or Blue Aluminum Handle:

Connect Cannula to suction hose or injection device.

Biplane or Aluminum Handle Only:

Attach a luer lock cannula to luer end.

Connect handle to suction hose and aspiration system.



CEpartner4U, ESDOORNLAAN 13
3951DB MAARN, THE NETHERLANDS
www.cepartner4u.eu

Phone: 303-754-0044 or 1-800-888-8663
Fax: 303-754-0318 or 1-800-284-0864
www.shippertmedical.com



Shippert Medical Technologies, Inc.

6248 South Troy Circle, Suite A

Centennial, CO 80111 USA

1-800-888-8663

Label PN
03L 1436 rev A