



# H0124 ALUMINIUM THRESHOLD RAMP

## INSTRUCTION MANUAL

This 3-piece threshold ramp kit is an aluminium portable wheelchair ramp system that is very lightweight, easy to carry and set up with a permanent textured non-skid free surface. The 3 pieces can be used all together to achieve a 60mm height. Alternatively, you can remove the largest piece to reduce to 40mm or remove the two larger pieces to reduce to a 20mm height.

**MAXIMUM WEIGHT CAPACITY IS 400kg**

### 1. WARNING

- 1.1 **NEVER** exceed the rated capacity of your ramp.
- 1.2 Use caution at all times when using the ramp.
- 1.3 **NEVER** stand in the path of equipment or vehicle. Keep bystanders and children away.
- 1.4 Ramp surfaces will have less traction when wet. Use caution.
- 1.5 The ramp is not a toy. **DO NOT** use the ramp as a jump, scaffold, bridge, ladder or anything other than its designed purpose.
- 1.6 **ALWAYS** follow manufacturer recommendations for wheelchairs and scooters.

### 2. OPERATION GUIDANCE

- 2.1 Position the back of the largest piece (tall side) against the step where it is to be used.
- 2.2 Then, place the middle size piece down over the groove of the largest piece to lock them together.
- 2.3 Continue with the smaller piece.

### 3. MAINTENANCE

- 3.1 It is the user's responsibility to check the loading ramp and components before each and every use. If the ramp is damaged, it should **NOT** be used.
- 3.2 Maintain the ramp by regular cleaning with a damp cloth.