



HEEVE ALUMINIUM MULTI-FOLD PREMIUM WHEELCHAIR RAMP - 272KG CAPACITY

INSTRUCTION MANUAL

MAXIMUM *OCCUPIED / **UNOCCUPIED HEIGHT LIMIT TO GROUND LEVEL

Ramp Length	Occupied Max Loading Height	Unoccupied Max Loading Height
2100mm	350mm	525mm
2400mm	400mm	600mm
2700mm	450mm	675mm
3000mm	500mm	750mm

I. OPERATION GUIDANCE

A. Loading Wheelchair:

1. To load the wheelchair, position it in the direction of the ramp with the front wheel facing forward.
2. Slowly and steadily roll or ride the wheelchair up the ramp, making sure the wheels follow the centre of the ramp.
3. Exercise extra caution in wet conditions to prevent the wheels from sliding on the ramp.

B. Unloading Wheelchair:

1. For unloading, position the ramp directly behind the rear wheel of the wheelchair as explained above.
2. Allow the wheelchair to roll backwards down the ramp, ensuring the wheels follow the centre of the ramp.
3. Always have a qualified assistant present when using the ramp.

II. MAINTENANCE

Keep your ramp functioning properly by following these maintenance tips:

A. Check before use:

1. It is the user's responsibility to check the loading ramp and components before each use.
2. If the ramp is damaged, it should NOT be used.

B. Regular cleaning:

1. Maintain the ramp by regular cleaning.
2. Periodically lubricate the hinges to keep them working smoothly.

MAXIMUM WEIGHT CAPACITY IS 272kg.

(weight rating is based on an evenly distributed load on 2 axles)

WARNING

- NEVER exceed the rated capacity of your ramp.
- NEVER exceed the maximum height limit stated above.
- Exercise caution when loading and unloading.
- ONLY load and unload onto a stable, level surface.
- NEVER stand in the path of equipment or vehicles. Keep bystanders and children at a safe distance.
- DO NOT leave attached ramps unattended.
- Beware of hinged areas and keep all body parts clear of them.
- Ramp surfaces may have reduced traction when wet. Use caution.
- Always fold and secure the ramp with the latches before carrying.
- The ramp should not be used for any purpose other than its intended design. It is not a toy, so do not use it as a jump, scaffold, bridge, ladder, or for any other similar purposes.
- ALWAYS ensure that the top of the ramp attaching lip is securely in place on the step, landing, or vehicle before use. We strongly recommend using the provided safety pins to prevent ramp kick-outs.
- ALWAYS follow the manufacturer's recommendations for chairs or scooters.