



## HEEVE ALUMINIUM SINGLE-FOLD PREMIUM WHEELCHAIR RAMP WITH CARRY BAG- 300KG CAPACITY

### PRODUCT INFORMATION



#### **MAXIMUM WEIGHT CAPACITY IS 300kg.**

( weight rating is based on an evenly distributed load on 2 axles)

#### **WARNING**

- NEVER exceed the rated capacity of your ramp.
- NEVER exceed the maximum height limit stated above.
- Exercise caution when loading and unloading.
- ONLY load and unload onto a stable, level surface.
- NEVER stand in the path of equipment or vehicles. Keep bystanders and children at a safe distance.
- DO NOT leave attached ramps unattended.
- Beware of hinged areas and keep all body parts clear of them.
- Ramp surfaces may have reduced traction when wet. Use caution.
- Always fold and secure the ramp with the latches before carrying.
- The ramp should not be used for any purpose other than its intended design. It is not a toy, so do not use it as a jump, scaffold, bridge, ladder, or for any other similar purposes.
- ALWAYS ensure that the top of the ramp attaching lip is securely in place on the step, landing, or vehicle before use. We strongly recommend using the provided safety pins to prevent ramp kick-outs.
- ALWAYS follow the manufacturer's recommendations for chairs or scooters.