



HEEVE HEAVY DUTY SINGLE FOLD EXTRA WIDE ALUMINIUM WHEELCHAIR RAMP

PRODUCT INFORMATION



MAXIMUM WEIGHT CAPACITY IS 400KG

(weight rating is based on an evenly distributed load on 2 axles)

WARNING

- NEVER exceed the rated capacity of your ramp.
- NEVER exceed the maximum height limit listed above.
- Use caution at all times when loading and unloading.
- ONLY load and unload onto a stable, level surface.
- NEVER stand in the path of equipment or vehicles. Keep bystanders and children away.
- DO NOT leave attached ramps unattended.
- Beware of hinged areas. Keep all body parts clear of hinged areas.
- Ramp surfaces will have less traction when wet. Use caution.
- Always fold and secure the ramp with the latches before carrying.
- The ramp is not a toy. DO NOT use the ramp as a jump, scaffold, bridge, ladder or anything other than its designed purpose.
- ALWAYS make sure the top of the ramp attaching lip is secured onto the step, landing or vehicle before using. We strongly recommend using the supplied safety pins to prevent ramp kick-outs.
- ALWAYS follow manufacturer recommendations for chairs or scooters.