


Easy Fit Instructions For Advance Dynamic ROM[®] Knee Orthosis (Extension)

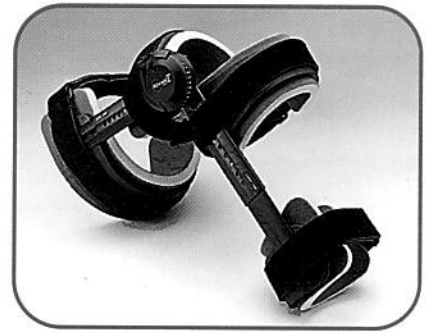

BEFORE APPLYING ORTHOSIS:

- Check leg for any neurological or vascular changes.
- Check device to be sure that lock and lock safety are engaged.

TO APPLY ORTHOSIS:

- Unfasten or loosen all straps.
- Slide device onto leg with strut placed on outside of limb and tension spring lined up at the knee joint.
- Align the struts by following the long bones of the leg.
- Fasten and tighten straps (outer straps first followed by middle straps). Straps should be snug enough for one finger to be inserted between the strap and skin.

FOR DYNAMIC TENSION:


- Swing lock lever safety away from lock lever.
- Disengage lock lever by slightly flexing leg.

TO INSPECT THE FIT:

- Check alignment and adjust straps for comfort.
- Check for any neurological or vascular changes.

TO REMOVE ORTHOSIS:

- Lock device into resident's present position (angle) by engaging lock lever safety with lock lever.
- Unfasten/loosen straps and carefully slide orthosis off leg.

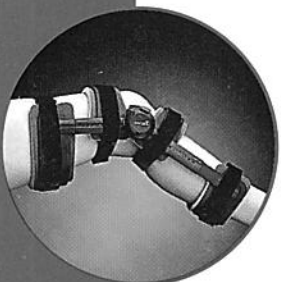

Empi[®]
ADVANCE
DYNAMIC ROM[®]