Instructions for Use

CTI[®] MISSION
KNEE BRACE

Össur Americas

27051 Towne Centre Drive
Foothill Ranch, CA 92610, USA
Tel: +1 (949) 382 3883
Tel: +1 800 233 6263
ossurusa@ossur.com

Össur Canada

2150 – 6900 Graybar Road
Richmond, BC
V6W OA5 , Canada
Tel: +1 604 241 8152

Össur Europe BV

De Schakel 70
5651 GH Eindhoven
The Netherlands
Tel: +800 3539 3668
Tel: +31 499 462840
info-europe@ossur.com

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Össur Europe B.V.


De Schakel 70
5651 GH Eindhoven
The Netherlands

Wellkang Ltd (www.CE-marking.eu)
The Black Church, St. Mary's Place
Dublin 7, D07 P4AX, Ireland

EN		Instructions for Use	10
DE		Gebrauchsanweisung	13
FR		Notice d'utilisation	16
ES		Instrucciones para el uso	19
IT		Istruzioni per l'uso	22
NO		Bruksanvisning	25
DA		Brugsanvisning	28
SV		Bruksanvisning	31
FI		Käyttöohjeet	34
NL		Gebruiksaanwijzing	37
PT		Instruções de Utilização	40
PO		Instrukcja Obsługi	43
JA		取扱説明書	46


1

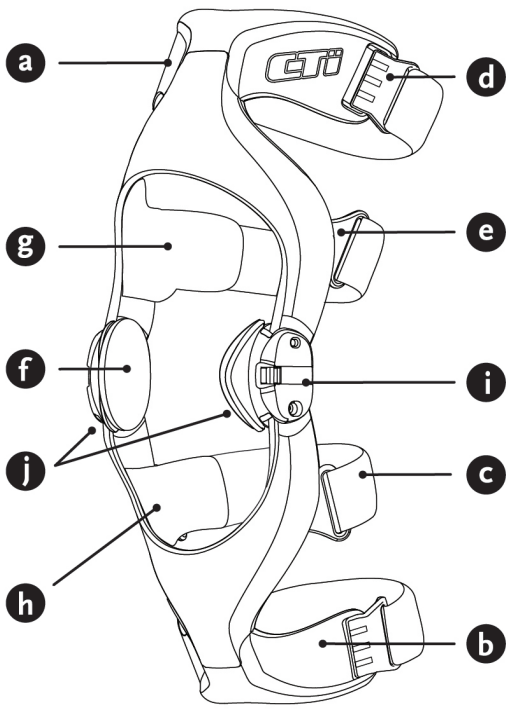
k

l

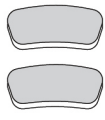
m







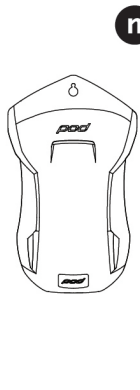
n







o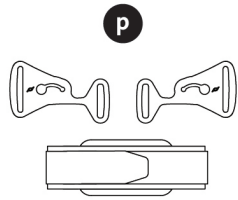

p

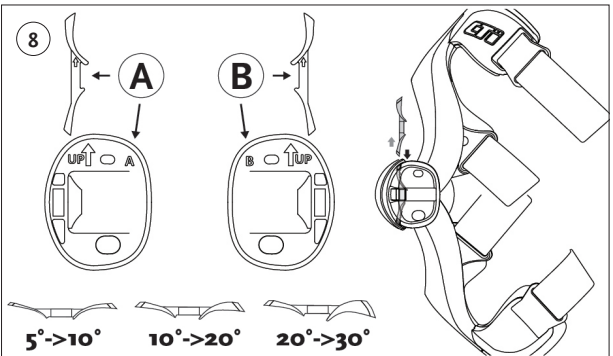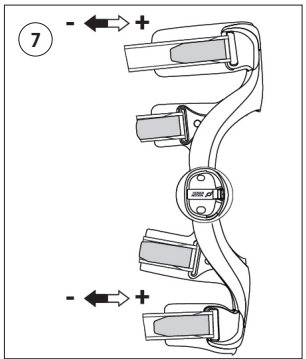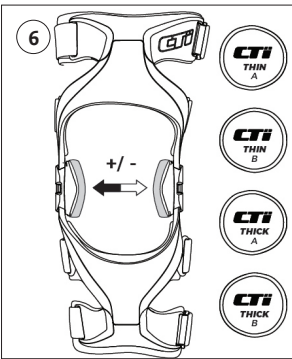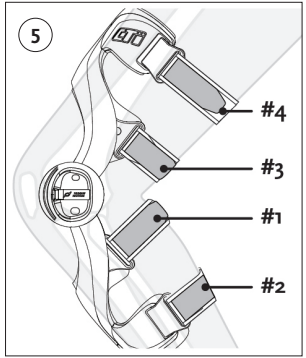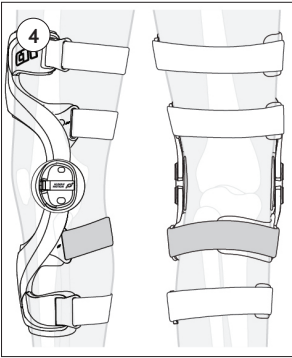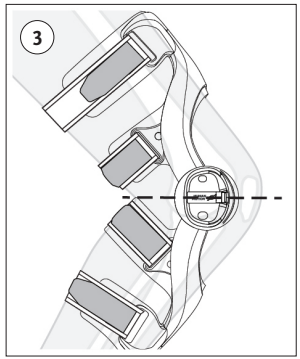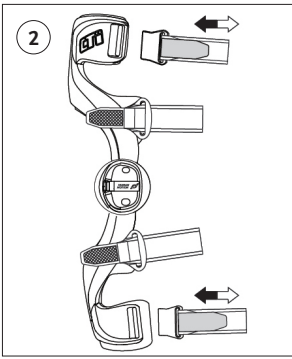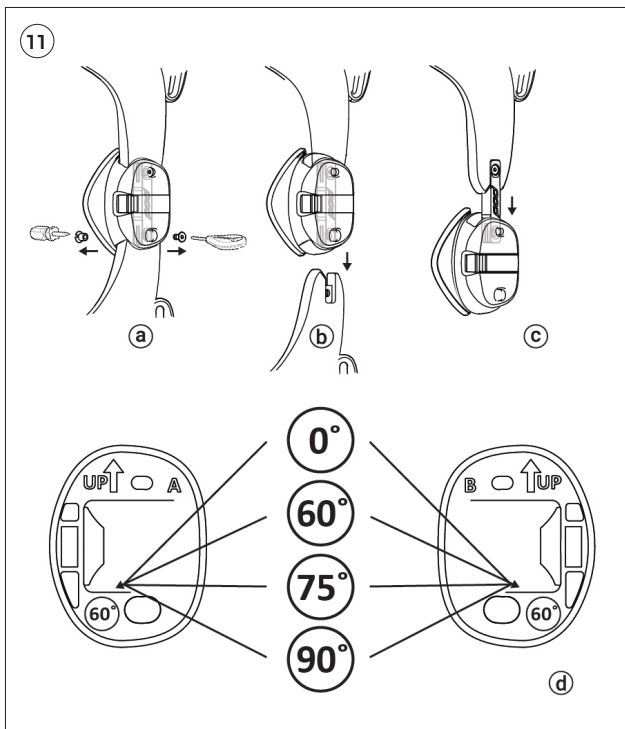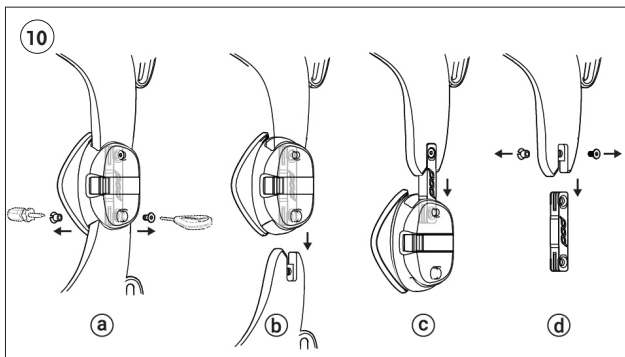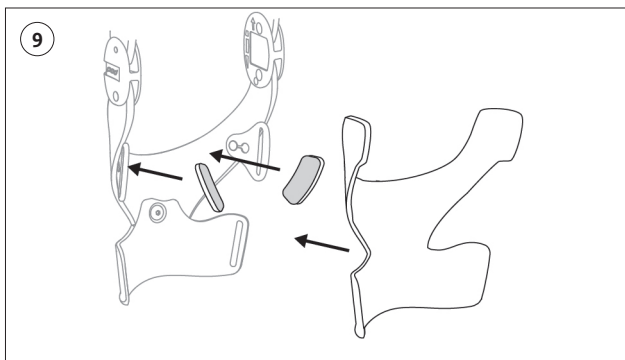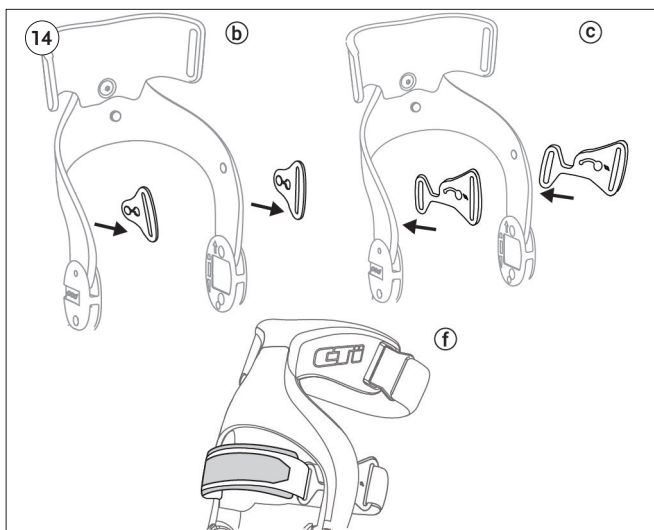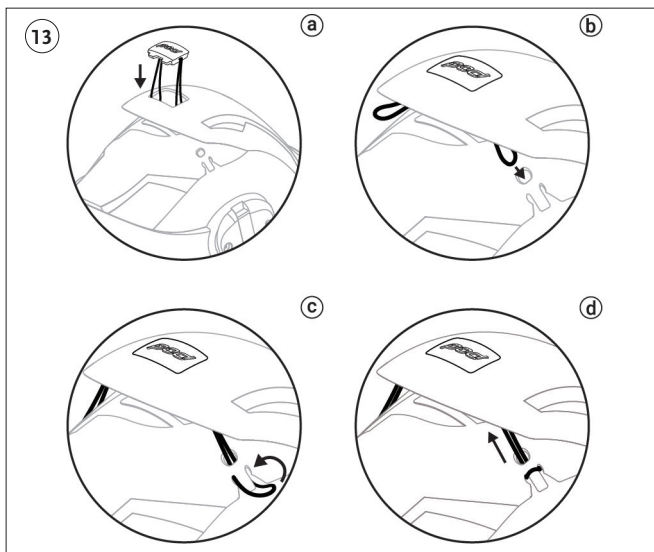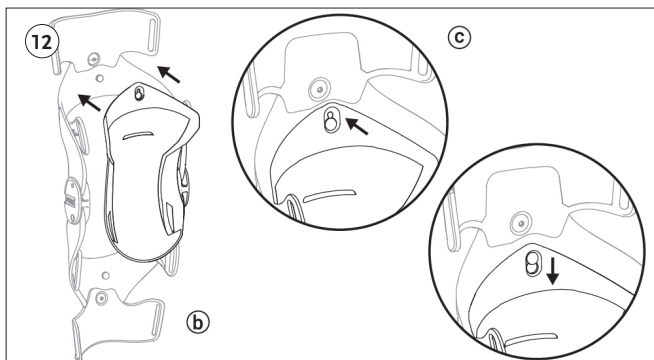INSTRUCTIONS FOR USE

Before using this device, please read the following instructions completely and carefully.

INDICATIONS FOR USE

Rotary and combined instabilities.
 Anterior Cruciate Ligament instability.
 Posterior Cruciate Ligament instability.
 Medial Collateral Ligament instability.
 Lateral Collateral Ligament instability.
 Prophylactic use.

CONTRAINDICATIONS

None known.

INTENDED USE:

The device is intended to support the knee during low to high impact/contact activities. The CTi Mission brace is side (left and right) and size specific. For maximum comfort and support wear a knee sleeve and ensure the product fits securely.

LIST OF PARTS (Figure 1):

a. Upper cuff	i. Hinge housing
b. Lower cuff	j. Knee gripper x2
c. Strap x4	k. Extension control x6
d. Strap clip x2	l. Synthetic ligament x2
e. Strap loop x4	m. Tibial wedge pad x2
f. Condyle pad	n. Guard set (sold separately)
g. Upper frame liner	o. Hinge flexion set x4 (sold separately)
h. Lower frame liner	p. PCL kit (sold separately)

WARNING

This knee brace is a supportive brace only and is not intended or guaranteed to prevent knee injury. The user agrees to the following:

1. Össur accepts no responsibility for any injury which might occur while using POD knee bracing products distributed by Össur.
2. With use of this type of device, it is possible to receive various types of skin irritation, including rashes and pressure points. If you experience any unusual reactions while using this product, consult your health care provider immediately.
3. If you have any questions as to how this knee brace applies to your own specific knee condition, consult your health care provider.
4. If you experience any pain, swelling, sensation changes or any unusual reactions while using this product, consult your health care provider.

If you do not agree to the above conditions, contact your brace provider before using this product.

BRACE APPLICATION

Correct application is vital to the proper functioning of the device. While seated and with your knee at an angle of 90°:

1. Unfasten all four straps from the lateral/outer side of the brace (**Figure 2**).
2. Place the brace over the knee so that the hinge is located against the side of the knee joint above the mid patella line (**Figure 3**).
3. Fit Strap #1 (directly below the knee) first as this strap is located above the calf muscle and aids in the correct positioning of the brace (**Figure 4**).
4. Fit remaining straps in numbered order (**Figure 5**).

ADJUSTMENT

Use any combination of the Condyle pads supplied to widen or narrow the width of the brace at the knee (**Figure 6**). Control the strap length by adjusting the medial/inside hook-and-loop strap end (**Figure 7**). The hinge absorbs shocks and forces on the knee, with extension stops available for 5–10°, 10–20° and 20–30°. Match the letter on the extension stop insert with the letter on the inside of the hinge for the correct fit (**Figure 8**).

Tibial wedge pads can be installed to fill in any gaps around the tibia (**Figure 9**) and achieve the desired fit. To add a wedge pad, pull the lower frame liner away from the brace frame and connect the wedge pad as needed (medial, lateral or both) to the liner using the hook fastener. Fine tune the wedge pad position until your brace fits snugly against the tibia.

LIGAMENT REPLACEMENT

To remove ligament for replacement (**Figure 10**):

- a. First remove the bottom screw off the hinge using the provided tool.
- b. Pull out the lower frame.
- c. Slide down the hinge housing.
- d. Remove upper screw to set the ligament free. Replace it.

To reassemble, just repeat the steps in opposite order (steps d-a).

HINGE FLEXION SET INSTALLATION

The Hinge flexion set (sold separately) is available for varying degrees of flexion control. To install the Hinge flexion set (**Figure 11**):

- a. First remove the bottom screw off the hinge using the provided tool.
- b. Pull down the lower frame.
- c. Slide down the hinge housing.
- d. Select the required pair of Hinge flexion housing. The angle of the housing is marked on the back. (**Figure 11-d**).

To reassemble, just repeat the steps in opposite order (steps c-a).

GUARD SET INSTALLATION

Guard sets (sold separately) are available to protect the knee against impacts. To install the Guard set (**Figure 12**):

- a. Remove the upper and lower frame liner from the brace.
- b. Position the upper guard over the frame pin in the upper frame.
- c. Pull down so it clips into place.
- d. Repeat steps b & c with the lower guard.
- e. Replace the upper and lower frame liner into the brace.

To replace the Guard set elastic cord (sold separately) or connect the upper and lower panel of the guard set (**Figure 13**):

- a. Install the Guard set elastic cord into the upper guard.
- b. Push the elastic cord through the hole in the lower guard.
- c. Pull the elastic cord through the hole and over the post on the side of

the lower guard.

- d. Pull the elastic cord to tighten the fit on the lower panel.
- e. Repeat steps b to d on the other elastic cord.

PCL KIT INSTALLATION

PCL kits (sold separately) are available to increase the support for the Posterior Cruciate Ligament. To install the PCL kit (**Figure 14**):

- a. Remove the upper frame liner and the #3 strap from the brace.
- b. Remove the lateral upper strap loop from the frame pin.
- c. Orientate the PCL strap loop as shown. Position the PCL strap loop over the lateral upper frame pin and pull rearward so it clips into place.
- d. Repeat steps b & c with the medial upper strap loop.
- e. Replace the upper frame liner and the #3 strap into the brace.
- f. Loosely install the thigh strap with the thigh pad into the PCL strap loops.
- g. Apply the brace
- h. Position the thigh pad so it will sit centrally on the thigh.
- i. Tighten the thigh strap with the leg in full extension so that the thigh strap and pad are comfortably snug.

CARE INSTRUCTIONS

Hand wash using mild detergent and water. Air dry only. Inspect the brace for obvious signs of wear and fatigue. If brace is used in salt water or chlorinated water, rinse well and air dry.

IMPORTANT NOTICE TO USERS AND/OR PATIENTS ESTABLISHED IN EUROPE:

The user and/or patient must report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

DISPOSAL

All components of the product and packaging should be disposed of in accordance with respective national environmental regulations. Users should contact their local governmental office for information on how these items can be disposed of in an environmentally sound manner.

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