

Donning instruction:

⚠ • Ensure hands are clean and/or wear appropriate gloves while handling the mask. ⚠

Remove the respirator (mask) from its bag, inspect it for damage prior to use.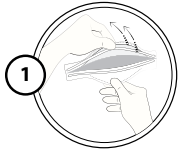

1. Hold respirator (mask) in one hand. Separate top flap (panel) from bottom.

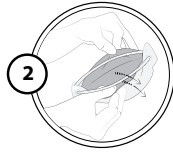2. Holding the top flap with one hand (panel), separate the bottom flap from it to form a 3D shape.

3. Bring the cupped respirator towards face, open side towards the face. Holding the bottom flap under the chin, place the nosepiece (on the top flap) on the nose, while the two head straps hang free.

4. While holding the mask on face with one hand, use the other hand to bring the upper head strap across the crown of the head.

5. Continue to hold the mask on face and use the other hand to bring the lower head strap across the crown of the head, under the ears.

6. Make sure that head straps are not twisted and follow the diagram to ensure that they are secured correctly.

7. Using both hands, pinch the nosepiece until it falls in shape onto the nose bridge to ensure a good fit and seal. Ensure that the nosepiece sits symmetrically on to the nose. If not, repeat pinching the nosepiece to re-adjust the nosepiece until it is secured firmly.

8. Using both hands to cover the front of the respirator, breathe in and out sharply to check if air leaks out around the nose and/or at the edges. If air leaks, adjust the nosepiece and head straps

• Repeat step 8 until a proper fitting is achieved.

DoFFing instruction:

1. Without touching the respirator, hold the upper head strap and remove it from the head.

2. Without touching the respirator, hold the lower head strap and remove it from the head.

3. Holding one of the head straps, without touching the respirator cup, dispose the respirator in accordance with local regulations.

Storage and transportation:

- Store in accordance with the instructions provided on the packaging.
- Use original packaging provided when storing/transporting.
- Respirators shall be stored in a manner that will protect them against dust, ozone, sunlight, heat, extreme cold, excessive moisture, vermin, damaging chemicals, oils, greases, and/or any other potential hazard that can have a detrimental effect on the respirator.
- Respirators shall be stored in a manner that will prevent deformation and/or physical damage.

Cautions and Limitations

- Before using the respirator, the wearer shall read and understand all the information provided.
- Check that the respirator is clean and in good condition before each use.
- Always check the manufacturing date of the respirator to make sure that the product is not exceeding 5-year shelf life.
- This respirator is not recommended to be used with beards or other facial hair that may compromise contact/seal between the product and skin of the face.
- Temples of eyeglasses or any other materials such as hair, cloth, tissue, straps, or jewellery shall not come between the skin and the sealing surface of the respirator or interfere with the operation of the respirator according to CAN/CSA Z94.4 guidelines.
- Do not use in oxygen-deficient atmospheres. CAN/CSA Z94.4 defines oxygen-deficient atmosphere as those where oxygen concentration is confirmed to be below 19.5 % (at sea level).
- Replace the respirator with a new one if it becomes damaged or breathing resistance becomes excessive and/or intolerable.
- If a person wearing this respirator develops any condition that leads to compromising the respirator seal to the face, the person shall restore the required fitting in a non-hazardous environment.
- This respirator shall not be used in conditions and/or with atmospheric contaminations which are considered Immediately Dangerous to Life or Health (IDLH). IDLH (determined by CAN/CSA Z94.4) is an atmosphere that poses an immediate threat to life or that will cause irreversible adverse health effects or impair an individual's ability to escape.
- This respirator shall not be altered, modified, repaired, or cleaned.
- Do NOT use this respirator for oil-based particles.
- Do NOT use this respirator in conditions where particle concentrations exceed the maximum allowable limit as per the applicable regulation and standard. The maximum allowable concentration is obtained by multiplying Assigned Protection Factor (APF*) and the Permissible Occupational Exposure Limit (OEL) of the contaminant. This type of disposable respirator is assigned an APF of 10 (APF=10), which means it helps protect against certain particles in concentration up to 10 × OEL.

* APF is the anticipated level of respiratory protection that would be provided by a properly functioning respirator or class of respirators to properly fitted and trained users and is used by regulators and standards development organizations, such as the U.S. National Institute of Occupational Safety and Health (NIOSH) and Canadian Standards Association (CAN/CSA Z94.4 Selection, use, and care of respirators).

Warning:

The user of respirator shall check the seal of the product immediately after donning.

Warning:

Do NOT wear a respirator that cannot pass the seal checks successfully.

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