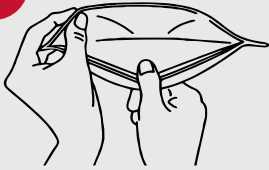

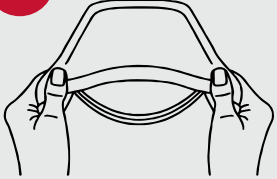


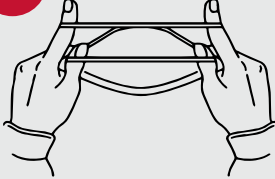
R10 N95 POUCH-STYLE, NON-STERILE RESPIRATOR


Directions for Application


- 


1. Separate the edges of the respirator to fully open it.
- 


2. Slightly bend the nose wire to form a gentle curve.
- 


3. Hold the respirator upside down to expose the two headbands.
- 


4. Using your index fingers and thumbs, separate the two headbands.
- 

5. While holding the headbands with your index fingers and thumbs, cup the respirator under your chin.
- 

6. Pull the headbands up and over your head.
- 

7. Release the lower headband from your thumbs and position it at the base of your neck.
- 

8. Position the remaining headband on the crown of your head.
- 

9. Conform the nose piece across the bridge of your nose by firmly pressing down with your fingers.
- 

10. Continue to adjust the respirator and secure the edges until you feel you have achieved a good facial fit. Now, perform a Fit Check.