



Instructions for Obtaining Ear Impressions

If you are going to order a “custom” ADEL™ Drum earbud from Asius Technologies, you will need to provide a full shell impression of both your ears.

THIS REQUIRES THE ASSISTANCE OF A LICENSED AUDIOLOGIST; PRINT THIS PAGE AND GIVE TO THE AUDIOLOGIST DURING YOUR VISIT.

REQUIREMENTS

The impressions should display the full contour of both the outer ear and the ear canal, to a sufficient depth. The photos below are a perfect example of excellent ear impressions:



Each ear impression must include the full *helix*, *crus* of the helix, *tragus* and *antitragus*. Your audiologist will know what those are.

IMPORTANT:

1. The impression must be taken just past the second bend of the ear canal.
2. The material utilized to shoot the impression must be made of high-viscosity silicone.
3. The impression must be taken with the client's mouth *open*. A 1-2" bite block should be used. Utilizing this method will reduce movement and will result in a more secure fitment as well as an optimum seal for maximum performance from your ADEL™ Drum earbuds.

If you have any questions at all or are unsure if your impressions meet the requirements, please send detailed photos to our support team at info@asiustechnologies.com.

Once you receive your impressions, please ship them to the following address:

Asius Technologies
1257 Whitehall Drive
Boulder, Colorado 80504