



H₂O Shield™

PICC Line Shower Cover

DIAGRAM OF THE H₂O SHIELD™

See below diagram. For external use on top of an adhesive dressing only.

TOP VIEW

- ① Neoprene seal
- ② Arm opening

SIDE VIEW

- ③ Neoprene seal
- ④ Logo and size
- ⑤ PVC body


CARE

Hand wash with a mild disinfectant or detergent.


If water gets inside the protector while cleaning, use a clean towel to dry.

Store in a dry place at room temperature.

TOP VIEW

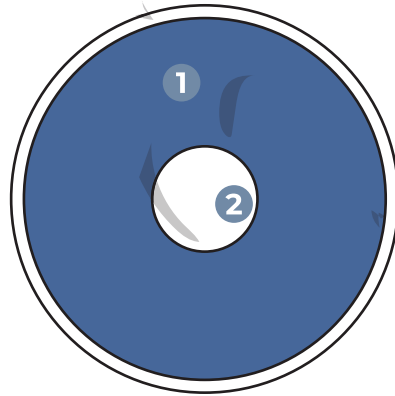

SIDE VIEW

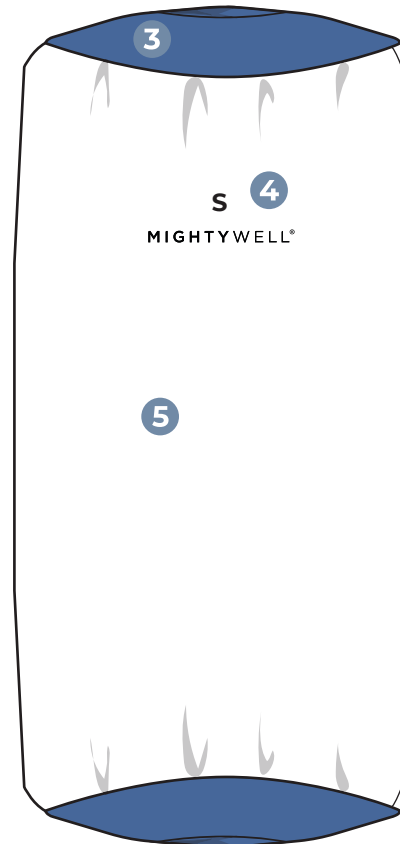

ABOUT

H₂O Shield™ provides reliable and comfortable water protection for PICCs, wounds, casts, and dressings so users can shower with confidence.

SIZE GUIDE

Measure the arm circumference of the area being protected with a flexible tape measure, and find the recommended size below.

RANGE	SIZE
8" - 12" _____	S/M
12" - 20" _____	M/L

PRECAUTIONS

Do not fully submerge your arm underwater.

Do not fold or pull down the H₂O Shield™ with force or quick movement.

The H₂O Shield™ should be worn for a **max of 20 minutes** to avoid trapped warm air causing perspiration.



H₂O Shield™

PICC Line Shower Cover

INSTRUCTIONS FOR USE

See below instructions of use for the H₂O Shield™. H₂O Shield™ provides an easy one-handed application design for users to apply themselves.



To apply: Stretch the upper navy neoprene seal with the logo facing up, and pull H₂O Shield™ over the arm to cover their device, bandage, or injury completely.



Confirm that the seal of the H₂O Shield™ lays flat against the skin and is wrinkle-free with a snug, comfortable fit to ensure waterproof protection.



Once secured, feel free to shower and move as needed.

Not designed to be submerged. Not recommended for bathing, swimming, or recreational water activities.



To remove: carefully stretch the upper neoprene seal and gently pull H₂O Shield™ down so it does not tug on the device, bandage, or injury.



Hang the H₂O Shield™ to dry, away from any heat source.



FOR MORE INFO

Scan the QR code for a demo video of the H₂O Shield™ PICC Line Shower Cover.