



H₂O Shield™

Waterproof Cast Cover

DIAGRAM OF THE H₂O SHIELD™

Review the below diagram before using the H₂O Shield™ Waterproof Cast Cover.

TOP VIEW

- ① Neoprene seal
- ② Arm opening

SIDE VIEW

- ③ Neoprene seal
- ④ Logo and size
- ⑤ PVC body
- ⑥ Thumb placement


CARE

Hand wash with a mild disinfectant or detergent.


If water gets inside the protector while cleaning, use a clean towel to dry.

Store in a dry place at room temperature.

TOP VIEW

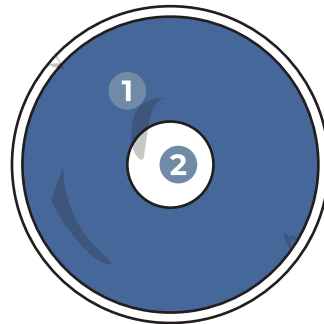

SIDE VIEW

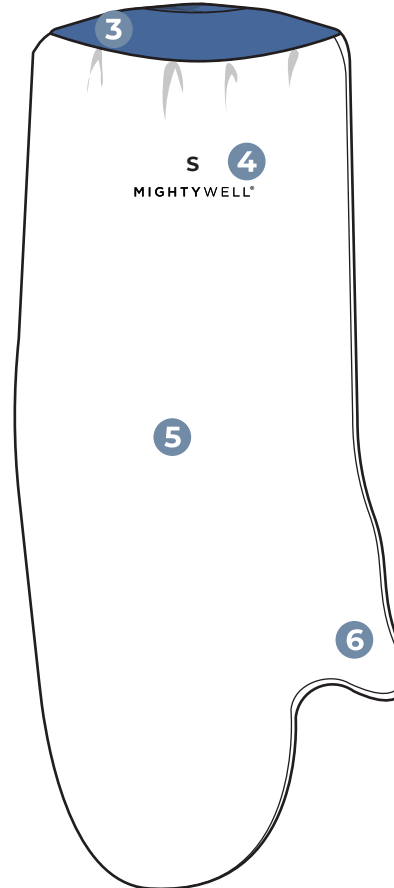

ABOUT

H₂O Shield™ provides reliable and comfortable water protection for wounds, casts, and dressings so users can shower with confidence.

SIZE GUIDE

Measure the arm circumference of the area being protected with a flexible tape measure, and find the recommended size below.

RANGE	SIZE
8" - 12" (22.5" L)	_____ S/M
12" - 20" (24.5" L)	_____ M/L

PRECAUTIONS

Designed for showering or bathing. However **do not** fully submerge the blue neoprene seal underwater.

Do not fold or pull down the H₂O Shield™ with force or quick movement.

The H₂O Shield™ should be worn for a **max of 20 minutes** to avoid trapped warm air causing perspiration.



H₂O Shield™

Waterproof Cast Cover

INSTRUCTIONS FOR USE

The H₂O Shield™ Waterproof Cast Cover provides a secure and comfortable watertight seal that keeps water out while allowing the user to move freely. Its easy-to-use design allows for a quick and hassle-free application.



1
To apply: Stretch the navy neoprene seal and pull the H₂O Shield™ with the logo facing up, over the arm to directly cover the device, bandage, or cast.



2
Confirm that the seal of the H₂O Shield™ lays flat against the skin and is wrinkle-free with a snug, comfortable fit to ensure waterproof protection.



3
Once secure over the area, feel free to move about as needed in the shower or bath.



4
To remove: Stretch the upper neoprene seal and pull H₂O Shield™ down the arm so it does not tug on the device, bandage, or cast.



5
Hang the H₂O Shield™ to dry, away from any heat source.

Not recommended for swimming, or recreational water activities.



FOR
MORE
INFORMATION

Scan the QR code for a demo video of the H₂O Shield™ Waterproof Cast Cover.