


SETTING HAND CONTROL MEMORY POSITIONS – MOJO HAND CONTROL

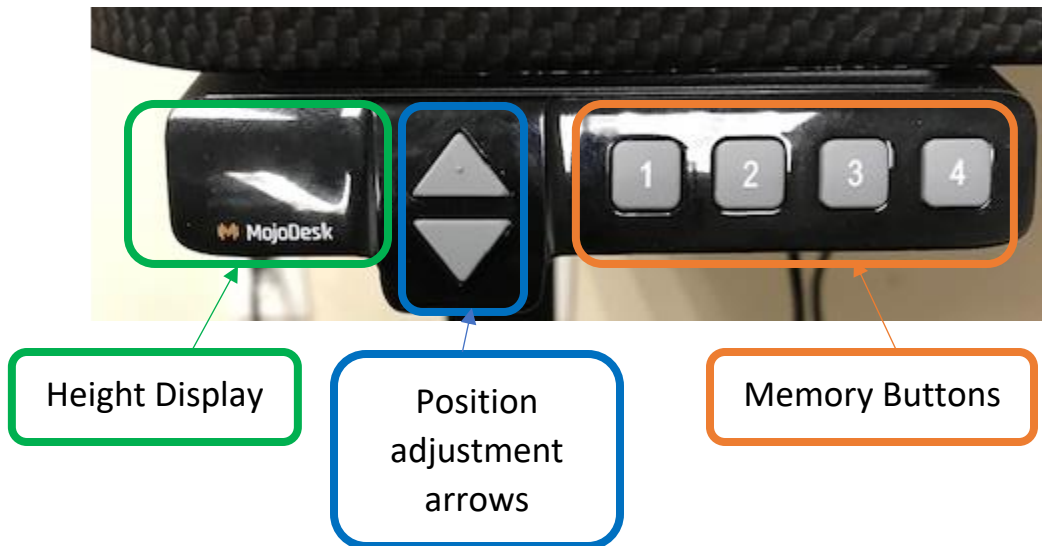

1. Using the Arrow buttons raise and lower the table to the desired height
2. Press and hold down the memory button you wish to set and the down arrow button at the same time until P1, P2, P3, or P4 appears on the screen.
3. The position is now set
4. To raise or lower the table to a preset height, hold the preset memory button and release. The table will automatically raise or lower to the preset position
5. To reset the hand control, hold down both arrow buttons. While holding down the arrow buttons the table will begin to lower. Continue holding the arrow buttons until the table reaches its lowest height and beeps. Once you hear a beep the table is reset.

SETTING HAND CONTROL MEMORY POSITIONS – MOJO HAND CONTROL



1. Raise or lower the table to the desired height.
2. Press and hold down the memory button you wish to set and the down arrow at the same time until P1 or P2 or P3 or P4 appears on the screen
3. The position is now set.
4. To raise or lower the table to a preset height, hold the preset button until the programmed height is achieved.