



## Detachment

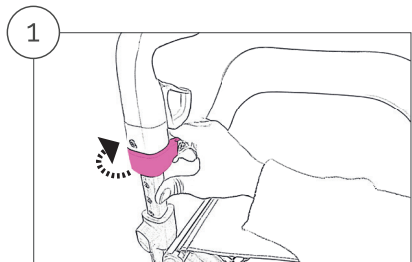

1-2. Grab the mounting clip and turn it clockwise until it releases from the handle bar. Repeat the same steps on the right side, but turn the mounting clip counter-clockwise until it releases from the handle bar.

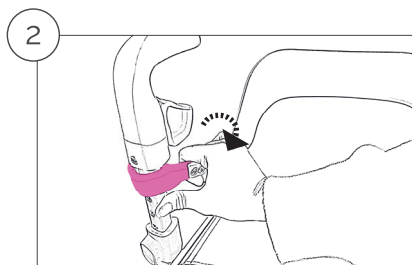

## Technical specifications

Material: PE

Height: 150 mm (from top to bottom)

Length: 850 mm (Regular Track), 890 mm (Wide Track)

Depth: 3 mm

Weight: 200g

## Maintenance

Can be cleaned with a wet cloth. If necessary, it can also be cleaned with washing-up liquid.

## Video version

Would you rather watch a video version of this manual? You can either scan the QR code or go to [bit.ly/backrestmanual](https://bit.ly/backrestmanual) to see a video demonstration of how to attach the backrest.



by **ACRE**

Designed in Denmark. Handcrafted in China.

byACRE ApS, I.C. Modewegsvej 1, 2800 Lyngby, DK

# Backrest

SUPPORTING YOUR LOWER BACK WHILE SEATED

CARBON ULTRALIGHT ACCESSORY



Congratulations with your new backrest!

Please read this user manual thoroughly before you start using the backrest. We have included some tips that will help you to quickly learn how to attach the backrest.

ENGLISH

by **ACRE**

# Use of the backrest

The backrest is easily attached in any optional hole on the handle bar, except the lowest hole. The curved shape ensures optimal support in the lower back, when you sit on the rollator. The backrest is attached in the opposite direction as you walk, so you can sit on the part of the rollator where the seat is widest and get support for your back on the part where the seat is most narrow.


- Be aware that the backrest is attached correctly before using it. The clips on the backrest should be completely secured on the rollator's left and right handle bar without being able to be moved to the side or up and down.
- Be aware that the backrest is facing correctly (on the opposite side of where you walk).
- Be aware that you need at least one visible hole on the handle bar in order to attach the backrest.

Tip!

You can adjust the backrest to the height that suits you best by either:


- 1) Attach it in the chosen height or
- 2) Attach the backrest in the top hole and subsequently adjust the height on the handles up or down. Try to adjust the height a bit up and down to see what works best for you.

## Attachment - left side

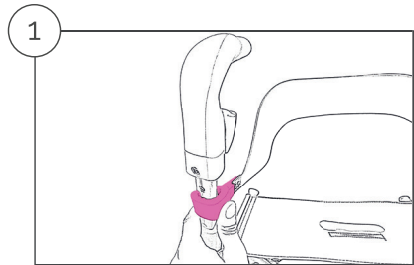


1. Attach the mounting clip on the backrest to the handle bar on the rollator. Hold the clip as shown on the illustration. Push the clip towards the handle bar until it is attached around the bar. Be aware that the clips can still slide up and down the bar at this point.

Tip! The clip is easiest to attach if you push it onto the handle bar from the side facing backwards and inwards the seat.

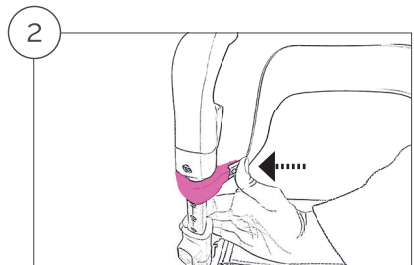

2. Find the preferred height and turn the clip counter clockwise until you hear a 'click'. Be aware that the small peak on the inside of the clips has to fit into one of the holes on the handle bar.

## Attachment - right side

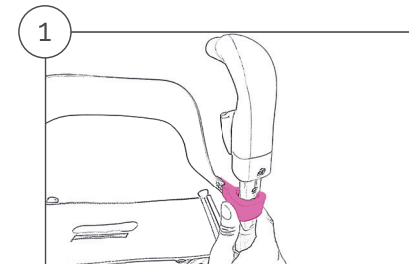

Repeat step 1 and 2 and push instead the mounting clip clockwise until you hear a 'click' and the clip is fastened.

Tip! If it is difficult attach the clips in the preferred hole, it might help to choose the top hole and subsequently adjust the handle height. If attached in the top hole the clip is moved all the way to the top of the handle bar before the clip is turned clock-wise and the clip is fastened.

