



Revision Date- 8/7/2013

HBV LONGITUDINAL PANEL

(Catalog No. 6529)

For Investigational Research Use Only

Application Notes: This 12 member panel was collected from a 20 year old Caucasian Male.

PANEL NUMBER	DATE OF DRAW	ABBOTT HBSAG S/CO	ABBOTT HBC TOTAL S/CO	ABBOTT HBC IGM S/CO	DIASORIN HBEAG S/CO	DIASORIN ANTI-HBE S/CO	ABBOTT ANTI-HBS S/CO	ORTHO ANTI-HCV ECI	ORTHO ANTI-HIV ½ ECI
6529-01	10/8/01	2.5	0.03	>4.7	1.29	0.6	0.6	0.04	0.07
6529-02	10/22/01	0.07	0.04	>4.7	0.29	0.4	0.5	0.04	0.07
6529-03	11/5/01	0.1	0.06	>4.7	0.13	0.2	0.5	0.03	0.07
6529-04	11/19/01	0.1	0.04	>4.7	0.14	0.1	0.5	0.03	0.07
6529-05	12/3/01	0.07	0.05	>4.7	0.04	0.8	0.6	0.04	0.07
6529-06	12/18/01	0.05	0.05	>4.7	0.30	0.09	0.9	0.03	0.07
6529-07	1/1/02	0.04	0.06	>4.7	0.10	0.03	2.0	0.04	0.05
6529-08	1/14/02	0.2	0.06	4.6	0.16	0.03	3.1	0.03	0.06
6529-09	1/28/02	0.1	0.06	4.3	0.10	0.03	3.5	0.03	0.06
6529-10	2/11/02	0.07	0.06	4.3	0.19	0.03	5.9	0.03	0.08
6529-11	2/25/02	0.1	0.06	3.8	0.04	0.03	5.6	0.03	0.08
6529-12	3/9/02	0.07	0.07	3.5	0.39	0.07	9.3	0.03	0.08