

EBV Seroconversion Panel

(Catalog No. EBV 9097)

Donor No. 283843421
For Research Use Only

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility. Donor was un-treated and asymptomatic throughout the collection period.

Panel Number	Donor Number	Bleed Number	Bleed Date	EA D IgM	EA D IgG	EBNA-1 IgM	EBNA-1 IgG	VCA IgM	VCA IgG	Mono Screen	ASI Titer	ASI Mono
9097-01	283843421-01	8520017029	12/18/2008	0.464	0.415	0.385	0.395	0.175	0.207	NEG	0.500	NEG
9097-02	283843421-02	8520017216	12/20/2008	0.411	0.374	0.249	0.331	0.117	0.172	NEG	0.500	NEG
9097-03	283843421-03	8520018960	1/15/2009	0.391	0.378	0.223	0.374	0.137	0.257	NEG	0.500	NEG
9097-04	283843421-04	8520019100	1/17/2009	0.299	0.429	0.219	0.387	0.116	0.214	NEG	0.500	NEG
9097-05	283843421-05	8520019667	1/24/2009	0.267	0.354	0.190	0.285	0.106	0.220	NEG	0.500	NEG
9097-06	283843421-06	8520020873	2/12/2009	2.736	1.956	0.524	0.394	2.243	1.192	POS	4.000	POS
9097-07	283843421-07	8520021550	2/21/2009	2.235	1.752	0.909	0.483	2.036	1.993	POS	4.000	POS
9097-08	283843421-08	8520021928	2/26/2009	1.717	1.715	0.776	0.510	1.598	2.092	POS	4.000	POS
9097-09	283843421-09	8520022083	2/28/2009	1.712	1.692	0.709	0.468	1.508	2.174	POS	5.000	POS
9097-10	283843421-10	8520030035	6/4/2009	0.532	0.648	0.428	3.694	0.373	2.806	POS	2.000	POS
9097-11	283843421-11	8520030257	6/6/2009	0.557	0.616	0.422	3.532	0.352	2.594	POS	2.000	POS
9097-12	283843421-12	8520030631	6/11/2009	0.519	0.615	0.402	3.643	0.342	2.679	POS	2.000	POS
9097-13	283843421-13	8520030853	6/13/2009	0.489	0.587	0.390	3.645	0.324	2.720	POS	2.000	POS
9097-14	283843421-14	8520031330	6/18/2009	0.557	0.657	0.453	3.683	0.363	2.759	POS	2.000	POS

Notes on Data: Data is presented as an S/Co ratio unless stated otherwise. Ratios >1.0 are considered to be reactive. This panel is for investigational use only and the data is offered for informational purposes only. Data is generated at ZeptoMetrix Corporation and by our Lab Partners in the United States and Europe. We will endeavor to update our data sheets whenever new test data becomes available to us. All specimens collected in this longitudinal series are naturally occurring, undiluted, unprocessed samples collected from a single plasma donor in the United States. No preservatives have been added.