

HIV Seroconversion Panel

(Catalog No. HIV 6243)

Donor No. 62238

For Research Use Only

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility. Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	BLEED DATE	ROCHE COBAS AMPLICOR HIV-1 MONITOR	ABBOTT HIV-1 p24 Ag	ABBOTT HIV-1 p24 Ag NT%	COULTER HIV-1 p24 Ag	COULTER HIV-1 p24 Ag NT%	ABBOTT HIV AB 1/2 EIA AxSYM	ABBOTT HIV AB 1/2 EIA rDNA	ABBOTT HIV AB 1/2 EIA rDNA 3rd GEN
6243-01	08/22/96	<400	0.313	NEG	0.091	NEG	0.31	0.503	0.47
6243-02	08/27/96	<400	0.266	NEG	0.091	NEG	0.30	0.508	0.36
6243-03	08/29/96	<400	0.344	NEG	0.076	NEG	0.28	0.518	0.48
6243-04	09/03/96	<400	0.328	NEG	0.091	NEG	0.31	0.452	0.43
6243-05	09/09/96	1,180	0.313	NEG	0.076	NEG	0.33	0.457	0.43
6243-06	09/11/96	3,930	0.297	NEG	0.091	NEG	0.33	0.472	0.44
6243-07	09/16/96	265,000	0.734	NEG (83)	1.424	98.4	0.38	0.482	0.47
6243-08	09/18/96	413,000	0.938	83.0	1.348	98.6	0.44	0.533	0.53
6243-09	09/23/96	295,000	0.828	NEG (77.6)	0.970	98.0	4.81	3.553	4.39
6243-10	09/25/96	447,000	1.047	80.3	1.667	98.3	5.75	3.548	4.33

All specimens tested for HBsAg and Anti-HCV and found to be negative by an FDA approved test.

Notes on Data: Data is presented as an S/Co ratio unless stated otherwise. Ratios >1.0 are considered to be reactive. This panel is for investigational use only and the data is offered for informational purposes only. Data is generated at ZeptoMetrix Corporation and by our Lab Partners in the United States and Europe. We will endeavor to update our data sheets whenever new test data becomes available to us. All specimens collected in this longitudinal series are naturally occurring, undiluted, unprocessed samples collected from a single plasma donor in the United States. No Preservatives have been added.

HIV Seroconversion Panel

(Catalog No. HIV 6243)

Donor No. 62238

For Research Use Only

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility. Donor was un-treated and asymptomatic throughout the collection period.

New 3rd Gen

New 4th Gen

PANEL MEMBER	ABBOTT MUREX HIV Ab 1/2 + O	ABBOTT PRISM ANTI HIV 1/2	BIOMERIEUX ORGANON TEKNIKA ANTI HIV-1	DADE BEHRING ENZYGNOST ANTI HIV 1/2 PLUS	GENETICS SYSTEMS HIV 1/2 EIA	CAMBRIDGE BIOTECH HIV-1 WESTERN BLOT	ABBOTT PRISM HIV Ag/Ab COMBO	ABBOTT MUREX HIV-1 Ab/Ag COMBO	ABBOTT HIV Ag/Ab COMBO ARCHITECT
6243-01	0.289	0.21	0.410	0.12	0.107	NEG	0.09	0.230	0.07
6243-02	0.308	0.20	0.403	0.09	0.121	p66	0.11	0.235	0.08
6243-03	0.334	0.24	0.423	0.09	0.096	p66	0.09	0.235	0.06
6243-04	0.348	0.24	0.444	0.10	0.085	p66	0.08	0.240	0.08
6243-05	0.423	0.22	0.405	0.10	0.121	p66	0.08	0.265	0.08
6243-06	0.370	0.22	0.405	0.09	0.081	p66	0.09	0.295	0.15
6243-07	0.361	0.27	0.483	0.11	0.143	p66	0.68	1.395	1.439
6243-08	1.020	0.24	0.394	0.15	0.088	p66	0.96	1.920	1.900
6243-09	11.298	4.41	0.451	3.98	0.132	p66	2.55	13.105	14.876
6243-10	11.462	8.13	0.641	5.18	0.235	INDETERMINATE	2.98	16.240	16.350

All specimens tested for HBsAg and Anti-HCV and found to be negative by an FDA approved test.

Notes on Data: Data is presented as an S/Co ratio unless stated otherwise. Ratios >1.0 are considered to be reactive. This panel is for investigational use only and the data is offered for informational purposes only. Data is generated at ZeptoMetrix Corporation and by our Lab Partners in the United States and Europe. We will endeavor to update our data sheets whenever new test data becomes available to us. All specimens collected in this longitudinal series are naturally occurring, undiluted, unprocessed samples collected from a single plasma donor in the United States. No Preservatives have been added.