

HIV Seroconversion Panel

(Catalog No. HIV 6248)

Donor No. 63331

For Research Use Only

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility. Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	BLEED DATE	CHIRON HIV-1 bDNA	ROCHE COBAS AMPLICOR HIV-1 MONITOR	ABBOTT HIV-1 p24 Ag NT%	COULTER HIV-1 p24 Ag	COULTER HIV-1 p24 Ag NT%	ABBOTT HIV AB 1/2 EIA AxSYM	ABBOTT HIV AB 1/2 EIA rDNA	New 3 rd Gen		
									ABBOTT MUREX HIV Ab 1/2 + O	BIOMERIEUX ORGANON TEKNIKA ANTI HIV-1	BIOTEST ANTI HIV 1/2
6248-01	02/12/97	<100	<400	NEG	0.275	0.072	0.46	0.195	0.384	0.27	0.16
6248-02	02/16/97	<500	<400	NEG	0.245	0.101	0.44	0.136	0.410	0.30	0.15
6248-03	02/19/97	<500	<400	NEG	0.245	0.072	0.38	0.059	0.466	0.29	0.14
6248-04	02/23/97	<500	<400	NEG	0.245	0.072	0.38	0.093	0.370	0.30	0.17
6248-05	02/26/97	561	2,300	NEG	0.265	0.072	0.39	0.085	0.354	0.39	0.18
6248-06	03/02/97	74,220	199,000	1.17	2.696	2.899	0.34	0.102	0.325	0.41	0.17
6248-07	03/09/97	>800,000	87,700,000	>86.3	38.255	36.232	1.35	1.508	5.230	1.08	0.34

New 4th Gen

PANEL MEMBER	BOEHRINGER MANNHEIM ENZYMNUTEST ANTI HIV 1/2 PLUS SUBTYPE O	DADE BEHRING ENZYGNOST ANTI HIV 1/2 PLUS	GENETICS SYSTEMS rLAV EIA	ORTHO ANTI HIV 1/2 ENHANCED ELISA	SANOVI GENSREEN ANTI HIV 1/2	WALDHEIM DIAGNOSTICA ANTI HIV-1	CAMBRIDGE BIOTECH HIV-1 WESTERN BLOT	ABBOTT PRISM HIV Ag/Ab COMBO	ABBOTT MUREX HIV-1 Ab/Ag COMBO	ABBOTT HIV Ag/Ab COMBO ARCHITECT
6248-01	0.36	0.19	0.097	0.02	0.18	NEG	NEG	0.08	0.230	0.12
6248-02	0.36	0.16	0.224	0.02	0.14	NEG	NEG	0.11	0.260	0.15
6248-03	0.37	0.20	0.173	0.02	0.19	NEG	NEG	0.21	0.235	0.12
6248-04	0.37	0.29	0.134	0.02	0.19	NEG	NEG	0.11	0.235	0.12
6248-05	0.36	0.28	0.108	0.02	0.16	NEG	NEG	0.10	0.240	0.11
6248-06	0.64	0.16	0.134	0.02	0.19	NEG	NEG	2.01	1.670	1.855
6248-07	1.14	0.69	0.097	0.03	15.75	NEG	NEG	293.16	17.990	464.579

All specimens tested for HBsAg and Anti-HCV and found to be negative by an FDA approved test.

Notes on Data: Data is presented as an S/Co ratio unless stated otherwise. Ratios >1.0 are considered to be reactive. This panel is for investigational use only and the data is offered for informational purposes only. Data is generated at ZeptoMetrix Corporation and by our Lab Partners in the United States and Europe. We will endeavor to update our data sheets whenever new test data becomes available to us. All specimens collected in this longitudinal series are naturally occurring, undiluted, unprocessed samples collected from a single plasma donor in the United States. No Preservatives have been added.