

HIV Seroconversion Panel

(Catalog No. HIV 9019)

Donor No. 65685
For Research Use Only

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility. Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	BLEED DATE	CHIRON HIV-1 bDNA	ABBOTT HIV-1 p24 Ag	ABBOTT HIV-1 p24 Ag NT%	COULTER HIV-1 p24 Ag	ABBOTT HIV AB 1/2 EIA AxSYM	ABBOTT HIV AB 1/2 EIA rDNA
9019-01	11/20/97	<50	0.317	NEG	0.64	0.35	0.256
9019-02	11/23/97	<50	0.317	NEG	0.56	0.39	0.171
9019-03	12/28/97	571,700	1.571	NEG	1.44	7.54	>17.094

New 3rd Gen

PANEL MEMBER	ABBOTT MUREX HIV Ab 1/2 + O	ABBOTT PRISM ANTI HIV 1/2	BIOMERIEUX ORGANON TEKNIKA ANTI HIV-1	BIOMERIEUX ORGANON VIRNOSTIKA HIV UNI-FORM II PLUS O	BIO-RAD MULTISPOT HIV 1 RAPID TEST	BIO-RAD MULTISPOT HIV 2 RAPID TEST	BIOTEST ANTI HIV ½	DADE BEHRING ENZYGNOST ANTI HIV 1/2 PLUS	GENETICS SYSTEMS HIV 1/2 + O
9019-01	0.314	0.23	0.50	0.43	NEG	NEG	0.052	0.02	0.168
9019-02	0.258	0.26	0.50	0.83	NEG	NEG	0.018	0.01	0.136
9019-03	12.728	70.38	2.33	7.67	POS	NEG	6.797	5.37	9.709

New 4th Gen

PANEL MEMBER	GENETICS SYSTEMS HIV 1/2 EIA	ORAQUICK ADVANCE HIV 1/2 RAPID Ab	ORTHO ANTI HIV 1/2 ENHANCED ELISA	TRINITY BIO-TECH UNI GOLD RECOMBIGN HIV-1	CAMBRIDGE BIOTECH HIV-1 WESTERN BLOT	ABBOTT PRISM HIV Ag/Ab COMBO	ABBOTT MUREX HIV-1 Ab/Ag COMBO	ABBOTT HIV Ag/Ab COMBO ARCHITECT
9019-01	0.150	NEG	0.05	NEG	P24 (vf)	0.17	0.259	0.08
9019-02	0.150	NEG	0.05	NEG	P24 (vf)	0.06	0.234	0.07
9019-03	8.147	POS	11.0	POS	P24 p55 p66 p120	10.54	17.934	57.28

All specimens tested for HBsAg and Anti-HCV and found to be negative by an FDA approved test.

Notes on Data: Data is presented as an S/Co ratio unless stated otherwise. Ratios >1.0 are considered to be reactive. This panel is for investigational use only and the data is offered for informational purposes only. Data is generated at ZeptoMetrix Corporation and by our Lab Partners in the United States and Europe. We will endeavor to update our data sheets whenever new test data becomes available to us. All specimens collected in this longitudinal series are naturally occurring, undiluted, unprocessed samples collected from a single plasma donor in the United States. No Preservatives have been added.