

# HIV Seroconversion Panel

(Catalog No. HIV 9022)

**Donor No. 64578**

For Research Use Only

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility. Donor was un-treated and asymptomatic throughout the collection period.

New 3<sup>rd</sup> Gen

PANEL MEMBER	BLEED DATE	CHIRON HIV-1 bDNA	ABBOTT HIV-1 p24 Ag	COULTER HIV-1 p24 Ag	ROCHE ELECSYS HIV p24Ag	ABBOTT HIV AB 1/2 EIA AxSYM	ABBOTT HIV AB 1/2 EIA rDNA	ABBOTT HIV AB 1/2 EIA rDNA 3rd GEN	ABBOTT HIV AB 1/2 EIA rDNA IMx	ABBOTT MUREX HIV Ab 1/2 + O	ABBOTT MUREX SUDS ANTI HIV-1
9022-01	7/30/97	<50	0.41	0.241	0.37	0.24	0.121	0.34	0.36	0.384	0.15
9022-02	8/02/97	<50	0.39	0.266	0.32	0.21	0.121	0.33	0.33	0.350	0.17
9022-03	8/06/97	<50	0.40	0.253	0.31	0.23	0.129	0.28	0.39	0.341	0.16
9022-04	8/09/97	<50	0.41	0.342	0.32	0.26	0.138	0.26	0.31	0.306	0.17
9022-05	8/14/97	<50	0.48	0.253	0.51	0.21	0.138	0.26	0.31	0.341	0.19
9022-06	8/16/97	<b>107</b>	0.43	0.177	0.58	0.19	0.129	0.34	0.36	0.394	0.17
9022-07	8/22/97	<b>32,296</b>	0.51	0.544	<b>1.33</b>	0.23	0.103	0.27	0.28	0.647	0.15
9022-08	8/24/97	<b>199,644</b>	<b>1.94</b>	<b>3.051</b>	<b>7.45</b>	0.21	0.129	0.26	0.33	0.431	0.16
9022-09	8/31/97	<b>&gt;500,000</b>	<b>&gt;24.4</b>	<b>36.329</b>	<b>320.9</b>	<b>10.80</b>	<b>14.828</b>	<b>&gt;14.00</b>	<b>11.71</b>	<b>10.631</b>	<b>9.30</b>

All specimens tested for HBsAg and Anti-HCV and found to be negative by an FDA approved test.

**Notes on Data:** Data is presented as an S/Co ratio unless stated otherwise. Ratios >1.0 are considered to be reactive. This panel is for investigational use only and the data is offered for informational purposes only. Data is generated at ZeptoMetrix Corporation and by our Lab Partners in the United States and Europe. We will endeavor to update our data sheets whenever new test data becomes available to us. All specimens collected in this longitudinal series are naturally occurring, undiluted, unprocessed samples collected from a single plasma donor in the United States. No Preservatives have been added.

# HIV Seroconversion Panel

(Catalog No. HIV 9022)

**Donor No. 64578**

For Research Use Only

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility. Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ABBOTT PRISM ANTI HIV 1/2	BIOMERIEUX ORGANON VIRNOSTIKA HIV-1 MICROELISA SYSTEM	BIOMERIEUX ORGANON VIRNOSTIKA HIV UNI-FORM II PLUS O	BIO-RAD NOVAPATH HIV-1 IMMUNOBLOT	BIOTEST ANTI HIV 1/2	BIOTEST ANTI HIV-1 TETRA ELISA	DADE BEHRING ENZYGNOST ANTI HIV 1/2 PLUS	DADE BEHRING ENZYGNOST ANTI HIV 1/2 PLUS	GENETICS SYSTEMS HIV 1/2 PEPTIDE EIA
9022-01	0.26	0.202	0.48	No Bands	0.04	0.29	0.14	0.05	0.246
9022-02	0.27	0.231	0.42	No Bands	0.06	0.28	0.13	0.04	0.429
9022-03	0.26	0.231	0.45	No Bands	0.03	0.26	0.04	0.04	0.314
9022-04	0.28	0.231	0.45	No Bands	0.03	0.34	0.13	0.04	0.429
9022-05	0.28	0.259	0.46	No Bands	0.05	0.26	0.12	0.03	0.257
9022-06	0.30	0.231	0.52	No Bands	0.04	0.27	0.15	0.03	0.314
9022-07	0.25	0.231	0.42	No Bands	0.03	0.25	0.14	0.05	0.257
9022-08	0.31	0.259	0.48	No Bands	0.04	0.25	0.14	0.40	0.200
9022-09	<b>8.79</b>	0.288	<b>3.00</b>	No Bands	0.71	<b>Max abs</b>	<b>6.11</b>	<b>5.79</b>	0.229

All specimens tested for HBsAg and Anti-HCV and found to be negative by an FDA approved test.

**Notes on Data:** Data is presented as an S/Co ratio unless stated otherwise. Ratios >1.0 are considered to be reactive. This panel is for investigational use only and the data is offered for informational purposes only. Data is generated at ZeptoMetrix Corporation and by our Lab Partners in the United States and Europe. We will endeavor to update our data sheets whenever new test data becomes available to us. All specimens collected in this longitudinal series are naturally occurring, undiluted, unprocessed samples collected from a single plasma donor in the United States. No Preservatives have been added.

# HIV Seroconversion Panel

(Catalog No. HIV 9022)

**Donor No. 64578**

For Research Use Only

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility. Donor was un-treated and asymptomatic throughout the collection period.

New 4<sup>th</sup> Gen

PANEL MEMBER	GENETICS SYSTEMS rLAV EIA	ORTHO ANTI HIV 1/2 ENHANCED ELISA	ROCHE ENZYUMUM ANTI HIV-1 COMBI	SANOFI GENSCREEN ANTI HIV 1/2	ABBOTT PRISM HIV Ag/Ab COMBO	ABBOTT MUREX HIV-1 Ab/Ag COMBO	ABBOTT HIV Ag/Ab COMBO ARCHITECT
9022-01	0.071	0.05	0.10	0.04	0.07	0.249	n/a
9022-02	0.074	0.04	0.09	0.09	0.08	0.213	0.10
9022-03	0.083	0.06	0.09	0.09	0.09	0.213	0.14
9022-04	0.080	0.06	0.09	0.09	0.08	0.228	0.08
9022-05	0.080	0.05	0.08	0.04	0.06	0.264	0.08
9022-06	0.062	0.06	0.09	0.08	0.06	0.254	0.15
9022-07	0.053	0.05	0.10	0.08	0.51	0.690	n/a
9022-08	0.062	0.06	0.56	0.06	<b>5.09</b>	<b>3.792</b>	<b>4.62</b>
9022-09	0.133	<b>12.963</b>	<b>11.80</b>	<b>Max abs</b>	<b>126.98</b>	<b>18.954</b>	<b>175.31</b>

All specimens tested for HBsAg and Anti-HCV and found to be negative by an FDA approved test.

**Notes on Data:** Data is presented as an S/Co ratio unless stated otherwise. Ratios >1.0 are considered to be reactive. This panel is for investigational use only and the data is offered for informational purposes only. Data is generated at ZeptoMetrix Corporation and by our Lab Partners in the United States and Europe. We will endeavor to update our data sheets whenever new test data becomes available to us. All specimens collected in this longitudinal series are naturally occurring, undiluted, unprocessed samples collected from a single plasma donor in the United States. No Preservatives have been added.