



## NATtrol™ Cocksackievirus B4 Stock

Catalog #: NATCXVB4-ST

### PRODUCT DESCRIPTION:

**NATtrol™ Cocksackievirus B4 Stock (NATCXVB4-ST)** is formulated with purified, intact virus particles that have been chemically modified to render them non-infectious and refrigerator stable\*. Each vial contains 1.0 mL of Cocksackievirus B4 NATtrol™. NATCXVB4-ST stock is supplied in a purified protein matrix that mimics the composition of a true clinical specimen.

\*NATtrol™ Patents Pending

### INTENDED USE:

- NATtrol™ Cocksackievirus B4 Stock is designed to evaluate the performance of nucleic acid tests for determination of the presence of Cocksackievirus RNA. NATCXVB4-ST can also be used for validation of clinical assays, development of diagnostic tests and training of laboratory personnel.
- NATCXVB4-ST contains intact virus and should be run in a manner identical to that used for clinical specimens.

### ETIOLOGIC STATUS/BIOHAZARD TESTING:

- NATtrol™ inactivation was carried out on the Cocksackievirus B4 stock used to formulate this product. The inactivation was verified by the absence of viral growth in validated tissue culture based infectivity assays.
- The purified protein matrix was manufactured from materials that were screened and found to be negative for HIV 1&2 Ab, HBsAg, HTLV I&II Ab, HCV Ab, HIV RNA, HBV DNA and HCV RNA using FDA cleared kits at the single donor level.

This product was manufactured in a facility which has a Quality Management System that is certified as being in compliance with ISO 13485.

PINATCXVB4-ST  
Rev.: 01/2013  
For Research Use Only

### PRECAUTIONS:

- Although NATCXVB4-ST contains inactivated virus, it should be handled as if potentially infectious.
- Use Universal Precautions when handling this product.
- To avoid cross-contamination, use separate pipette tips for all reagents.

### RECOMMENDED STORAGE:

- NATtrol™ Cocksackievirus B4 Stock should be stored at 2-8°C.

### INSTRUCTIONS FOR USE:

- Extract Cocksackievirus B4 RNA prior to use in downstream assays.

TABLE 1:

Catalog Number	Concentration (Ct Range)*
NATCXVB4-ST	22-25

\*Cycle threshold (Ct) range based on in-house real time PCR assay targeting the 5' UTR gene region.

### DO NOT USE IN HUMANS

These products are intended for research, product development, quality assurance or manufacturing use. These products are NOT intended for use in the manufacture or processing of injectable products subject to licensure under section 351 of the Public Health Service Act or for any other product intended for administration to humans.

ZeptoMetrix Corporation  
878 Main Street  
Buffalo, New York 14202  
Phone: 716-882-0920  
Fax: 716-882-0959  
Customer Service: 800-274-5487  
www.zeptometrix.com