

USA FDA Registration Number 3000202849



NATtrol[™] SARS-Related Coronavirus 2 Positive Control Part Number: NATSARS(COV2)-ERC1-IVD

INTENDED USE:

The NATtrol[™] SARS-Related Coronavirus 2 Positive Control is an unassayed *in vitro* diagnostic external run control intended to be used with qualitative molecular assays for detection of nucleic acids from these organisms. The control is intended to be used as an aid to diagnosis in that it is used to verify performance of the assays used to detect a physiological or pathological state. The routine and repetitive use of external run controls enables laboratories to monitor daily test variation, lot-to-lot test kit performance, individual operator variation, and can provide assistance in identifying increases in random or systemic errors. NATtrol[™] SARS-Related Coronavirus 2 Positive Control contains intact organisms and should be run in a manner identical to that used for clinical specimens. The control does not have an assigned value and it is the responsibility of the end user to establish their own target specifications for the control using their laboratory's molecular procedures.

PRODUCT SUMMARY AND EXPLANATION:

NATtrol[™] SARS-Related Coronavirus 2 Positive Control is formulated with purified, intact organisms that have been chemically modified to render them non-infectious and refrigerator stable^{*}.

Each control pack contains 6 x 1.0 mL vials of NATtrol[™] inactivated virus listed in Table 1 formulated in a proprietary purified protein matrix.

*Pat.: http://www.zeptometrix.com/patent-information/

PRINCIPLE:

NATtrol[™] SARS-Related Coronavirus 2 Positive Control contains SARS-CoV-2 inactivated by ZeptoMetrix's patented NATtrol[™] process formulated in a proprietary purified protein matrix. These are full process controls designed to monitor the effectiveness of extraction, amplification, and detection in nucleic acid testing procedures. These controls are suitable for use with in-house molecular assays and commercially available molecular assays.

PRECAUTIONS:

NATtrol[™] inactivation was carried out on virus stocks prior to formulation of the controls. The inactivation was verified in a standard microbiological growth protocol.

This control contains material of human and animal origin, and the user should observe Universal Precautions when handling and disposing of this product. Disposal must follow local regulations if more stringent then regulations enforced by the CDC or the FDA.

Do not pipette by mouth.

To avoid cross-contamination, use separate transfer pipettes or tips for all materials.

Do not use beyond the expiration date shown on the label.

If product is received damaged or leaking, contact ZeptoMetrix LLC for instructions.

NOT FOR USE IN HUMANS:

These products are NOT intended for use in the manufacture or processing of injectable products subject to licensure under the USA Food and Drug Administration Section 351 of the Public Health Service Act or for any other product intended for administration to humans.

RECOMMENDED STORAGE:

NATtrol™ SARS-Related Coronavirus 2 Positive Control should be stored at 2-8°C.

INSTRUCTIONS FOR USE:

The NATtrol[™] SARS-Related Coronavirus 2 Positive Control is for single use only.

Mix tube vigorously for at least 5 seconds.

Process according to manufacturer's instructions for sample to result assays.

Extract nucleic acid prior to use in downstream assays that are not samples to result.

LIMITATIONS:

NATtrol[™] SARS-Related Coronavirus 2 Positive Control is a USA FDA Class 1 exempt, unassayed *in vitro* diagnostic external run control and are for professional use only. NATtrol[™] SARS-Related Coronavirus 2 Positive Control is not intended for use as a substitute for the internal controls provided by *in vitro* diagnostic kit manufacturers. Quality control materials should be used in accordance with local, state, federal and accreditation requirements.

EXPECTED RESULTS:

Qualitative results are shown in Table 1 below. This is provided for informational purposes only.

Each laboratory must evaluate the controls and establish their own acceptance criteria.

Table 1:

Part Number	Organism/ Strain	Target Range	Expected Result
NATSARS (COV2)- ERC1-IVD	SAR-CoV-2 (Isolate: USA- WA1/2020) ¹	3.97E+04 - 6.29E+04 Copies/mL	Detected

¹ This reagent was deposited by the Centers for Disease Control and Prevention and obtained through BEI Resources, NIAID, NIH: SARS-Related Coronavirus 2, Isolate USA-WA1/2020, NR-52281

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038, USA • Tel (508) 553-5800 • Fax (508) 520-1525 This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified. For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com



USA FDA Registration Number 3000202849



NATtrol[™] SARS-Related Coronavirus 2 Positive Control Part Number: NATSARS(COV2)-ERC1-IVD

ETIOLOGIC STATUS/BIOHAZARD TESTING:

NATtrol[™] inactivation was completed on the virus stocks used to formulate each control and verified in a standard microbiological growth protocol.

The Purified Protein Matrix used in the manufacture of this product is treated with 0.09% sodium azide. It was manufactured from materials that have been tested and found non-reactive at the donor level for HIV-1/HIV-2 Antibody, HBsAg and HCV Antibody by FDA licensed donor screening test methods. All materials are also tested for HIV-1 and HCV by FDA approved Nucleic Acid Test (NAT) methods. Heat inactivated bovine based source materials used in the manufacture of this product meet applicable USDA requirements for abattoir sourced animals, traceability and country of origin. The materials were collected at USDA licensed establishments or legally imported from countries recognized by the USDA as negligible or controlled for risk for Bovine Spongiform Encephalopathy (BSE) and other exotic disease agents. Donor animals were inspected ante and post mortem at the abattoir as required by the USDA.

PRODUCT WARRANTY:

ZeptoMetrix LLC's limited product warranty and other terms and conditions related to the purchase and use of ZeptoMetrix products are set forth in ZeptoMetrix's Terms and Conditions of sale found on ZeptoMetrix's website at <u>Sales Terms and Conditions</u>. If you have any questions, please contact ZeptoMetrix Customer Service at <u>zepto.customerservice@antylia.com</u>.

DISCLAIMER AND LIMITATION OF LIABILITY:

ZeptoMetrix LLC disclaims all warranties with respect to this document and the information contained herein, expressed or implied, including but not limited to those of merchantability, fitness for a particular purpose, or non-infringement. To the extent allowed by law, in no event shall ZeptoMetrix LLC be liable, whether in contract, tort, warranty, or consequential damages or lost profits in connection with or arising from this document and the information contained herein, including but not limited to the use thereof even if ZeptoMetrix is advised of the possibility of such damages.

	Manufacturer	X	Temperature Limitation
IVD	In vitro Diagnostic Use	Σ	Use-By Date
CE	European Mark of Conformity	Ŕ	Biological Risk
REF	Part / Catalogue Number	EC REP	Authorized Representative
LOT	Batch Code	Ĩ	Consult Instructions for Use
BIO	Contains biological material of animal origin	BIO	Contains biological material of human origin
UDI	Unique Device Identifier		

LEGEND OF LABELING SYMBOLS:







©2022 ZeptoMetrix LLC. All rights reserved. The trademarks mentioned herein are the property of ZeptoMetrix LLC (ZM) or the respective owners.