



# MANUAL *Erection System*®

Vacuum erection device for erectile dysfunction  
(Impotence)

## Application:

The **MANUAL Erection System**® is a medical device for achieving an erection. The manual pump creates a vacuum in the cylinder around the penis. This draws blood into the shaft of the penis and causes it to swell and become erect. One of the constriction rings is then slid down over the shaft of the penis. This prevents blood from flowing back out of the penis when the cylinder is removed and also maintains the erection. The thus obtained erection allows for normal and satisfying sexual intercourse.

The **MANUAL Erection System**® can be easily cleaned with warm water and mild soap. All parts, including the constriction rings in four different sizes, can be reordered years later. All parts are kept for re-use in the bag supplied.

## Indication: erectile dysfunction caused by:

- Diabetes
- Hypertension (high blood pressure)
- Disorder of the cardiovascular system
- high blood lipid concentration
- Depressions or other mental disorders
- neuropathic disorder (multiple sclerosis, paraplegia)
- Prostate gland disorder and/or prostate surgery
- Vascular disease of the glands penis
- Disorder of the liver or kidneys
- cardiac infarction or stroke
- side effects of pharmaceuticals (medication)  
(for e.g. against hypertension, irregular heartbeat, high blood lipid concentration, anti-inflammatory, anti-depressive and for dehydration)
- hormonal or mental disorder
- alcohol and drug abuse
- smoking

## The **MANUAL Erection System**® consists of:

- manual vacuum pumphead
- translucent cylinder
- constriction ring guide sleeves 1 and 2
- cone
- 4 different constriction rings  
(10; 12.5; 15 and 20 mm inner diameter)
- lubricant
- instructions for use
- storage bag
- Animation video on DVD

EAN: 4013273001175

Art. Nr.: MES 01

Made in Germany

 medical device



KESSEL medintim GmbH  
Nordendstr. 82-84  
64546 Moerfelden-Walldorf, Germany

Tel.: +49 (0) 6105 20 37 20  
Fax: +49 (0) 6105 20 37 221  
Email: [service@medintim.de](mailto:service@medintim.de)



[www.medintim.de](http://www.medintim.de)