



Carex Health Brands



**Mobility & Comfort for
your active lifestyle**

Carex Health Brands • Sioux Falls, SD • www.carex.com
42-A22300-02 ©2012

Step N' Rest[®]

Rolling Walker



Carex Health Brands

1 Check Parts

2 Brake Assembly


Rolling walker frame



Backrest



Handles with brake assembly

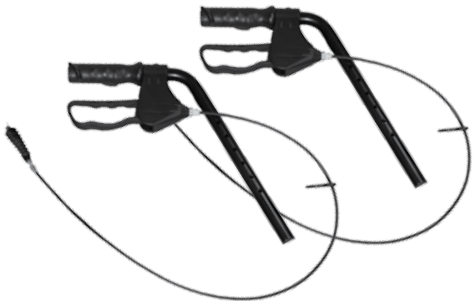

Rear wheels





C-clips & hand screws with washers



Front wheels




BRAKE may come pre-assembled, if so skip to step 3

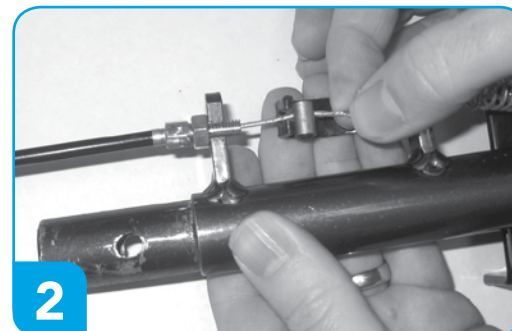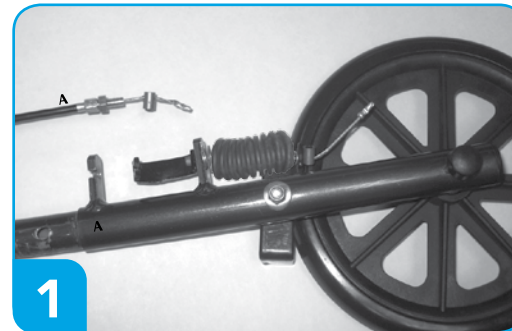

Note: actual items may vary slightly from what is shown.

3 Handle Assembly




 **NOTE:** Hex shaped bolt should fit onto hex shaped hole in tubing. Hand screw faces out.


Cable should be outside frame.



4 Front & Rear Wheel Assembly

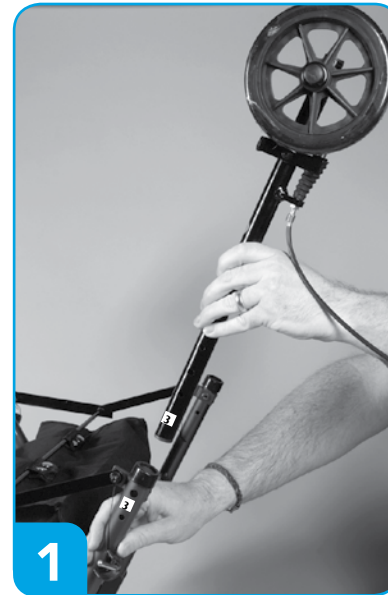

Line up holes.
C-clip in small hole.

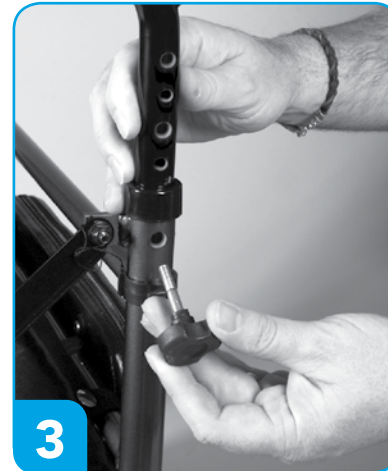

Hand screw in large hole



5 Backrest Assembly & Braking System




1

Push in side buttons and insert backrest. Flip up backrest for compact storage.



2

6 Adjusting the Handles & Seat Height

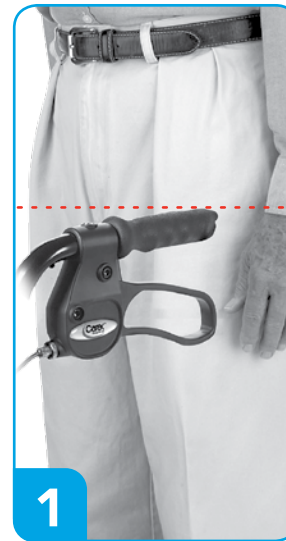

1



2




3

Using the brakes



Pull up to apply brakes

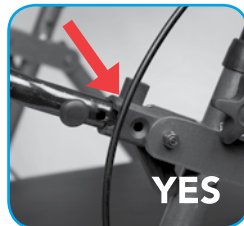

YES




Release brakes

Note: Cable needs to be placed outside seating area and frames.




Push down to lock brakes

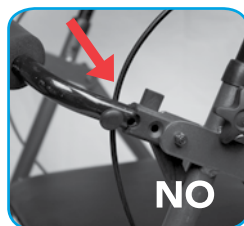

NO



Seat height adjustment all four wheels.

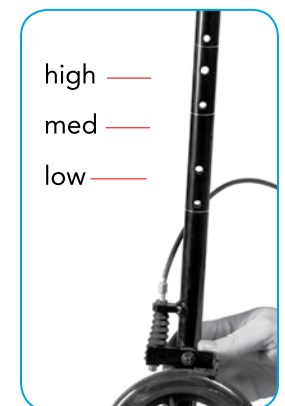

high —
med —
low —