

# SPECIFICATIONS: INFECTION CONTROL COVERS

## Disposable Mesh & Mattress Covers

### SPECIFICATIONS

- Medical grade, breathable, and waterproof fabric
- 100% Latex-free Polypropylene
- Fabric meets AAMI PB70 Level 3 Testing

### INSTRUCTIONS

- For single patient use. DO NOT share covers between babies.
- Dispose of and replace cover if damaged, cut, or torn. Cover may be used for baby's length of stay if material is not compromised.
- Swaddle the baby in the SNOO Sack and attach the wings to the SNOO safety clips at all times during use.



## Reusable Mesh Cover & Sealed Mattress

### SPECIFICATIONS

- Waterproof and flexible fabric
- 75% Polyester
- 25% Polyurethane
- 100% free of PFAS and PFOA
- CPSIA certified

### INSTRUCTIONS

- Covers **MUST BE** cleaned between patients to meet hospital infection control requirements.
- Additionally, clean the cover and sealed mattress if visibly soiled and every 7 days (if in use).
- Replace if damaged, cut, or torn.
- Swaddle the baby in the SNOO Sack and attach the wings to the SNOO safety clips at all times during use.



### HOW TO CLEAN

*Option 1:* Use CaviWipes or hydrogen peroxide-based disinfectant to wet all exposed surfaces on the bassinet and the mesh and mattress covers. The surface should remain visibly wet for 1 min. at room temperature.

*Option 2:* Wash cover on hot setting between 160°F-165°F using liquid detergent; may be washed with zero-residue oxygenated bleach containing no chlorine. Dry cover on medium heat at a temperature not exceeding 130°F.

## ASSEMBLY STEPS: REUSABLE COVERS

- 1 Remove mattress pad from inside SNOO bassinet.
  - 2 With the shiny side of the bassinet cover facing out, locate the cover warning label and align with the warning label at the back of SNOO.
  - 3 Loosely drape the cover material inside the SNOO. Note the two elastic seams – these go around the top rim of SNOO.
  - 4 Situate the elastic seams so that the seams sit on either side of the top mesh rim of the SNOO.
  - 5 Pull the cover down along the outer perimeter of the SNOO on one side and then the other.
  - 6 Locate the cover's elastic openings inside the bassinet, near the safety clips; stretch the cover openings around the safety clips on both sides.
  - 7 Cover the sealed mattress with a cotton fitted mattress sheet and place it in the SNOO on top of the bassinet cover.
- \* *Note: The cover is designed for a snug fit; the material is durable and can withstand stretching.*



Elastic Seams



Covers on Outer Perimeter




Safety Clips



Sealed Mattress with Fitted Sheet

## ASSEMBLY STEPS: DISPOSABLE COVERS

- 1 Remove mattress pad from inside SNOO bassinet.
- 2 With the disposable bassinet cover label facing out, align the cover label with the warning label at the back of SNOO.
- 3 Loosely drape the cover material inside the SNOO.
- 4 Pull the cover down along the outer perimeter of the SNOO on one side and then the other.
- 5 Locate the cover's elastic openings inside the bassinet, near the safety clips. Stretch the cover openings around the safety clips on both sides.
- 6 Slide the mattress into the disposable cover (the label should be on the same side as the opening).
- 7 Cover the mattress with a cotton fitted mattress sheet and place it in the SNOO on top of the bassinet cover.

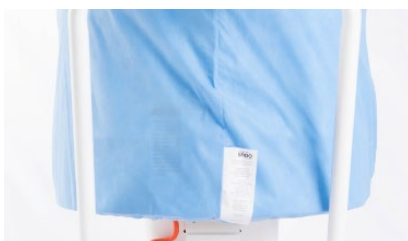


*Bassinet Cover Label*



*Safety Clips*



*Mattress with Cover*

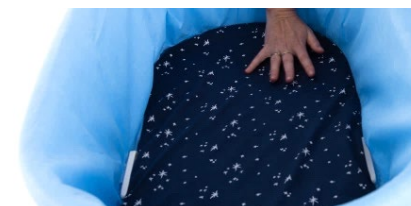

*Covers with Sheet*