


IMPRESSION SYSTEM

For Crowns, Bridges & Veneers

NEVER USE CORDS, RETRACTION or HEMOSTASIS AGAIN!


- A **Chemical-Free** and **Cord-Free**, fluid-displacement system.
- If you want perfect impressions with **NO Retraction or Hemostasis**, the patented MiTi 2-step system is the **ONLY** solution!
- **Fast: 4 to 5 minutes** from preparation to completion! (2 steps, with a 95 second set-time for each). No Retraction and Hemostasis will save at least 8-10 minutes.
- MiTi VPS is **fully scannable**, and delivers **GAG-FREETM** impressions.
- **Bio-compatible system:** NO 'Retraction and Hemostasis' means never tearing the critical junctional epithelium seal. Toxic biofilm of the sulcus is not forced into the surrounding blood vessels by cords or astringents.

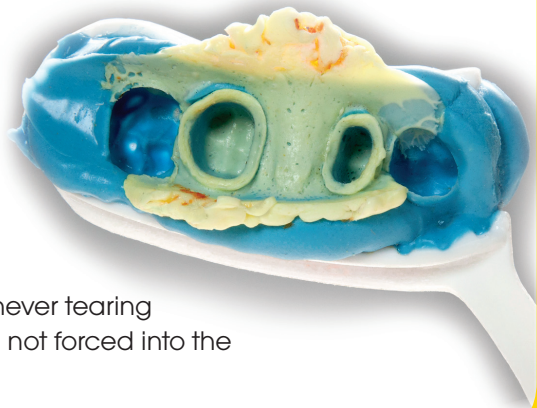

Carefully following the instructions will insure reliably perfect results every time.




MiTi trays and MiTi VPS are engineered to produce perfect results when used together. Product substitutions produce flawed impressions.

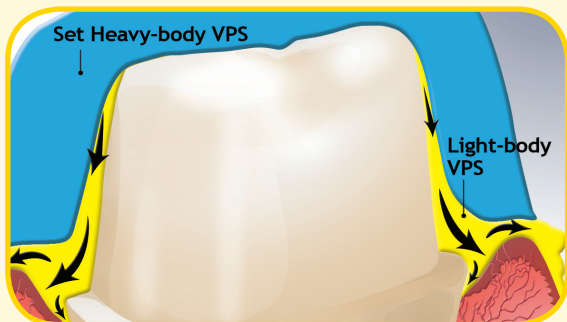

Why is there No Retraction?

MiTiTM system reliably flares the gingival margins and fills the deepest, bloodiest sulcus with MiTi Light-body VPS. Used correctly, there will never be bubbles, tears, drags or voids. The use of cords and pastes to displace the gingival margins is completely unnecessary, and creates an impression with less detail.

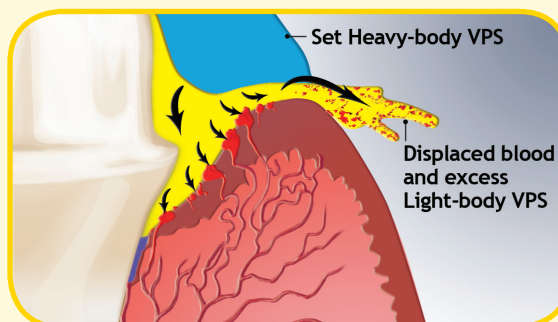

Why is there No Hemostasis?

The constant, controlled pressure of the Set Heavy-body VPS on the Light-body VPS causes flushing of blood and moisture from the sulcus and prevents new bleeding from the gingiva into the sulcus. Unlike chemical hemostasis, which takes 8-12 minutes to stop bleeding, this physical method is instant, complete and reliable, saving valuable chair time.

The MiTiTM Impression System is highly rated by independent, non-profit, Dental Education and Product Testing Foundation, Clinicians Report[®], November, 2018.

The full report is available from Dentalree.