



## Instructions - Heat Pad – Pet use

'Redzone' flexible heat pads are hand made by Adloheat Australia and manufactured from polyurethane to give water resistant properties combined with durability & flexibility. A protective steel cord cover of 400mm in length is added to the lead to prevent the animal chewing the lead.

The heat pads are specifically designed to provide bottom heat for the purpose of pet care, sick and old animals and whelping use. The surface of the heat pads will give approximately 12°C rise above the ambient temperature. The temperature of the heat pad will rise and fall with the ambient (air) temperature. Thermostat controllers are available where exact temperatures are required.

### Directions for Use:

1. Place the heat pad on a heat resistant surface. If using on a concrete surface use polystyrene or similar as insulation underneath the heat pad. This will prevent unnecessary heat loss.
2. Your pet may be placed directly onto the heat pad. No need for cushioning. A pillow slip or newspaper may be used.
3. Do not wrap or place insulating material, blankets, pillows or doonas on top or around the heat pad. The heat produced by the heat pad must be able to radiate away.
4. Plug the lead into a power point.

### Information:

- Allow room for baby or sick animals to move off the pad to a cooler area.
- Do not leave sick animals unattended on the heat pad.
- The Heat pad may feel cool to touch, this is normal. When the pet lies on the pad, a thermal mass is created and heat will transfer quickly to the animal.

### Features

- Water and UV resistant
- Manufactured from high quality polyurethane
- 240 Volt AC, not thermostatically controlled
- Lead protection - Stainless steel chew resistant cord cover - 400mm length
- Tough & durable construction
- Inexpensive to run
- Safe - Double insulated
- Provides constant low heat
- Complies with Australian Safety Standards AS.3100 N11384
- Australian made by Adloheat in Pakenham Victoria
- Easy to clean—use warm soapy water
- Resistant to most chemical detergents
- Flexible pad will sit on an uneven surface

### Sizes & wattages available:

Small 280 x 340mm (10 Watts) - for Cats & Dogs

Medium 340 x 600mm (20 Watts) - for medium to large dogs

Jumbo 1200 x 600 (120 watts) - for very large dogs, new foals, pigs

### Care:

Do not immerse in water.

To clean spillage off your heat pad, disconnect power supply and wipe dry. Allow to dry completely.

The heat pad should not be in contact with any other object like bedding, blankets pillows etc.

Do not tamper with or alter this heat pad or electrical lead in any way.

Do not use if damaged

12 month warranty.

