


Broda Pressure Testing Summary

785 Broda Chair with Rubber Cushion



Subject 170 lb Male
Sat in chair for 10 minutes before testing started

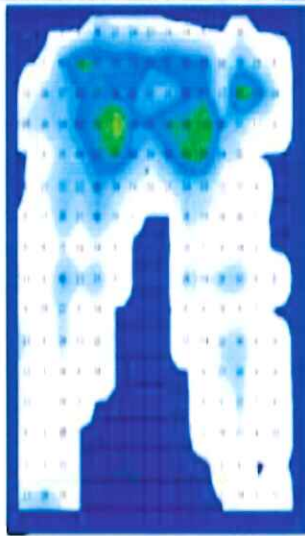


45 Degrees Tilt and 25 Degrees Recline

25.23 mmHg

785 Broda Chair with Foam Cushion



Subject 170 lb Male
Sat in chair for 10 minutes before testing started

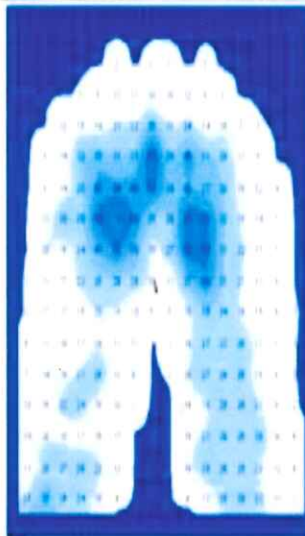


45 Degrees Tilt and 25 Degrees Recline

20.15 mmHg

785 Broda Chair with Custom Sheepskin Pads



Subject 170 lb Male
Sat in chair for 10 minutes before testing started

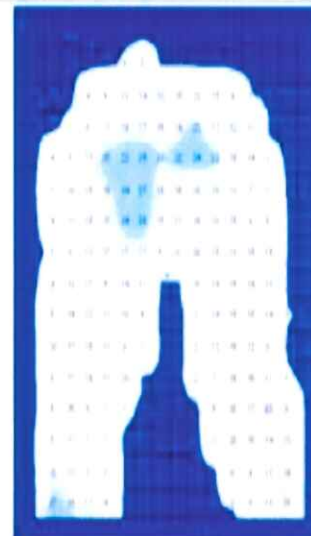

45 Degrees Tilt and 25 Degrees Recline

11.89 mmHg

NO cushions are needed to effectively relieve pressure on the Broda Chairs, if they are used properly.

The addition of High Temperature Medical Grade Sheepskin is the **MOST** effective Seating Surface to add to Broda chairs.

The combination of the Broda Strapping (Reduces Pressure) and The Sheepskin (Reduces Pressure, Friction and Moisture whilst maintaining standard body temperature 98.3 degrees - defeating the primary causes of pressure sores) is **highly recommended**.