LAUNDERING INSTRUCTIONS
For
AUSTRALIAN MEDICAL (GREEN) SHEEPSKINS

MACHINE WASH AT HIGH TEMPERATURE:

- For effective washing and disinfection, always use Skinsan® detergent, which has no enzymes, phosphates, peroxide, alkali or bleach. Use approximately 15 ml (1 tbs) of Skinsan® detergent per pound of clean or dry sheepskin weight.
- A wet or soiled sheepskin (IE: Urine or fecal matter) should be washed at 25 mL per 1 lb
- A soiled skin should be immediately rinsed in cold water if delayed for laundry
- The washing machine should first be put through a rinse cycle to remove any Phosphates, enzymes, peroxide, alkali &/or bleaching agents which could cause irreversible damage the leather.
- Machine wash in hot water, up to 60°C, on gentle cycle for 10 minutes. Set machine for warm water rinse.
- To achieve high level thermal disinfection wash again (with Skinsan®) for 8 minutes in 80°C water. Set machine for cold water rinse.
- Spin off excess water. Run spin cycle a 2nd time
- Drying: Tumble dry on warm setting – do not exceed 60°C. Avoid over-drying.
- Alternatively, hang dry away from direct heat or sunlight.
- Before leather is fully dry, flex skin vigorously to restore leather softness.

HAND WASH:

- Alternatively, the skin can be hand washed in warm water with Skinsan® detergent. Rinse and squeeze to remove excess moisture. Hang dry away from direct heat and direct light. Before leather is fully dry, flex skin to restore leather softness.

Note: Initial laundering will appear to have excessive fibre loss. This is due to freshly cut sheepskin edge. Coccyx pads should be pre-washed to remove any loose fibres which may adhere to wounds. Fibre loss should not continue after initial laundering. **Add a tablespoon of salt to the initial wash to keep the colour vibrant.**

Velcro straps should be fastened before laundering to minimize fibre entrapment in the Velcro.

- * **Warranty void if laundered in anything other than Skinsan Detergent.**
- * **Warranty is valid for 90 Days from date of purchase.**

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