


**verilux**<sup>®</sup>  
LIFE IN A BETTER LIGHT<sup>®</sup>

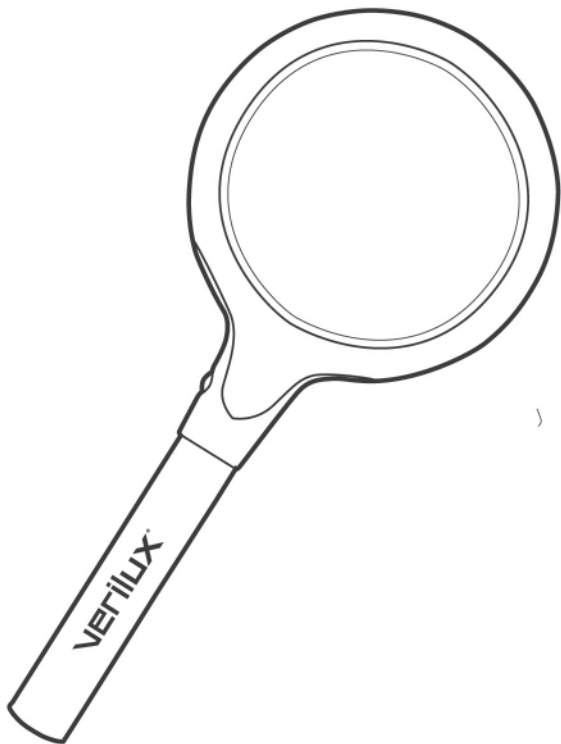

# EasyRead<sup>™</sup>

Natural Spectrum<sup>®</sup> Illuminated Magnifier

**VM02**



Dear Customer,

Thank you for purchasing the EasyRead™ Natural Spectrum® Illuminated Magnifier. You have received an innovative and quality manufactured product, backed by a one-year limited warranty. As a Verilux® customer, your satisfaction means everything to us. We look forward to serving you now and in the future.

Many other healthy lighting products are available through our catalog and online. Visit us on the web at [www.Verilux.com](http://www.Verilux.com) to request a free catalog and learn more about all our quality Verilux® products, or call our toll-free number 1-800-454-4408.

Have a Bright Day!

A handwritten signature in black ink that reads "Nicholas Harmon". The signature is written in a cursive, flowing style.

Nicholas Harmon  
President and CEO

## FEATURES

Made of premium components including a solid metal frame and optical-quality glass, this portable magnifier is perfect for seeing details on maps or projects. 12 super-bright Natural Spectrum® LED bulbs provide even illumination without glare to highlight even the smallest detail. Operates on 2-AA batteries (sold separately) with high/low illumination. Includes padded storage/travel case.

- 12 powerful LED bulbs with exclusive Natural Spectrum® daylight illumination
- High or low light operation from ergonomically positioned switch
- Optical quality glass lens provides 2.5X magnification
- 3.5" lens diameter
- Includes lens cloth and padded storage/transport case
- 1-year limited warranty

## OPERATION

The EasyRead™ Illuminated Magnifier has two brightness levels. Press the button on the handle once for the lowest setting. Press the button a second time for the highest setting. Press the button a third time to turn the unit off.


## BATTERY INSTALLATION

*When replacing batteries, please ensure that you use only 2 AA batteries.*

Unscrew the magnifier's handle. Take out any dead batteries and dispose of in a proper manner. (Fig. 1)

Insert 2 AA batteries into the handle. The "+" end should point towards the lens. Please make sure that the "+" and "-" are facing in the right direction, otherwise the magnifier will fail to operate.

Once batteries are inserted, carefully screw into place the magnifier's handle.

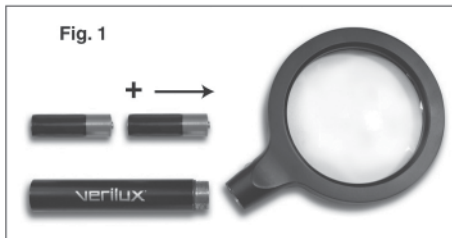

# ONE YEAR LIMITED WARRANTY

**ATTENTION!** ONCE OPENED, PLEASE **DO NOT** RETURN THIS PRODUCT TO THE STORE WHERE IT WAS PURCHASED FOR REPAIR OR REPLACEMENT!

Many questions may be answered by visiting [www.Verilux.com](http://www.Verilux.com), or you may call our customer service department at 1-888-544-4865.

This limited warranty is provided by:

Verilux, Inc., 340 Mad River Park, Waitsfield, VT 05673

Verilux® warrants this product to be free from defects in material and workmanship for a period of one year from the date of the original retail purchase. During the limited warranty period, Verilux, Inc. will, at its option, repair or replace defective parts of this product, at no charge to the customer, subject to the following limitations. This limited warranty does not include any postage, freight, insurance or delivery fees. This warranty does not cover damage, defect or failure caused by or resulting from accident, external destruction, alteration, modification, abuse, misuse, or misapplication of this product.

If, during the first year of ownership, this product fails to operate properly, it should be returned as specified below:

Please pack the defective product in its original protective packing and box, with a copy of your sales receipt showing Date of Purchase, along with \$6.95 (check or money order to cover shipping and handling), and send to:

**For U.S. Postal Service Only:**

**Verilux Warranty  
Service Center, c/o WTB  
P.O. Box 119,  
West Redding, CT 06896**

**For Fed Ex & UPS Only:**

**Verilux Warranty  
Service Center, c/o WTB  
32 Henry Street, Bldg 17A,  
Bethel, CT 06801**

Please allow 4-6 weeks before you receive the repaired or replaced product.

**ATTENTION:** When returning a product, be aware that damage during shipping is the responsibility of the purchaser and will void any warranty claim. We recommend that you insure the shipment by shipper when returning this product or any parts. Be sure to return product in original packaging.

Due to ongoing improvements, actual product may have slight variations from the product described in this manual.

Printed in China for Verilux, Inc.

© Copyright 2008 Verilux, Inc. All rights reserved.