

SAMPLE LOG

The sample log suggests records that should be maintained by the purchaser or user of rescue equipment.

Equipment Inspection and Maintenance Log			
Item _____ # _____		Date in Service _____	
Brand/Model _____		Strength _____	
Date	How Used or Maintained	Comments	Name

IMPORTANT INFORMATION - PLEASE READ AND SAVE



Adjustable Litter Strap

Made in USA
of US and foreign components

PRODUCT LABEL

! WARNING

- **SERIOUS INJURY OR DEATH MAY RESULT FROM THE IMPROPER USE OF THIS EQUIPMENT.**
- **THIS EQUIPMENT HAS BEEN DESIGNED AND MANUFACTURED FOR USE BY EXPERIENCED PROFESSIONALS ONLY.**
- **DO NOT ATTEMPT TO USE THIS EQUIPMENT WITHOUT PROPER TRAINING.**
- **USE, INSPECT, AND REPAIR ONLY IN ACCORDANCE WITH MANUFACTURER'S INSTRUCTIONS.**

CMC Rescue, Inc.

6740 Cortona Drive, Goleta CA 93117
USA
805-562-9120 / 800-235-5741
cmcrestue.com

ISO 9001:2008 Certified

©2015 CMC Rescue, Inc. Control No. 201102.00.050913

The **CMC Adjustable Litter Strap** provides an efficient way to connect litter tenders to the litter during a low angle evacuation. It can be easily adjusted for the most comfortable position and quickly released in order to set the litter down. It can also be used as a Pick Off Strap.

INSPECTION

Inspect the Adjustable Litter Strap according to your department's policy for inspecting rescue equipment. Equipment should be inspected after each use, and at least once a year by an inspector that meets your department's training standard for inspection of rescue equipment. Record the date of the inspection and the results in the equipment log. Each user should be trained in equipment inspection and should do a cursory inspection and check component compatibility before each use.

When inspecting the Adjustable Litter Strap, check the webbing for cuts, worn or frayed areas, broken fibers, soft or hard spots, or discoloration. Check the stitching for pulled threads, abrasion, or broken stitches. If any damage is noted, the Adjustable Litter Strap should be removed from service and retired.

If the Adjustable Litter Strap is dropped or impact loaded, it should be inspected by a qualified inspector prior to being returned to service. In most cases, a visual inspection will not be able to determine if the equipment has been damaged. Based on the history of the incident, if there is any doubt regarding the safety of the equipment, it should be removed from service and destroyed.

USE

Use a Girth Hitch to attach the Adjustable Litter Strap to the top rail of the litter at a point ahead of where you want to be positioned. Clip the V Ring into your harness. Adjust the strap to the length that works best.

If you have to move away from the litter, lengthen the strap to give you as much slack as you need.

CARRYING, MAINTENANCE & STORAGE

During use, carrying, and storage, keep the Adjustable Litter Strap away from acids, alkalis, exhaust emissions, rust and strong chemicals. Do not expose the equipment to flame or high temperatures. Carry the Adjustable Litter Strap where it will be protected as the equipment could melt or burn and fail if exposed to flame or high temperatures.

If the Adjustable Litter Strap becomes soiled, it can be washed in cold water with a mild detergent. CMC Rescue recommends the use of LifeLine Cleaner. Dry out of direct sunlight. Do not dry in an automatic dryer. Store in a cool, dry location. Do not store where the equipment may be exposed to moist air, particularly where dissimilar metals are stored together.

REPAIR

All repair work shall be performed by the manufacturer. All other repair work or modification may void the warranty and releases CMC Rescue from all liability and responsibility as the manufacturer.