Installation and Operating Instructions

Model TEL-MC

Telephone Transmitter 418MHz RF Transmitter

Transmitter Features:

TEL-MC transmitters are battery operated. TEL-MC is coded at the manufacturer with a unique address ID. Transmitter has a test button for programming and follow on operational testing. When activated the RED LED will illuminate and the TEL-MC will transmit a phone alert to the Silent Call Medallion series receiver. Low Battery YELLOW LED will flash every 10 seconds when batteries are in need of replacement.

Transmitter Mounting and Location:


The TEL-MC Medallion Series Transmitter is designed for indoor use only. The transmitter case has provisions for mounting to a nail or screw on the wall. Velcro strip is also provided for mounting on desktop or wall. Avoid mounting transmitter near large metal objects, other transmitting devices in close proximity or high current appliances that will degrade the transmitter / receiver operation. Test transmitter mounting location before mounting to ensure proper system operation. Adjust antenna for best results. Transmission range with a Silent Call Medallion Series receiver is approximately 500 feet.

Installation:

1. The telephone transmitter will work with standard telephone lines and with most video phone RJ-45 outputs.
2. **To install the TEL-MC telephone transmitter on a conventional telephone line**, plug the telephone transmitter into any telephone outlet. Use the supplied dual adapter if you need to connect a telephone AND the transmitter to the same outlet.
3. **To install the TEL-MC telephone transmitter on a videophone**, plug the telephone transmitter into the telephone outlet on the videophone. Use the supplied dual adapter if you need to connect a telephone AND the transmitter to the videophone.
4. If plugged into a two line telephone outlet, the telephone transmitter will respond to line one only.
5. The TEL-MC transmitter may not work with digital phone systems. Please contact your telephone service provider as filters may be necessary to filter out the modem tones associated with digital (DSL) or VOIP type services.

Operation:

1. Whenever the telephone (or videophone) rings, the Red light on the telephone transmitter will light up and the transmitter will send a signal to your Silent Call Medallion Series Receiver causing the receiver to activate Phone Alert.

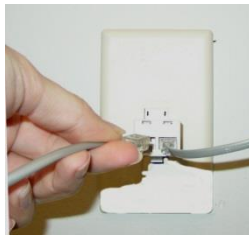Connect TEL-MC to Dual Adapter

Replacement Batteries:

Use two AA 1.5V Alkaline Batteries. AA Batteries may be purchased from local retail stores. Transmitter is shipped with 2 AA Alkaline Batteries. Batteries may also be purchased Online at www.silentcall.com.

Installation and Operating Instructions

Model TEL-MC

Telephone Transmitter 418MHz RF Transmitter

Regulatory Information Notice:

This device complies with Industry Canada license-exempt RSS standard(s).

Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation of the device. This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- *Reorient or relocate the receiving antenna.*
- *Increase the separation between the equipment and the receiver*
- *Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.*
- *Consult the dealer or an experienced radio/television technician for help*

Unauthorized changes or modifications could void the user's authority to operate the equipment.

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Technical Support

For technical support on this or any other Silent Call product, please feel free to contact us. You can reach us by phone at 800-572-5227 (voice or TTY) or by Email at support@silentcall.com

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Limited Warranty

Your transmitter is warranted to be free of defects in material and workmanship for five years from the date of initial purchase. During that time, the unit will be repaired or replaced free of charge when shipped prepaid to Silent Call Communications. This warranty is void if the defect is caused by customer abuse or neglect.

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Return for Serving To:

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Silent Call Communications

5095 Williams Lake Road, Waterford Michigan 48329

(800) 572-5227 v/tty (248) 673-7360 fax

Website: www.silentcall.com Email: silentcall@silentcall.com