

Installation and Operating Instructions

Bed Mat Transmitter

Model BM418-SS

Installation

1. Place the bed mat under the sheet on the bed in the area of where the users back will lay.
2. Allow the wire and transmitter to hang over the head of the bed so that the user is not lying on either part or does not trip on the wire when getting out of bed.
3. The transmitter is connected to the Bed Mat and must not be mounted directly to a metallic surface. Doing so will severely reduce the range of the transmitter.

Operation

1. When someone is lying on the bed mat it will not transmit. The user can roll around and it will not transmit.
2. The transmitter will activate once the person gets up out of bed and pressure is removed from the mat.
3. The transmitter will then send a signal from 250-2000 feet away depending on which Signature Series receiver you are using. The normal indication is strobe and the door indicator light on the Sidekick II Signature Series Receiver. The Watch Receiver or Communicator Receiver will vibrate and the door indicator ICON will illuminate.

CAUTION: *When handling or shipping the Bed Mat Transmitter, do not bend the mat sharply or fold it. It must be kept flat or rolled loosely or it will be damaged.*



Address Switch Settings

The Silent Call system is digitally encoded. All Silent Call receivers and transmitters are tested and leave the factory programmed to a factory default address. ***You do not need to change the address unless someone in your area has Silent Call products and they are interfering with your equipment.***

1. Make sure that all Silent Call transmitters in the area are turned off.
2. Located on the back of the transmitter case is a removable access panel. Remove the access panel and take out the batteries. Note that you MUST remove the batteries first or the switch setting will not take effect.
3. Locate the address switch on the transmitter circuit board that has 5 small dip switches. Set the switches to any combination that you want. For Example: 1, 2 ON 3, 4, 5 OFF. This gives your transmitter an "address". Note: Do not set the switches to the all "ON" or all "OFF" position.
4. Reinstall the batteries and replace the access panel.
5. Refer to your specific Signature Series Receiver instruction manual for programming your receiver to your newly changed transmitter address.

Technical Support

For technical support on this or any other Silent Call product, please feel free to contact us. You can reach us by phone at 800-572-5227 (voice or TTY) or by Email at support@silentcall.com

Limited Warranty

The transmitter itself is warranted to be free of defects in material and workmanship for five years from the date of initial purchase. The pressure sensitive mat is warranted for one year. During that time, the unit will be repaired or replaced free of charge when shipped prepaid to Silent Call Communications. This warranty is void if the defect is caused by customer abuse or neglect.