

Installation and operations manual

Shakeup Receiver with vibrator

Model SU5001-V

General

The Silent Call Shakeup receiver with vibrator is a table top AC operated alerting device. It is used in conjunction with any Silent Call smoke or fire transmitter. When the Shakeup receiver detects a signal from a Silent Call smoke or fire transmitter, it illuminates the front panel FIRE LED and activates the bed vibrator. The indication will continue for approximately 15 seconds after the transmission ceases.

Installation

Installing the Shakeup receiver consists of locating it within approximately 100 feet of the Silent Call transmitters and plugging the power supply into a household AC outlet. Place the vibrator in between the mattress and box spring, of your bed, under where you normally sleep. For best operation, the Shakeup receiver should not be placed closer than 10 feet to a personal computer. Some computers emit strong signals on random frequencies and this can interfere with the operation of your Shakeup. For best range, the Shakeup receiver should also not be placed on or near large metallic objects.

Operation

Each time it receives a signal from a Silent Call smoke or fire transmitter with a matching system code, the Shakeup receiver will illuminate the FIRE LED and activate the vibrator. This indication will continue for 15 seconds after the transmission(s) cease. Since the Smoke Sensor and Fire Alarm transmitters transmit as long as the alarm condition is present, the Shakeup will remain in alarm as long as the smoke or fire transmitter is transmitting.

The Shakeup receiver will work with any Silent Call smoke or fire transmitter.

Programming

NOTE: Your Shakeup receiver leaves the factory already programmed for the factory default code. It does NOT need to be programmed unless you have changed the code on your transmitters.

The Silent Call alerting system is digitally coded. Each Silent Call transmitter contains a five position code switch. The Shakeup receiver can be set to respond to one or more different transmitter codes. All Silent Call devices including the Shakeup receiver leave the factory set to the default code of 1=off, 2=off, 3=off, 4=on, 5=on. Ordinarily, this code will not need to be changed unless your system is within 100-200 feet of another Silent Call system. In that case, simply set all transmitters to a different code, erase the code memory in the Shakeup, and then reprogram the Shakeup receiver as follows:

To erase all programmed codes:

1. Unplug the Shakeup.
2. Open the access hatch on the bottom of the Shakeup and locate the PROGRAM button.
3. Press and hold the PROGRAM button. While pressing the program button, plug the Shakeup in and then release the PROGRAM button. This will erase all codes in the Shakeup memory.



5095 Williams Lake Road, Waterford Michigan 48329

(800) 572-5227

<http://www.silentcall.com>

To program a new code:

1. Press the program button. Both LEDs will illuminate. Hold the program button until the LEDs go out (about 8 seconds). Release the button and the Shakeup receiver is in program mode. It will remain in program mode for approximately 30 seconds. While in program mode, it will memorize the code of the first transmitter it receives and then exit program mode.
2. While the Shakeup receiver is in program mode, press the button on any Silent Call transmitter that has been set to the desired system code. When the Shakeup receiver receives the transmitter, it will memorize the code and exit program mode. The Shakeup receiver can memorize an unlimited number of system codes.
3. Note that you do not have to program the Shakeup receiver with every transmitter in your system as long as all transmitters are set to the same code.
4. **Programmed codes are retained in memory even when the Shakeup power is removed.**

Technical Support

For technical support on this or any other Silent Call product, please feel free to contact us. You can reach us by phone at 800-572-5227 (voice or TTY) or by Email at support@silentcall.com

Limited Warranty

Your Shakeup Receiver is warranted to be free of defects in material and workmanship for five years from the date of initial purchase (one year for the strobe and/or vibrator). During that time, the unit will be repaired or replaced free of charge when shipped prepaid to Silent Call Communications.

This warranty is void if the defect is caused by customer abuse or neglect.

Rev 2



5095 Williams Lake Road, Waterford Michigan 48329

(800) 572-5227

<http://www.silentcall.com>