


Sleep Alert Charger Model # SA-CHGR3

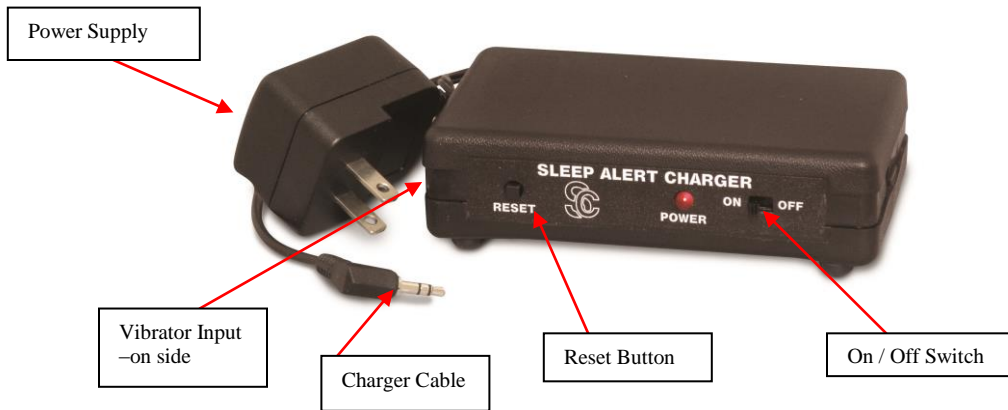

Installation

1. Turn the Personal receiver switch to the on position allowing it to receive incoming signals.
2. Plug the power supply attached to your Sleep Alert Charger into any standard 110 volt wall outlet.
3. Turn the Sleep Alert Charger to the “OFF” position.
4. Plug the Sleep Alert Charger wire (cable with 3.5mm jack) into the jack on the side of your Personal Receiver.
5. Plug the vibrator into the auxiliary jack on the side of the Sleep Alert Charger.
6. Place the vibrator between the mattress and the box springs.
7. Turn the Sleep Alert Charger switch to the “on” position and the RED power indicator light of the Sleep Alert Charger will come on.



Operation

Test operation by pressing the test button on any transmitter. The transmitter signal will be received by the Personal Receiver and will turn on the vibrator which is plugged into the Sleep Alert Charger.

Reset

To reset the Sleep Alert Charger, press the reset button. To reset the personal receiver, turn the on/off switch of the receiver to off and then turn it back on to receive the next transmitted signal.

Important

To receive signals from various transmitters while your receiver is charging, you must turn the Personal Receiver and Sleep Alert Charger to the “ON” position. The Personal Receiver can be charged in both the “on” or “off” position, but the Sleep Alert Charger must be in the “on” position.

Leaving the Personal Receiver on charge for extended periods of time does not harm the unit.