**SRE-3000 ELAN**  
**Straight Rail Stairlift**  
**OPERATOR'S MANUAL**

Please contact your Bruno dealer with any questions or service concerns, using the contact information shown below:

(Dealer: Place contact information below.)

Dealer Name:  
Address:  
Telephone No.:

(Español 19-37)  
(Français 39-57)

P/N 3000-O  
06-12-2017

# PRODUCT REGISTRATION FORM

 **Product Registration Form**  
for your independence **Please print in CAPITAL letters.**

Online registration available at <http://www.bruno.com/warranty.html>

Please be assured that Bruno **does not** share or sell the information you provide.

\* = required field

\*First Name \_\_\_\_\_ M.I. \_\_\_\_\_

\*Last Name \_\_\_\_\_

\*Street Address \_\_\_\_\_

\*City \_\_\_\_\_ \*State/Province \_\_\_\_\_

\*Zip/Postal Code \_\_\_\_\_

Email Address \_\_\_\_\_

Telephone No. \_\_\_\_\_

Date of Purchase (mm/dd/yy) \_\_\_\_\_

**Dealer:** Locate the Model No. sticker on the unit. No. decal on the unit is shipped with the unit and place it here.

**Model No.** \_\_\_\_\_

**\*Serial No.** \_\_\_\_\_

**End-User:** For faster registration, please go to the website shown above.

**Please rate your satisfaction with your Bruno dealer:**

excellent  good  average  poor

**Please rate your satisfaction with your Bruno product:**

excellent  good  average  poor

Bruno reserves the right to use information indicated on this form in its online, video, audio and printed materials. Names will be abbreviated to ensure the privacy and anonymity of the individual.

©2014,2005 Bruno Independent Living Aids, Inc.® Rev. 04-17-2014  
P/N PROD REG STD

Bruno is pleased to provide you with this mobility-enhancing product.

The Product Registration Form is shipped in a plastic bag with this manual. It is very important that you fill out and return this form at your earliest convenience so that we may complete the warranty registration process for your unit.

The **serial number** is found on labels affixed to the manual, as well as on the unit itself.

**Serial and model numbers must be provided** when filing a warranty claim, requesting service or ordering parts. We encourage you to keep this information readily available at all times.

### Please Note:

**The warranty for the SRE-3000 Stairlift is rendered null and void if the unit is installed or maintained by anyone other than an authorized Bruno dealer.**

Regulatory Information and Specifications . . . . .4  
Safety. . . . .5  
Circuit Breaker. . . . .6  
Seating. . . . .6  
Operation . . . . . 7-12  
    Key Operation . . . . .7  
    Standard Swivel Seat . . . . . 7-8  
    **Optional** Power Swivel Seat . . . . .8  
    **Optional** Power Footrest. . . . .9  
    **Optional** Manual Folding Rail . . . . .10  
    **Optional** Power Folding Rail . . . . . 11-12  
Battery Charger. . . . .13  
    Charging . . . . .13  
    LEDs. . . . .13  
Transmitters . . . . .14  
Maintenance . . . . .15  
Troubleshooting . . . . .16  
Vacation and Long-Term Storage . . . . .17  
Warranty. . . . .18

This lift is intended to be installed in accordance with the current edition of:


- EN 81-40:2008 Safety rules for the construction and installation of lifts - Special lifts for the transport of persons and goods - Part 40: Stairlifts and inclined lifting platforms intended for persons with impaired mobility
- ASME A18.1:2011 Safety Standard for Platform Lifts and Stairway Chairlifts
- CAN/CSA-B613-00 (R2005) - Private Residence Lifts for Persons With Physical Disabilities
- ASME A17.5/CSA B44.1 - 2011 Elevator and Escalator Electrical Equipment
- NFPA 70: National Electrical Code®
- CSA C22.1 Canadian Electrical Code

Verify applicable local codes with the appropriate regulatory authorities.

## SPECIFICATIONS

<i>Weight Capacity:</i>	300 lbs. (136 kg)
<i>Control:</i>	constant pressure (armrest and two remotes)
<i>Maximum Incline:</i>	45 degrees, specials to 52 degrees
<i>Seat Swivel:</i>	0, 67 and 90 degrees at top landing only
<i>Battery Charger:</i>	120VAC, 60 Hz, 0.35A input; 24VDC, 1A output, continuous monitoring

## COMPONENTS OF THE SRE-3000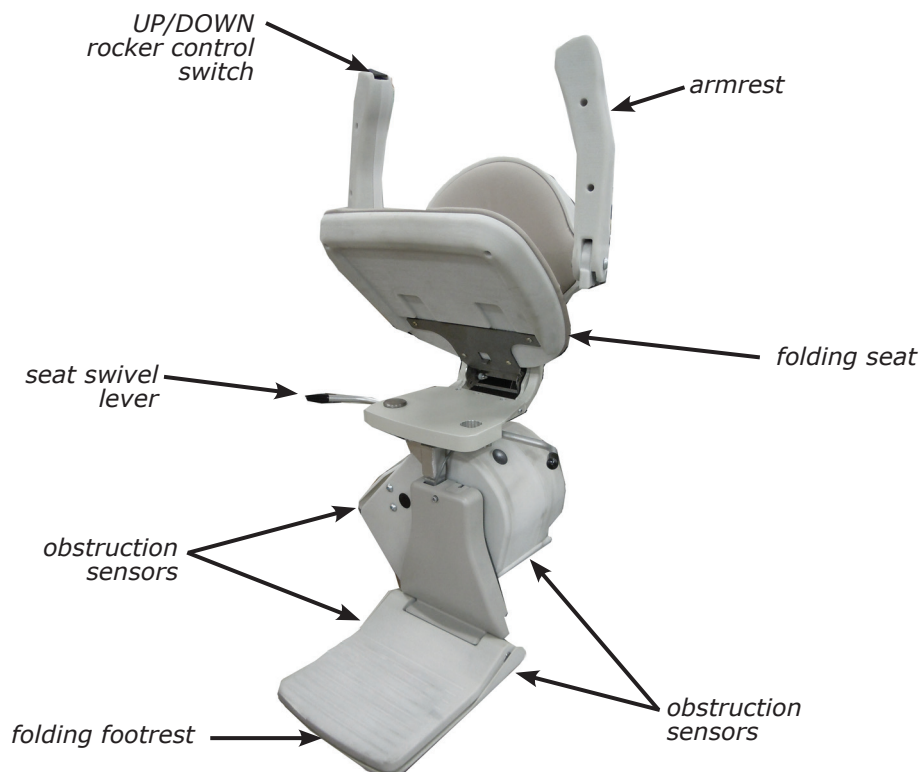

## **FOR YOUR SAFETY**

- Read and understand the operating instructions, and all safety precautions prior to using the SRE-3000.
- Be sure stairs are clear of objects before operating the stairlift.
- Always be aware of the surface onto which you are stepping as you get on and off the stairlift.
- Avoid carrying items with you on the stairlift which could hinder movement of the stairlift up and down the stairs.
- Do not wear LOOSE clothing that could interfere with stairlift movement.
- Fasten your positioning belt before operating the stairlift.
- Make sure the seat is locked in the correct riding position.
- Be sure that the seat swivel path is clear before swiveling in or out of the ride position, and before repositioning the seat.
- Be certain the seat swivel is locked in place before attempting to get on or off the stairlift.
- Make certain that there are no obstacles that would prevent complete raising/lowering of the optional power footrest (if unit is so equipped).
- NON-AMBULATORY USERS: Bruno recommends carrying a cellular or cordless telephone or an emergency communication device when riding the stairlift while alone in the house.

Each stairlift is designed to travel down the stairway at a slightly slower speed than is the case when travelling UP.

The footrest is equipped with *obstruction sensors* which stop the carriage if an obstacle is encountered on the stairs.

If this should occur, press the opposite side of the UP/DOWN rocker control switch (on the armrest) to run the carriage in the opposite direction. Clear the obstacle from the stairway or rail then resume travel in the desired direction.

Your SRE-3000 is battery powered and will continue to operate for some time during a power outage. The length of operation will depend on the battery condition, the length of the stairs and the weight of the rider.


*Only in the event of a prolonged power outage (more than one hour) will it be necessary to turn the power off (circuit breaker switch on the carriage) and turn off or unplug the charger from the wall outlet to conserve battery charge.*

The ***power swivel seat option*** is equipped with current overload sensing to prevent further seat swivel if a jam is detected.

The ***power footrest option*** is equipped with current overload sensing that will cause the footrest to return to its last position if a jam is detected.

The ***power folding rail option*** is equipped with current overload sensing to prevent damage if an obstruction or jam is detected.

### CIRCUIT BREAKER

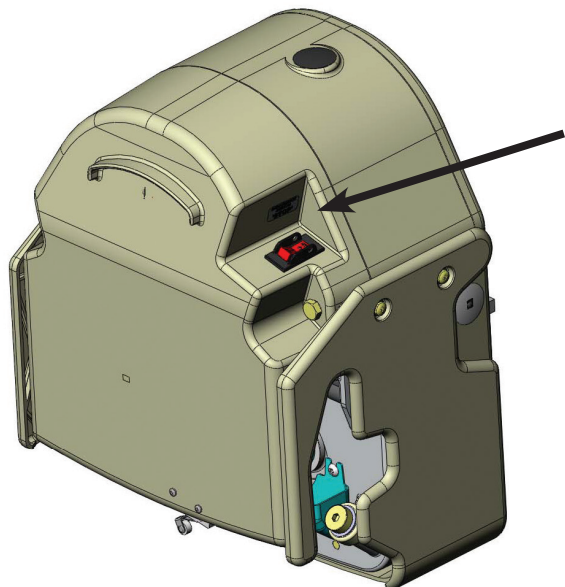The SRE-3000 includes a circuit breaker with a built-in ON/OFF switch located on the back side of the carriage.

The circuit breaker protects the battery, controller and motor circuits in the stairlift carriage.

It is unlikely that this circuit breaker will trip during normal use. However, if the stairlift becomes inoperative, check the circuit breaker as a first troubleshooting step. Reset it, if necessary.

Possible causes of circuit breaker tripping:

- foreign object jamming the rail or gear rack;
- exceeding the rated weight capacity of the stairlift.

---

### SEATING


The seat assembly on the SRE-3000 is designed to swivel and to lock at 0, 67 and 90 degrees.

Pushing down on either side of the swivel lever allows you to swivel the seat to a convenient position for transferring into or out of the seat.

Once you release the swivel lever, the seat will lock at the next available position.

To lower the seat assembly from a folded position to the riding position, simply push down on the seat.

## KEY OPERATION

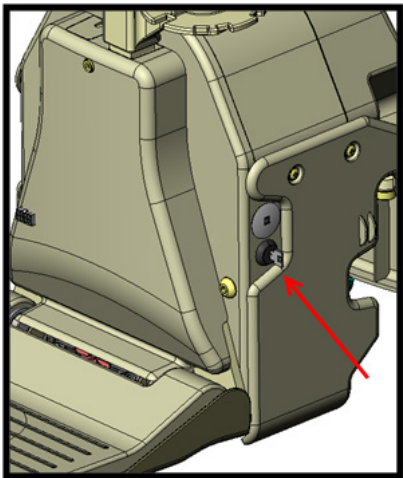The SRE-3000 includes a key and lock to disable use of the stairlift (see arrow).

Turn the key to its horizontal (ON) position (rotate clockwise) to allow the unit to operate.

When the key is in its vertical (OFF) position, the unit cannot be operated using the rocker switch on the chair or the remote transmitters.

**NOTE: Simply turning the key to its vertical (OFF) position does not remove all power from the unit.**

**To remove all power from the unit, you must turn off the circuit breaker (page 6).**

The key is designed to be removed in either the horizontal (ON) or vertical (OFF) position. If you do not need the added security of a key, Bruno recommends turning the key to the horizontal (ON) position, removing it from the carriage and storing it in a safe place.

## STANDARD SWIVEL SEAT

The SRE-3000 includes a "**soft-start**" feature which prevents the unit from moving abruptly the instant the UP/DOWN switch is pressed.

Press and hold the switch for a moment to begin the gentle acceleration of the stairlift to operating speed.

Depressing and holding the rocker switch will cause the carriage to move until the carriage stops automatically at the upper or lower end of the stair rail, or until the switch is released.

Two (2) remote transmitters enable you to move the carriage to or from either end of the stair rail.

Remote transmitters are designed to call / send an **empty seat only!** The transmitters are NOT designed to move a seat with a person on board.

1. Turn the keyswitch to ON.
2. Position yourself in the seat.
3. Fasten the positioning belt.
4. Depress and hold the seat swivel lever.
5. Rotate the seat so that you are at right angles to the rail (with your back to the stairway wall).
6. Release the seat swivel lever to secure the seat in that position.  
A safety switch prevents operation of the SRE-3000 unless the seat is locked in position at right angles to the rail (the "riding" position).
7. Lower both armrests completely.

**NOTE: If the armrests are not completely lowered, a safety switch disables the rocker switch and the stairlift will not operate.**

### STANDARD SWIVEL SEAT (continued)

8. Position both feet securely on the footrest and place your arms on the armrests.
9. Press and hold the rocker control switch (located on the front edge of one of the armrests) to travel UP / DOWN the rail.
10. Once you have reached the top or bottom landing, release the rocker switch.
11. Depress and hold the seat swivel lever as you rotate to the position for transferring from the seat.  
*NOTE: The seat will only swivel at the top landing.*
12. Release the seat swivel lever and allow the seat to lock in this position.
13. Unbuckle the positioning belt and carefully exit the seat.

### POWER SWIVEL SEAT (optional)

1. Position yourself in the seat.
2. Fasten the positioning belt.
3. Position both feet securely on the footrest and place your arms on the armrests.
4. Press and hold the rocker switch (located on the front edge of one of the armrests) to travel UP / DOWN the rail.
5. Once you have reached the top or bottom landing, release the rocker switch.
6. To rotate the seat **at the top landing**, press the rocker switch again on the same side.
7. Release the rocker switch when you have fully rotated out.
8. Unbuckle the positioning belt and carefully exit the seat.

#### **NOTES:**

- ***You may use the remote call/send transmitters to return the rotated power seat back to the ride position.***
- ***If the unit stops unexpectedly while traveling, check for obstructions or excessive loading. You will only be able to run the unit in the opposite direction until the overload is cleared.***

### POWER FOOTREST (optional)

If your unit is equipped with an optional power footrest, follow the operating instructions indicated below.

1. Raise the seat to raise the footrest to its stored position.
2. Lower the seat to lower the footrest to the ride position.

#### **NOTES:**

- ***Do not try to manually raise or lower the footrest. Always raise/lower the footrest by raising/lowering the seat. Failure to follow this instruction could damage the unit.***
- ***If a jam occurs while raising or lowering the footrest, the footrest will attempt to return to its last position. This will produce a sequence error.***

#### **To restore normal footrest operation:**


- ***Clear the jam.***
- ***Move the seat up and down until the footrest moves again.***
- ***If normal footrest operation is not restored, contact a Bruno Service Representative.***

## OPERATION: MANUAL folding rail (optional)

When folding the rail UP, the carriage **MUST** be either at the **mid-park** position or parked at the **top** position!

Damage to the carriage **WILL** occur if the carriage is below the mid-park position when the rail is folded up.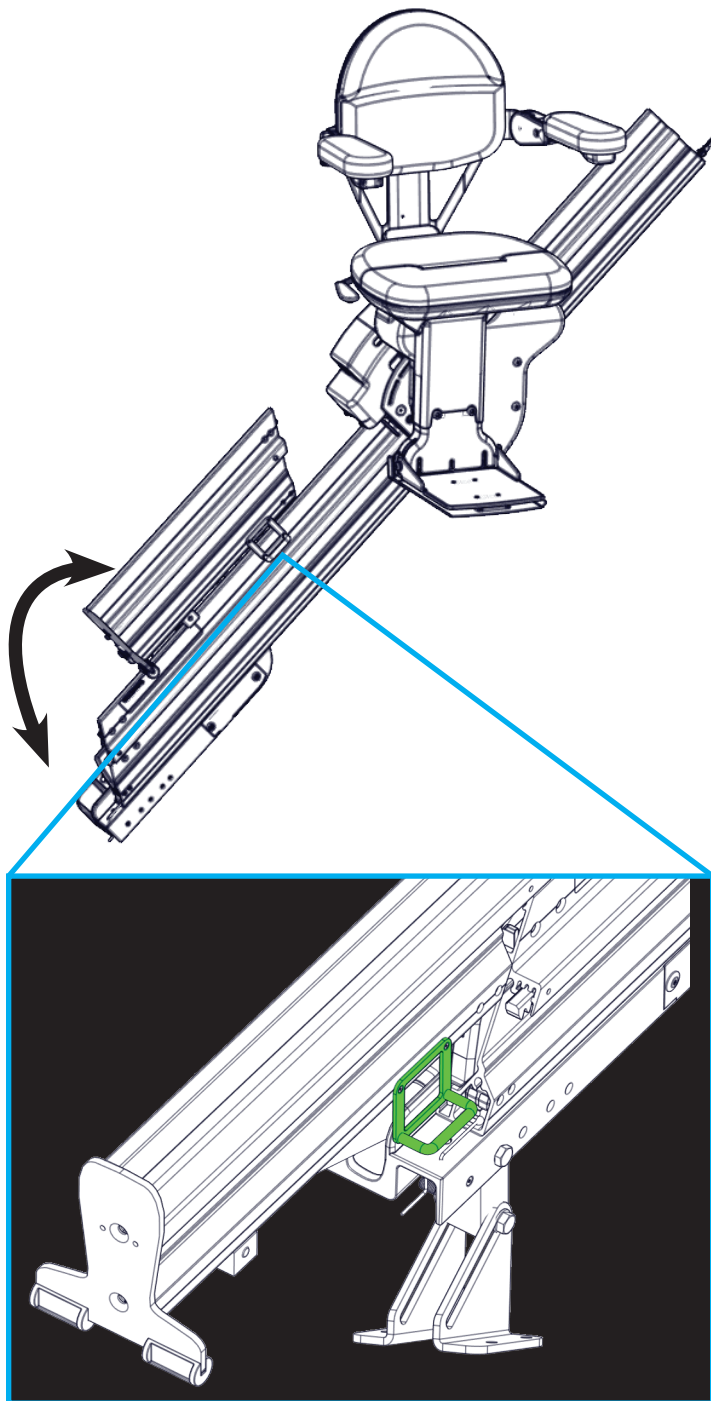

### **WARNING**

#### **PINCH POINT!**

Make sure the areas above and below the folding rail are clear before operating the folding rail.

Keep children and pets at a safe distance.

Failure to observe this warning could result in injury or damage to equipment.

#### **To fold rail UP:**

1. Using the handle, gently pull up and toward the top of the rail.
2. Using the handle, gently lower the folding rail section to the position shown to the right.

#### **To fold rail DOWN:**

1. Using the handle, gently pull up and toward the bottom of the rail.
2. Using the handle, gently lower the folding rail section to the ground.

### **WARNING**

#### **PINCH POINT!**

Make sure the areas above and below the folding rail are clear before operating the folding rail.

Keep children and pets at a safe distance.

Failure to observe this warning could result in injury or damage to equipment.

The **power** folding rail is very similar to the manual folding rail except it has a motorized actuation built into the rail hinge to automatically raise and lower the folding section of the rail.


The power folding rail includes a mid-travel stop position.

- If you are **above** this stop, the rail should always be **folded up**.
- If you are **below** this stop the rail should be **folded down**.

#### **Do not attempt to manually fold the power rail!**

Using power from the carriage, fold or unfold the rail:

- by riding the stairlift, or
- by using the remote call/send transmitters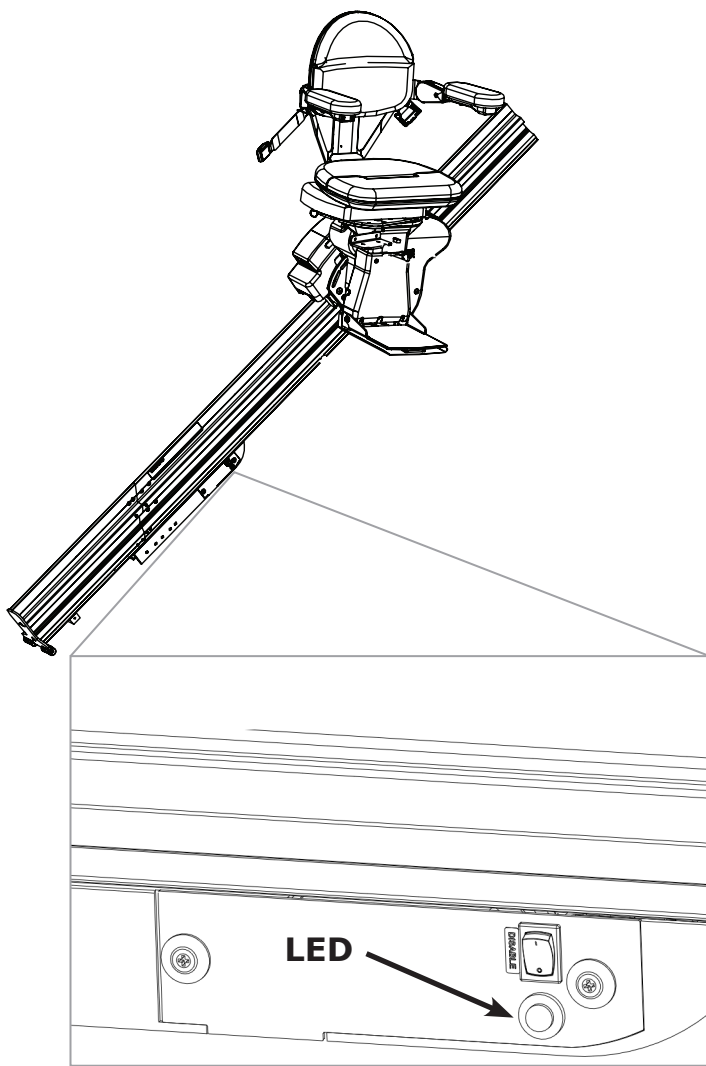

#### **Note**

You will hear a continuous beep whenever the folding rail is in motion. This is to warn people nearby of rail operation.

In addition, the yellow LED on the control panel flashes.

#### **Riding the stairlift:**

**If you are at the top of the staircase,** press the armrest switch to ride down the rail until you contact the mid-stop, then release the switch. Look down the staircase to be sure there are no obstructions to prevent the rail from lowering!

Press the armrest switch again, in the down direction. The folding rail will lower. The stairlift will not move down until the rail folds down completely and touches the ground. Release the armrest switch once the rail reaches the ground.

Once the rail is folded down, press and hold the armrest switch again until you reach the bottom of the staircase.

**If you are at the bottom of the staircase,** ride up until you contact the mid stop, then release the armrest switch. Look down the staircase to be sure there are no obstructions to prevent the rail from folding up!

Press the armrest switch again, in the up direction. The rail will rise up. The stairlift will not continue up until the folding rail is completely folded up.

When the rail is in its fully up position, release the UP armrest switch, then press it again to travel the rest of the way up the staircase.

## OPERATION: POWER folding rail (optional)

### Using the transmitters:

**If you are at the bottom of the staircase**, use the transmitter to power the stairlift down.

**If the stairlift is above the mid-stop position**, the folding rail will not lower until the chair contacts the mid-stop position.


Once the stairlift has reached the mid-stop position, press the transmitter DOWN button again to unfold the rail.

**Be sure to stand clear of the rail as it unfolds!**

**If you are at the bottom of the staircase** and want to store the stairlift with the rail folded up, use the transmitter to send the stairlift up to the mid-stop position. Then press the transmitter UP button again to raise the rail.

**Be sure to stand clear of the rail while it folds up!**

The power folding rail is equipped with safety features to reduce the chance of injury to people and pets, as well as damage to property.

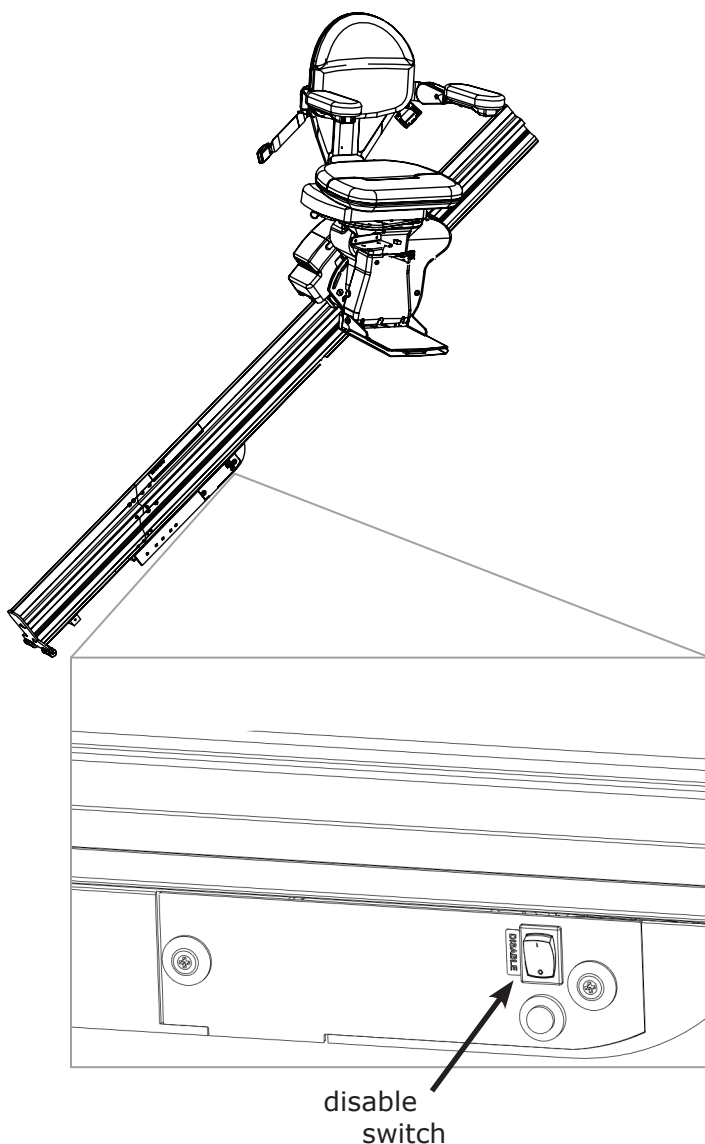If the power folding rail detects an obstruction, the rail will stop and briefly reverse direction.

To resume operation in the desired direction,

1. clear the obstruction,
2. press the armrest switch or transmitter button in the opposite direction,
3. press the armrest switch again, in the desired direction.

### **Disable switch**

The control panel includes a disable switch.

Pressing this switch ON prevents the rail from folding up or down. You would use this function if you wished to keep the rail folded down for an extended period of time.

Be sure to press the switch to OFF allow the rail to fold and unfold correctly under power!

### **IN AN EMERGENCY ONLY!**

Should you need to raise or lower the power folding rail, and you are unable to do so with the armrest switch or the transmitter, press the disable switch ON. Manually, and GENTLY, raise or lower the rail.

**CHARGING**

**Keep the batteries charged** to ensure optimal operation of your SRE-3000.

The batteries are charging whenever the carriage is parked anywhere on the rail.

Failure to keep the batteries charged, or allowing them to fully discharge, will significantly shorten the life of the batteries.

Occasionally check the battery charger LED to make sure the stairlift is being charged correctly.

**LEDS**

<b>LED COLOR/ FUNCTION</b>	<b>STATUS/PROBLEM</b>	<b>REMEDY</b>
red	AC power on	normal condition when connected to a live outlet
red (flashing)	defective batteries in unit	Contact your authorized Bruno dealer for service.
yellow	charging	status OK
green	battery fully charged/float mode charging	status OK

## TRANSMITTERS

---

The SRE-3000 hand-held transmitter may experience certain types of interference caused by portable and mobile radio frequency (RF) communications equipment. However, the SRE-3000 uses infrared (IR) communications to minimize the possibility of interference.

Also, receivers are mounted on both sides of the SRE-3000 carriage to minimize interference. Should interference occur, the unit will stop. This feature has been integrated into the SRE-3000 to ensure your safety.

The direct line between the transmitter to either of the (2) transmitters should be clear of obstacles for optimal operation. It may be necessary to reposition the transmitters so that they are aimed at the carriage.

To reduce the possibility of interference:

- While riding in the seat, **ALWAYS** operate the elevator using the rocker switch on the armrest.
- Operating the SRE-3000 with a transmitter while riding in the seat can lead to signal interference.
- **DO NOT** mount the transmitters behind an obstacle such as a rail post or around a corner.
- **DO NOT** allow direct sunlight to shine on the receivers (blinding the receivers on the carriage).
- **DO** replace transmitter batteries regularly.
- Depleted or nearly-depleted batteries alter the effective range of the transmitter.
- **DO** keep the transmitter and receiver lens free of dirt and debris.
- Use a non-abrasive cleaner suitable for glass or acrylic surfaces.
- Do not use polishes or cleaning products containing wax. These products will leave a film on the lens that will reduce the signal transmission range.

### Transmitter Battery Replacement

To replace the batteries in the remote transmitters, remove the access panel on the back of the transmitter.

Two AA **alkaline** batteries required.

***Change batteries annually.***

The SRE-3000 is designed to provide many years of reliable service with minimal maintenance.

For best service life:

- Keep the rail channel clean and free of debris.
- Keep the battery charger plugged into a live outlet at all times.
- **At least once a year**, have the stairlift examined, cleaned and lubricated by a qualified, authorized Bruno service technician.

### Seat Upholstery Cleaning

All seating and operator contact areas are fabricated of common materials without a history of causing operator reactions or irritations. The SRE-3000 has been designed assuming that the operator will be wearing clothing covering the torso and footwear. If the user has special sensitivities, normal clothing precautions are suggested.

*For day-to-day cleaning*, use a mild soap and water solution and a soft, lint-free cloth. Do not use vinyl conditioners or protectants.

*For more difficult stains*, use an all-purpose cleaner such as Formula 409® or Fantastik® and a damp, soft cloth or sponge. Rinse with clean water and dry with a lint-free cloth or towel.

## TROUBLESHOOTING

---

*Your SRE-3000 is not operating correctly. What should you do?*

*Before calling your Bruno dealer, refer to the solutions listed below. This could save you an unnecessary service call!*


- **Seat swivel.** Is the seat locked in the riding position?
- **Barrier arms.** Are both arms completely down (rider in seated position) when pressing the rocker switch?
- **Footrest.** Is there an obstruction on the stairway that the footrest is contacting?
- **Carriage safety panels.** Is there an obstruction on the stairway that the carriage safety panels are contacting?
- **Main circuit breaker (or fuse) panel of residence.** Has a breaker tripped or a fuse blown?
- **Battery charger.** A discharged battery will cause the carriage to move more slowly until the voltage drops to the point where the controller will shut off. If this should occur, pause for a moment, then run the carriage down to exit the seat.  
*Partially charged batteries will run the carriage down, but not up.*  
Confirm that the charger is plugged into a live outlet and wait for the batteries to charge. Red and green LED's are illuminated.  
Is the battery charger plugged into a functional outlet? *Do not use outlets which are controlled by a wall switch.*  
Is the RED power light on?  
Has the main circuit breaker or the residence tripped? Has a fuse blown in the electrical panel of the residence?
- **Visual check.** Do you notice anything about your stairlift that is different or out of place?

If your SRE-3000 continues to function improperly or not at all after completing the preceding checklist, contact an authorized Bruno dealer to perform all required service, repair and maintenance operations.

## VACATION AND LONG-TERM STORAGE

If the stairlift will remain unused for an extended period of time, turn off the circuit breaker (ON/OFF switch on the carriage), turn off the battery charger power switch (if equipped) and unplug the charger from the wall outlet.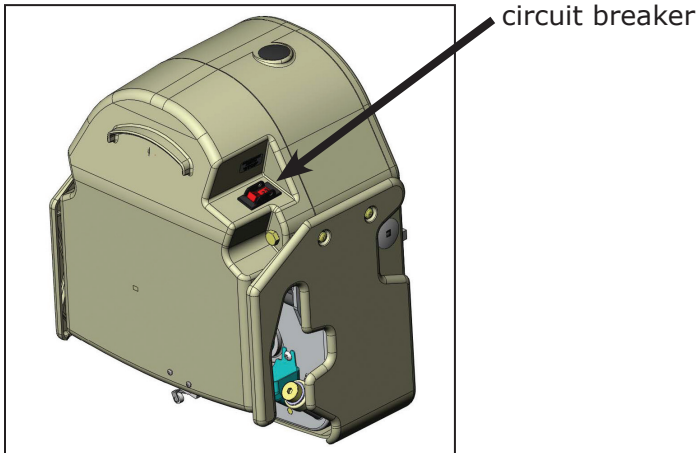

***The batteries may require recharging before normal use if the stairlift remained in the OFF position for an extended period of time.***

***To recharge, turn on the circuit breaker, reconnect the charger to the wall outlet, and turn on the battery charger power switch (if equipped). Then wait until the charger LED's indicate that the batteries are fully recharged.***

## **BRUNO GOLD LIMITED WARRANTY FOR STAIRLIFT 5 Years Major Components – 2 Year Parts**

Bruno Independent Living Aids, Inc.© (Bruno) is so confident in our USA designed and built units that we provide every original buyer with the following "Industry Leading" warranty for your peace of mind.

The quality of parts we use and the care our employees take allow Bruno to provide the original buyer of its Bruno Stairlift with a **2-year warranty** that all parts are free of defects in material and workmanship.

Because of the reliability found in Bruno's Stairlift product Bruno is able to provide the original buyer with an extended **5-year warranty** of the Major Components of the product, to include the motor/gearbox drive train and rail system. Bruno's warranty periods begin with **the date of the installation** of the Bruno Stairlift.

Bruno Authorized Dealers are trained to evaluate your unique situation and then install the best Bruno product to meet your needs. The trained installation and service technicians employed by Bruno Authorized Dealers have the appropriate equipment and resources needed to correctly install and maintain your new stairlift and are the first contact for any warranty or service needs.

### **SPECIFICS OF THE WARRANTY COVERAGE:**

- ◇ All Parts are free of defects in material and workmanship for a period of two years after the date of installation.
- ◇ All MAJOR COMPONENTS are free of defects in material and workmanship for a period of five years after the date of installation.
- ◇ All labor to address any product defect by a Bruno Authorized Dealer is covered for the first 30 days after installation.

### **WHAT IS NOT WARRANTY COVERAGE:**

- ◇ Damage to units caused by misuse, abuse, accidents, neglect or modifications NOT made by a Bruno Authorized Dealer.
- ◇ Consumable items such as remote batteries.
- ◇ All labor by a Bruno Authorized Dealer after the first 30 days since installation.
- ◇ Reasonable wear and tear.

### **ADDITIONAL NOTES:**

All Service and Warranty issues are handled through the Bruno Authorized Dealer that installed the unit.

- ◇ In no event shall Bruno be responsible for indirect, incidental or consequential damages, whether such damages arise based on claims based on contract, warranty, tort (including negligence, strict liability or product liability).
- ◇ Bruno reserves the right to replace, repair or, if not commercially practical and/or feasible, to elect to refund the purchase price at its sole discretion.
- ◇ This warranty gives the original buyer specific legal rights, and you may also have other rights which vary from state to state. Bruno specifically does not authorize any person to extend the time or scope of this warranty.

For more detailed information regarding this limited warranty, please contact Bruno by writing:

Bruno Independent Living Aids, Inc.  
Attn: Service Department  
1780 Executive Drive  
Oconomowoc, WI 53066 USA  
or call: 1-800-882-8183 toll free

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**Bruno Independent Living Aids, Inc.<sup>MD</sup>  
Attention: Service Department  
1780 Executive Drive  
Oconomowoc, WI 53066 É.-U.**

# First in Performance. Built to Last!

**Straight and  
Custom  
Curved-Rail  
Stairlifts**

**Over 25  
Different  
Vehicle Lift  
Solutions**

**Vertical Platform Lift**

**Valet® Signature Seating**

# **BRUNO**®

for your independence

Bruno Independent Living Aids, Inc.<sup>®</sup>  
1780 Executive Drive, Oconomowoc, WI 53066  
262-567-4990 · FAX: 262-953-5501 · [www.bruno.com](http://www.bruno.com)

©Bruno Independent Living Aids, Inc. All illustrations and specifications in this document are based on the latest product information available at the time of publication. Bruno Independent Living Aids, Inc. reserves the right to make changes at any time without notice. Manual Back Cover 09/13.

P/N 3000-O