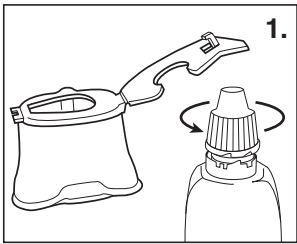
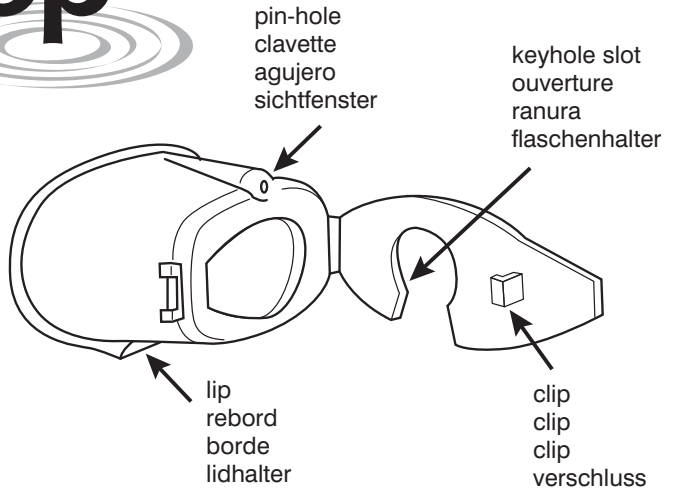
# AutoDrop®

OP6000

## Instructions Mode d'emploi Instrucciones de uso Gebrauchsinformation

AutoDrop® has been developed to assist in the instillation of eye drops.  
AutoDrop® a été conçu pour aider à la distribution des gouttes oculaires.  
AutoDrop® se ha diseñado para facilitar la aplicación de gotas para los ojos.  
AutoDrop® ist die optimale Applikationshilfe für Augentropfen.

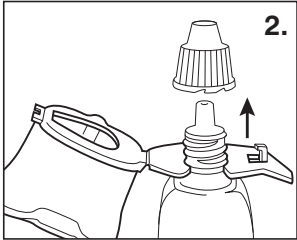
Open AutoDrop. Loosen cap from eye drop bottle but do not remove completely.

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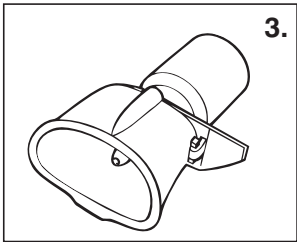
Locate the keyhole slot around the base of the bottle neck below the thread. Remove bottle cap.

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Close AutoDrop. Do not force the bottle nose into AutoDrop, it should close easily.

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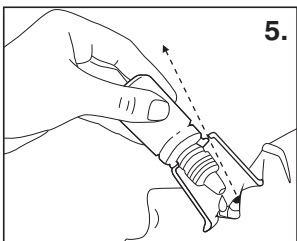
Hold down lower eyelid. Place AutoDrop over the eye with the lip against the cheek.

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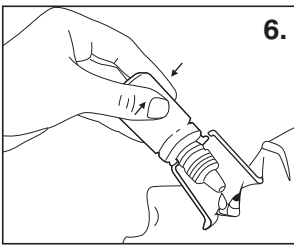
Tilt the head backwards. Look through pin-hole towards the light.

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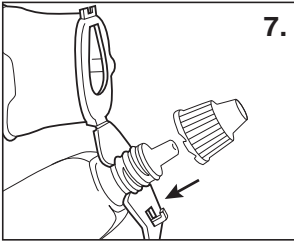
**6.** Squeeze bottle gently to allow prescribed drops to fall into the eye.

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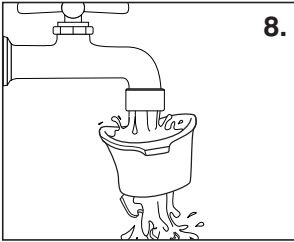
**7.** Open AutoDrop and replace top on eye drops bottle between doses.

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**8.** To clean AutoDrop remove eye drop bottle and wash AutoDrop in warm soapy water. Rinse and dry thoroughly.

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We recommend that AutoDrop is used for the whole course of treatment for an eye disorder. When an eye drop bottle is discarded remove AutoDrop and clean thoroughly before fitting to the new bottle. AutoDrop may be used on more than one type of eye drop at the same time but if you suffer from more than one eye complaint we recommend that you use a separate AutoDrop for each type of eye drop. In hospitals and clinics: AutoDrop may be autoclaved on a low pressure cycle for 30 minutes. Release clip to allow moisture to evaporate. AutoDrop can also be cleaned in sterilising fluid. Rinse and dry thoroughly.

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**GUARANTEE:** AutoDrop is guaranteed for 3 months from purchase. If AutoDrop breaks within this period a full refund will be given.

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Made in England by

 Developed in collaboration with the RNIB

 **OWEN MUMFORD**

AutoDrop® is a trademark of Owen Mumford Ltd

[www.owenmumford.com](http://www.owenmumford.com)

 **HEAD OFFICE INTERNATIONAL SALES & MARKETING**  
Owen Mumford Ltd  
Brook Hill, Woodstock  
Oxfordshire OX20 1TU,  
United Kingdom  
T: 01993 812021  
F: 01993 813466  
E: [info@owenmumford.co.uk](mailto:info@owenmumford.co.uk)

**UK / IRELAND SALES & MARKETING**  
Owen Mumford Ltd  
Primsdown Ind. Estate  
Worcester Road, Chipping Norton  
Oxfordshire OX7 5XP  
United Kingdom  
T: 01608 645555  
F: 01608 646546  
E: [info@owenmumford.co.uk](mailto:info@owenmumford.co.uk)

**NORTH AMERICA**  
Owen Mumford Inc.  
1755 West Oak Commons Court  
Marietta, Georgia 30062, USA  
T: 800-421-6936  
F: 770-977-2866  
E: [info@owenmumfordinc.com](mailto:info@owenmumfordinc.com)

**FRANCE**  
Owen Mumford  
BP 3333 27950  
Saint Marcel  
France  
T: 02 32 51 88 70  
F: 02 32 21 96 95  
E: [contact@owenmumford-fr.com](mailto:contact@owenmumford-fr.com)

**GERMANY**  
Owen Mumford GmbH.  
Alte Häge 1 63762  
Großostheim-Ringheim  
Germany  
T: 06026/97750  
F: 06026/977529  
E: [info@owen-mumford.de](mailto:info@owen-mumford.de)