**PRODUCT REORDER NUMBER**

**AeroTrach Plus®** aVHC

52510

**Directions for Use (Single patient Use Only)**

	<p><b>ENGLISH</b></p> <p>Important: Follow instructions that came with inhaler (MDI).</p>
<p>1</p>	<p>Remove cap from MDI.</p>
<p>2</p>	<p>Look for foreign objects.</p>
<p>3</p>	<p>Insert inhaler (MDI) into back.</p>
<p>4</p>	<p>Carefully connect <b>AeroTrach Plus®</b> aVHC to tracheostomy tube connection.</p>
<p>5</p>	<p>Depress inhaler (MDI) at beginning of slow inhalation. Hold in place for 4–6 breaths after depressing inhaler.</p>
<p>6</p>	<p>Wait 30 seconds before repeating steps 4–5 as prescribed.</p>

- aVHC = anti-static Valved Holding Chamber
- Do not leave **AeroTrach Plus®** aVHC unattended with children. This is a medical device, not a toy.

Manufactured for:  
 Monaghan Medical Corporation, Plattsburgh, NY 12901  
 Manufactured by:  
 Trudell Medical International 725 Baransway Drive, London, ON, Canada N5V 5G4

REGISTERED  
**ISO 13485**  
 QUALITY SYSTEM

*anti-static Valved Holding Chamber*

**R<sub>x</sub> ONLY**

**Cleaning Instructions**

	<p><b>ENGLISH</b></p> <p>Clean <b>AeroTrach Plus®</b> aVHC only as per instructions, daily or more often as needed.</p>
<p>1</p>	<p>Remove back piece only.</p>
<p>2</p>	<p>Soak both parts for 15 minutes in lukewarm water with liquid detergent. Agitate gently.</p>
<p>3</p>	<p>Rinse in clean water.</p>
<p>4</p>	<p>Shake out excess water. <b>Do not rub dry.</b></p>
<p>5</p>	<p>Let air dry in vertical position.</p>
<p>6</p>	<p>Replace back piece when unit is completely dry and ready for use. Store device in bag provided.</p>

**Notes**

- Do not disassemble the product at any time. (even for cleaning, except for flexible back piece).
- Do not boil or autoclave.
- **This product contains no natural rubber latex.**
- Life expectancy of the device approximately 3–4 months with proper care and maintenance.
- This device does not deliver supplemental oxygen.

**Disinfection**

- After the clean water rinse the following may be safely used: Enzyme Solvents, Vital-Oxide<sup>†</sup>, 1 part bleach/10 parts water solution. These should be soaked for 30 minutes, followed by a clean water rinse.

**Cautions**

- Product should be replaced if valve is damaged or missing.
- If unsure on how to use this product, talk to your physician or health care professional.
- Administer one puff at a time.
- **AeroTrach Plus®** aVHC is designed for use only with 15mm tracheostomy tubes.
- Only to be used on spontaneously breathing patients.
- This device is designed as a slip fit onto the tracheostomy tube connection. It must be held in place during administration of medication.
- **RX only**

**Warning.** Remove the device if patient shows signs of difficulty.

**Indications for Use:** The **AeroTrach Plus®** anti-static Valved Holding Chamber is intended to be used by patients who are under the care or treatment of a licensed health care professional or physician. The device is intended to be used by these patients to administer aerosolized medication from most pressurized Metered Dose Inhalers, prescribed by a physician or health care professional. The intended environments for use include the home, hospitals and clinics.

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Fabricado por:  
Monaghan Medical Corporation, Plattsburgh, NY 12901

Fabricado para:  
Trudell Medical International 725 Baransway Drive, London, ON, Canada N5V 5G4

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