

# Insertion Technique for Flexible Partial

1. Examine the partial carefully on and off the master model and check the path of insertion.
2. Submerge the TCS partial denture in very hot tap water for approximately one minute.

**The Hot Water Treatment is very important as it allows for a smooth initial insertion and a good adaptation with your patient's natural tissue. (This is only recommended for 1<sup>st</sup> insertion).**

3. When the partial cools to a tolerable temperature, gently insert it into the patients mouth.

**You may choose to spray the partial with TCS Cool Mint Spray prior to initial insertion for a minty taste experience.**

The **TCS Cool Mint Spray** is also available for patients to use as an on-the-go freshener. Also used by dentists for try-ins and initial insertion for a minty fresh taste experience. (1 fl oz. yields ~200 misty sprays)

