

Karadent™

Semi-rigid microcrystalline polymer for full dentures, semi-flexible partial, clear clasps, frameworks & nightguards



- › Highly aesthetic
- › BPA & monomer free
- › Maximum strength and durability
- › No volume shrinkage-exceptional fit
- › Minimal water absorption
- › Quick and easy chairside adjustments
- › Easy to repair, rebase, and reline with acrylic





BENEFITS

Economical, aesthetic solution for missing teeth
Metal-free
Ideal for patients allergic to monomer or nickel
Used as a transitional restoration during the healing period for implants
An upgrade from acrylic dentures



TIPS

1. For best results use lightweight alginate impression materials.
2. Adjusting Karadent™ is quick and easy. You can adjust using the same techniques as acrylic dentures.
3. Use a carbide bur for making small adjustments and Green Mounted Stones to smoothen area.
4. For a touch-up polish, Green Hi-Shine polishing paste (item #4209-02) is recommended.
5. Recommend TCS Dental Appliance Cleaner to your patients for optimal maintenance.



SHADES



Standard Pink | Clear



INSURANCE CODES

D5225 Maxillary Partial Denture
D5526 Mandibular Partial Denture
D5110 Maxillary Complete Denture
D5120 Mandibular Complete Denture



clear framework



nightguard

Combines the unbreakable strength of nylons with the simplicity of acrylics to create what technicians and dentists have been waiting for in removable appliances!

Specify Karadent™ on your Lab Slip

