

iFlex™

The easy to use thermoplastic



- › Polyolefin thermoplastic
- › Highly aesthetic
- › Clinically unbreakable
- › **Easy chairside adjustments**
- › **Stain free**
- › Biocompatible
- › Lightweight yet durable
- › Ideal degree of flexibility
- › High translucency
- › Repairable and rebasable





BENEFITS

- Ideal for partial dentures and unilaterals
- Economical, aesthetic solution for missing teeth
- Metal-free
- Gingival veneers to mask exposed root surfaces
- Ideal for patients allergic to monomer or nickel
- Used as a transitional restoration during the healing period for implants
- Presence of tori, oral cancer, cleft palate



TIPS

1. For best results use lightweight alginate impression materials.
2. Use carbide burs for making small adjustments and TCS Brown Rubber Points to smooth adjusted areas. Use light pressure so you don't gouge the material.
3. Soak the partial in very hot clean tap water for about 1 minute before initial insertion. When case cools to a tolerable temperature gently insert it into patient's mouth.
4. Recommend tcs[®] Dental Appliance Cleaner to your patients for optimal maintenance.
5. Use the TCS Animation Video (1.5 min.) & Health Briefs with Terry Bradshaw (5 min.) for patient chairside education.
Visit www.tcsdentalinc.com to download videos.



SHADES



Light Pink | Standard Pink | Light/Dark Pink | Dark Pink | Natural (clasps)



INSURANCE CODES

- D5225 Maxillary Partial Denture
- D5526 Mandibular Partial Denture

"After many years of wearing an uncomfortable acrylic partial denture that would constantly break, my dentist prescribed me an iFlex™. I can finally smile and eat without any worries."

~Ann, 36 years old

Specify iFlex™ on your Lab Slip

