

How Airtraq simplifies Video Intubation

- Airtraq's channel simplifies ETT insertion reducing intubation time
- No stylet needed. Avoids potential injuries.
- Eliminates stylet costs
- 90% shape minimizes hyperextension and reduces force required
- Reduces hemodynamic reactivity
- Facilitates intubation from any position (face to face etc)
- High intubation success rate in difficult airways
- Easier to learn than other video laryngoscopes

Visualisation Options for any Airtraq model



Product code	Description	ETT size	Mouth opening	Units per box	NHSSC code
SP					
A-011	Airtraq SP Regular (size 3)	7.0 - 8.5	16mm	6	FDS2530
A-021	Airtraq SP Small (size 2)	6.0 - 7.5	15mm	6	FDD2531
A-031	Airtraq SP Paediatric (size 1)	4.0 - 5.5	11.5mm	6	FDD2532
A-041	Airtraq SP Infant (size 0)	2.5 - 3.5	11mm	6	FDD2533
A-061	Airtraq SP Adult Nasal	-	15mm	6	FDD2535
A-071	Airtraq SP Double Lumen	35 - 41 Fr.	19mm	6	FDD2536
AVANT					
A-501	Airtraq Avant Optics	-	-	1	FSM6157
A-511	Airtraq Avant Regular Blade (size 3)	7.0 - 8.5	17mm	50	FSM6158
A-521	Airtraq Avant Small Blade (size 2)	6.0 - 7.5	17mm	50	FSM6159
A-590	Airtraq Avant Docking Station	-	-	1	FSM6160
ACCESSORIES					
A-307	Airtraq Phone Adapter	-	-	1	FSM6155
A-309	Airtraq Phone Protection Sleeve	-	-	100	FSM3654
A-390	Airtraq Wi-Fi Camera	-	-	1	FSM3778
A-390-KIT	Airtraq Wi-Fi Camera Kit	-	-	1	FSM3779

Where and when you need it



www.fannin.eu

Fannin (UK) Ltd.
Westminster Industrial Estate
Repton Road, Measham,
Swadlincote, Derbyshire,
DE12 7DT
T: +44 (0) 1530 514566
E: serviceuk@fannin.eu



Simple, effective,
affordable video
laryngoscopy

FANUK18-4407

Two complementary models for one complete solution



Airtraq AVANT

Reusable Optics + Disposable Blades

Minimum cost per intubation.
VL for frequent use.



Airtraq SP

Fully Disposable

One piece. Ready to use.
VL for remote locations and emergencies.



1. Slide midline around the tongue
2. Centre vocal cords
3. Push ETT
4. Remove midline



- Keep midline
- Do not insert too deep
- Lift gently
- Twist Airtraq to centre vocal cords
- Corkscrew ETT if needed

Airtraq improves routine intubations and it is specially effective for:

- Cervical immobilization
- In combination with a fibrescope
- Obese
- Frontal or lateral intubations
- Awake
- Poor dentition

Airtraq SP

GUIDED VIDEO INTUBATION
Adults
Paediatrics
Double Lumen
Nasal (no channel)



Adult regular 7.0 to 8.5 ETT M.O.*: 16mm
Adult small 6.0 to 7.5 ETT M.O.*: 15mm
Double lumen 35 to 41 Fr. M.O.*: 18mm
Adult Nasal M.O.*: 15mm

NEW WiFi Camera



iOS / Android
Mac / Windows



Paediatric 4.0 to 5.5 ETT M.O.*: 12mm
Infant 2.5 to 3.5 ETT M.O.*: 11mm

Adult regular 7.0 to 8.5 ETT M.O.*: 17mm
Adult small 6.0 to 7.5 ETT M.O.*: 17mm



Reusable optics Disposable blades Docking

Phone Adapter



iOS / Android



Airtraq Avant