

# QuickMove

SystemRoMedic™

**Small & Convenient**

**Safeguards and  
strenghtens the user's  
abilities**



## **Safe and secure sit-to-stand and transfer**

QuickMove is a safe and easy-to-use aid for sit-to-stand and transfer of users with impaired balance and reduced lower limb strength. QuickMove supports the user when rising, using his/her own strength and then transfers the user, either standing up or sitting down, from bed to chair or wheelchair or to and from the toilet. QuickMove is an excellent transfer aid which spares the backs of the caregivers and reduces the risk of work related injuries.



# Safe and secure sit-to-stand and transfer

QuickMove supports and activates the user when rising up to a standing position and during standing or sitting transfers, from bed to chair, from chair to chair or to and from the toilet. QuickMove has been developed for users with impaired balance and strength in their legs but the user must have some strength in the arms and must be able to weightbear to actively participate in the transfer.



*QuickMove has adjustable base width which makes it very easy to get close and in the right position for the user.*



*The adjustable buttock cushions are angled to make room for the user and caregiver.*



*The user places his/her feet on the stable anti-slip foot plate, and gets support from the adjustable lower leg support.*



*The user chooses a comfortable horizontal or vertical grip and rises using his/her own strength. When the user has risen using his/her own strength, the caregiver or the user folds down the cushions behind the buttocks, to provide a stable support.*



*The user then has the option to remain standing or to be seated during the transfer. The support of the cushions behind the buttocks and the knee-brace facilitates a very stable and safe transfer.*



*When the user is rising again, the carer can, with a push on the handle, angle the cushions to assist in achieving a standing position.*

# Safeguards and strengthens the user's abilities

QuickMove provides a more secure sit-to-stand procedure for both user and carer, and, at the same time, it contributes to safeguarding and strengthening the user's abilities. QuickMove is suitable both for long-term and for occasional needs, for example in post-operative care when balance is temporarily impaired.

## Easy to use, everywhere

QuickMove is made of non-corrosive materials. It's also compact and convenient and very easy to maneuver which makes it very useful in most spaces and situations.

QuickMove has a stable construction and can manage users weighing up to 375 lbs.



# QuickMove

## Product specification:

QuickMove \_\_\_\_\_ Art. No. 400801334

### TEKNISK INFORMATION

Material: \_\_\_\_\_ Powdercoated steel  
 Length: \_\_\_\_\_ 98 cm/39.2"  
 Width: \_\_\_\_\_ 60,5 cm/24.2"  
 Height: \_\_\_\_\_ 112 cm/44.8"  
 Base width: \_\_\_\_\_ 51-90 cm/20.4"-36"  
 Castor size: \_\_\_\_\_ Ø 75 mm/3"  
 Weight: \_\_\_\_\_ 35 kg/77 lbs  
 Safe working load: \_\_\_\_\_ 170 kg/375 lbs



### Accessories for extra support and security

BackSupport is a new accessory which can be used as torso/side support. BackSupport provides extra stability and security for users during transfers with QuickMove.

## BackSupport

SystemRoMedic™



Art.no	Description
70200037	BackSupport, accessory for QuickMove



SystemRoMedic™ is a complete solution for easy transfers. The concept encompasses four product areas that meet most needs in different transfer situations.

**Transfer**, between two locations. **Positioning**, within the same location.

**Support**, for mobility. **Lifting**, both manual and mechanical.



For more QuickMove videos visit  
[www.handicareproducteducation.com](http://www.handicareproducteducation.com)

