

RJ Cooper & Associates, Inc.
22600-A Lambert St., Suite 708
Lake Forest, Ca 92630
1-800-RJCooper 949-510-2620 Fax: 949-582-3169
Internet: www.rjcooper.com Email: infoRJ@rjcooper.com

Say-It Talking Button

Insert 3 AAA batteries, turn it on, record a message, and you're good to go! Super-simple.

Recording your message

- 1) Press/hold the button for 5 seconds until you hear a beep.
- 2) Continue holding while you speak/record your message (or music). (If you're a male, speak in a slightly higher voice than normal)
- 3) Let go within 20 seconds and you'll hear a second beep.

Switch Input (if ordered)

- 1) Plug external switch into port. If the switch has a control box or battery, you might need to use the included Polarity Reverser cable.

Switch Output (if ordered)

- 1) Plug included 'patch' cable into port on Button.
- 2) Plug other end into device/toy.
- 3) If that doesn't work, use the included Polarity Reverser cable.

Mounting Kit (if ordered)

- 1) Apply 2 drops of super-glue to the *outside* of the ridges surrounding each rubber 'foot' on the bottom of the Button.
- 2) Attach the plastic disk, with washers facing the button, to the bottom of the button, aligning the washers with the feet. The washers will contact the super-glue and form a permanent bond.
- 3) Use the enclosed Super-Velcro to attach the disk/button to one of our Arms with disk, or a table or laptray.
- 4) The disk is attached to the washers by our Super-Velcro. Use a credit card to separate the Velcro's.
- 5) View a video of this process at <www.youtube.com/watch?v=rNTkcvVjF1I>.

That's it! Enjoy!