


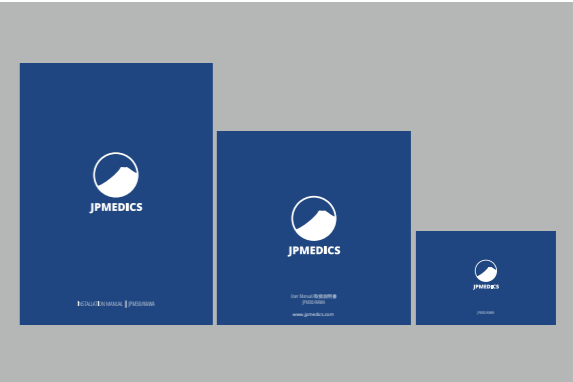

The JPM30/KAWA massage chair arrives in 2 boxes.



1 **Open box 1 and remove all contents.** Remove the packing materials. Take the chair out of the box. Collect all the accessories and put them aside.




2 **Open box 2 and take the footrest out.** Get rid of packing materials. Move the footrest close to the chair. Connect wire connectors and air hose connector.




3 **Fully connect the footrest to the base.** Hang the footrest on bolts of the base and lock with the two catch pins.



4 **Zip the zipper between seat pad and footrest cushion.**

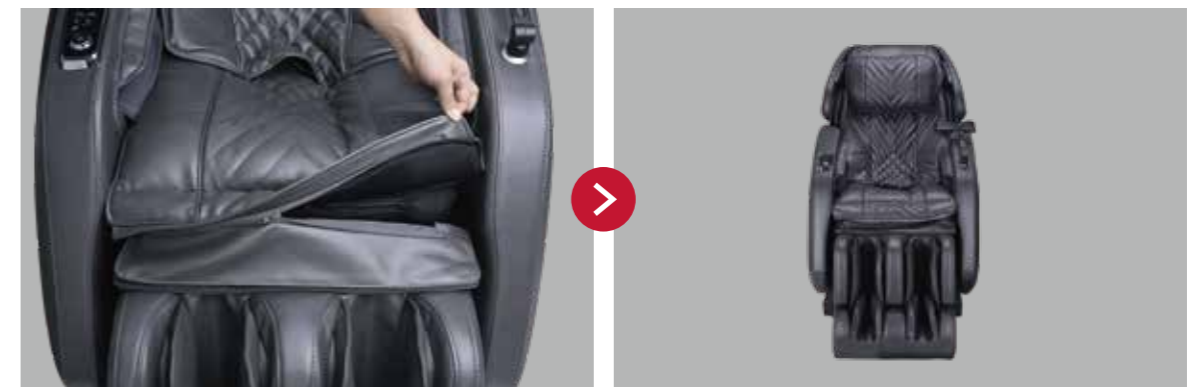

- 5 **Attach the remote control onto tablet holder.** Take the remote control out of box. Then attach it onto the tablet holder.

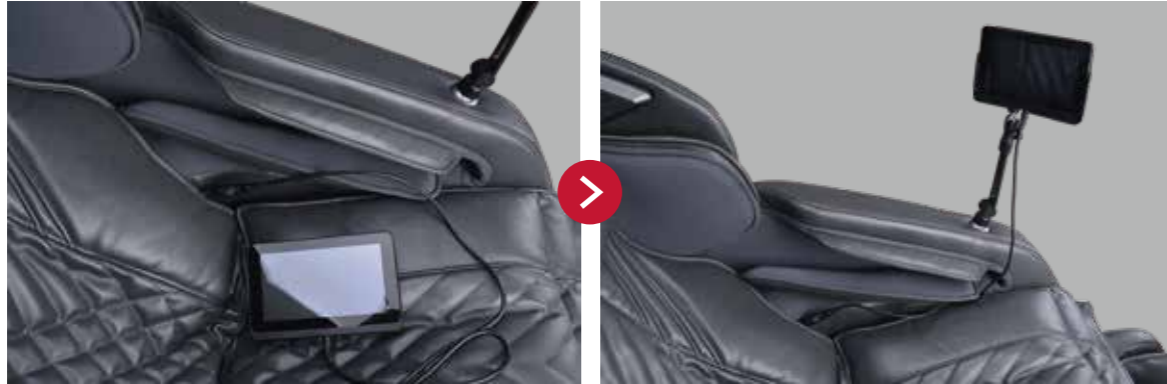

- 6 Connect the chair to a power supply and turn on the power switch. Now your massage chair is ready to go!

