

Power Lift & Recline Chair

OWNER'S MANUAL AND LIMITED LIFETIME WARRANTY



UC559
Power Lift and Recline

UltraComfort[™]
AMERICA

PROUDLY MADE



BY AMERICANS

with Domestic & Imported Materials

401 Bridge Street
Old Forge, Pa. 18518
Phone 866-793-5100
Fax 866-255-9962
www.ultracomfort.com

MSWARB-UC-06.21.2021

Dear Valued Customer,

Thank you for purchasing this UltraComfort recliner. As a family-owned and operated company, we are proud to be the leading manufacturer of power lift recliners in the furniture industry. Our hope is that with this recliner, you will not only have a favorite seat in the house, but that you will also gain a new level of comfort with our patented wellness positioning and lift options.

We take great pride in building a quality product with innovative designs and features to support your lifestyle. Please take a few moments to read through this manual and become familiar with the operation of this recliner and the warranties that support it. To learn more about your recliner and other UltraComfort products, visit our website at www.ultracomfort.com.

Also, please note the following safety information: This power lift recliner is a motor operated appliance approved for household and commercial use designed to help you sit down and stand up. Do not allow children to operate the chair at any time.

Thanks again for choosing UltraComfort America.



Sincerely,

A handwritten signature in black ink that reads "Richard Golden". The signature is fluid and cursive.

*Richard Golden
Chief Executive Officer*



Complies with
UL 73 and
CSA C22.2 No. 68

Safety Guidelines

The symbols below are used in the owner's manual and/or on the product to identify warnings and notify you of important safety information. Make sure you read and understand them completely.



WARNING! Indicate(s) a hazardous situation which, if not avoided, could result in death or serious injury. This icon is represented as a black symbol on a yellow triangle with a black border.



MANDATORY! Failure to perform mandatory actions could cause personal injury and/or equipment damage. This icon is represented as a white symbol on a blue dot with a white border.



PROHIBITED! Indicate(s) an action(s) that should not be performed under any circumstance. Failure to adhere could cause personal injury and/or equipment damage. This icon is represented as a black symbol with a red circle and red slash.

Safety Symbols

The symbols below identify warnings, mandatory actions, and prohibited actions. They can be found in the owner's manual and/or on the product. Make sure you understand all safety labels before operating the product. **Do not** remove safety labels from your product.



Read and follow owner's manual!



Child and/or Pet Crush Hazard!



Pinch/Crush Point Hazard!



Transformer Crush Hazard!



Adult, Child and/or Pet Tip Over Hazard!



Fire Hazard!



continued from previous page


Safety Guidelines



Indoor use only!



Do Not Smoke!

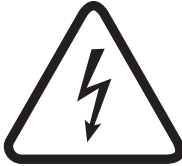

Electrical hazard!



Do not dispose in trash!



Batteries required!



MANDATORY

READ THIS MANUAL FIRST BEFORE INSTALLING OR OPERATING YOUR POWER LIFT AND RECLINE CHAIR.

Your power lift and recline chair has been engineered from the leg levelers up to provide the ultimate in strength and reliability. Great care has been taken to provide you with total comfort and safety. UltraComfort thoroughly inspects and tests each unit prior to shipment.

⚠ WARNING



Care should be taken in the unpacking of this unit, so as not to cause injury to persons or damage to the unit. Dispose of all plastic bags and covers, as they can be a smothering hazard to small children.



Do not allow children or pets near the chair while the chair is operating.



Do not allow children to play with or operate the chair at any time.



Hand control should be unplugged and placed out of the reach of children when the chair is not in use.



Do not flop down into the chair when getting into it, or while it is in its lift position. Flopping down into the chair will damage it and could cause personal injury.



Do not allow anyone to sit on your lap, or on the arm of the chair, or on the footrest while in or operating the chair, as it may cause tipping and could result in personal injury.



Do not place fingers or hands between the seat cushion and the inside arm of the chair while sitting in the chair or when the chair is in operation or plugged into a power source.



Do not retrieve fallen objects from inside or under the chair while the chair is plugged into a power source.



Do not operate the chair if there are any persons, pets or other obstructions under or in front of the footrest, under the chair or behind the chair.



Do not position yourself, child and/or pet too high on the chair while the chair back is reclined. This can cause a tip over hazard. Make sure you are properly fitted for the lift chair prior to operating the chair.



Do not smoke while operating, sitting or resting in the chair.



To reduce risk of electrical shock, do not immerse any of the electrical components into a liquid, or spill any liquids onto electrical components. Do not open any of the electrical components. There are no serviceable parts.



Warranty will be void if any of the electrical components are opened by other than a qualified service person.



Make sure the chair is placed at the specified distance from a wall/corner (page 8). Failure to do so can damage the chair and will void the warranty.



Do not use any type of electrical extension cord to plug your chair into an electrical outlet. Make sure that all electrical products and their wires are positioned away from your chair as the metal lift frame mechanism could crush them and create a fire hazard and/or may cause personal injury.



Do not operate your lift chair while intoxicated or taking medications that impair your mental or physical abilities to operate the chair safely. Consult with your doctor about any medications you are taking that may affect the safe operation of the chair.



Do not operate this product if you have any medical conditions that result in limited, debilitating mental or physical capacity. You should consult your doctor before using this product.



Consult with your physician regarding using the Trendelenburg position.



Placing the lift chair in the center of a room or too far away from a wall will create a tripping hazard, because the power wires for the lift chair will be crossing main traffic areas in the residence.



Leakage from the 9 VDC backup batteries can result in property damage. Periodically inspect and replace the batteries. Read the “Battery Back Up System” and “Care and Maintenance” sections in this manual.



If the lift chair user is unconscious or sleeping the hand control may inadvertently become activated in a way that may force the user out of the chair.



Should you ever encounter a problem with the operation of your power lift chair, discontinue all use of the product upon discovery of the problem. Contact the provider from whom you purchased your chair to arrange service.



Exceeding the stated weight limit listed in this manual can result in instability of the lift chair, causing a potential tip over hazard.



Consult the owner's manual of the chair and a physician to determine if you are capable of moving or lifting the chair, as this will cause serious injury if not done properly.



Verify that all objects including hands and fingers are not between the arm of the chair and the open compartment before closing the compartment as this will result in property damage or serious injury.



This is an electrical device and will be impacted by any power failure.



To operate this chair properly, you must read and understand the intended use of the product, and overall operation of the chair as misuse, or lack of proper training may lead to serious injury.




Use extreme care when unpacking, assembling, and disassembling of this unit, so as not to cause injury to persons or damage to the unit.

MAKING POWER CONNECTIONS IN YOUR NEW LIFT CHAIR

Models UC214, UC332-M, UC332-L, UC480, UC520, UC536-MLA, UC542-SMA, UC542-M, UC542-L, UC544-SMA, UC544-MED, UC544-LAR, UC549-SMA, UC549-MED, UC549-LAR, UC550-JPT, UC550-M, UC550-L, UC550-M26, UC556-M, UC559-SMA, UC559-MED, UC559-LAR, UC559-M26, UC790, UC798-MLA

1. Remove AC/DC adapter (transformer) from packaging located under base of chair. Place two 9 volt alkaline batteries (not included) into the slots provided as shown. Place transformer on floor behind the chair.
2. Locate the power cord from the motor. It is attached to the motor shaft under the chair with a white zip tie. Cut white zip tie, extend the cable under and out behind the chair and connect to the transformer as shown below.
3. Make sure all connections are tight. Plug the adapter wire into a standard (115-120V A.C.) wall outlet.

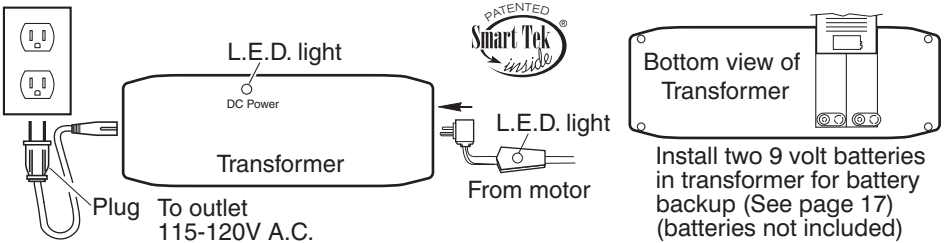

CHAIR LOCATION

You will want to place your chair in the room where it will be most convenient for you, such as near the television, radio, reading light, end table, or other furniture. Leave enough space between back of chair and wall or other obstruction to allow chair to recline (about 18 inches for standard models, 25 inches for Stellar Comfort models). Recline space needed may vary based on chair model.


Do not allow any objects to restrict movement of the footrest. Some carpets have a high plush pile that can rub on the bottom of the footrest. This situation can damage the mechanism. You can correct this situation by turning the levelers (located under the front of the chair) to raise the footrest enough to clear the carpet.

Your chair is equipped with a battery back up source. Two 9 volt alkaline batteries must be installed. Once used, batteries must be replaced. Keep the telephone or some means of communicating with someone within reach.

MAKING POWER CONNECTIONS IN YOUR NEW LIFT CHAIR

Model UC542-ME6, UC549-M26

1. Install two 9 volt alkaline batteries in the inline battery holder as shown. The battery holder is zip tied (white) to the rear of the metal frame. Cut the white zip tie and extend the cable under and out behind the chair.

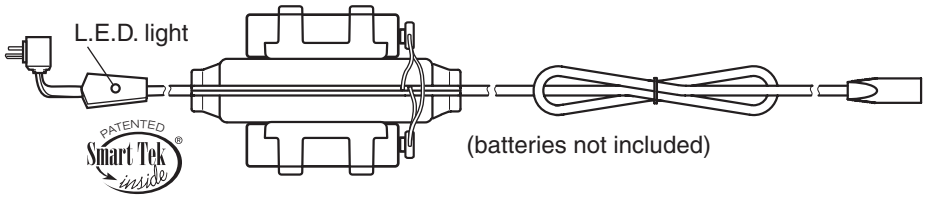

2. Remove the transformer from the transformer box follow steps 1, 2 and 3 on page 8 to connect the transformer to the chair and AC outlet.
3. Pull the hand control out of the magazine pocket and secure the cord into the quick release cable clamp located above the pocket on the arm of the chair.
4. Place the chair in the desired position in the room, leaving the chair a safe distance from the wall. See chair location on page 8 for the proper distance from the wall.
5. Adjust the leg levelers on the bottom of the chair if necessary. Some high pile carpets can obstruct the footrest operation by rubbing against it. Adjust the levelers to a height above the pile.

POWER LIFT CHAIR OPERATION

Models UC214, UC332-M, UC332-L, UC480, UC536-MLA, UC542-SMA, UC542-M, UC542-L, UC542-ME6, UC544-SMA, UC544-MED, UC544-LAR, UC549-SMA, UC549-MED, UC549-LAR, UC549-M26, UC790

- 1. To recline your chair**, press the down button (S2) and hold until you reach your desired position. Then release the button.
- 2. To return to a seated position from recline**, press the up button (S1) and hold until the chair returns to a closed seated position. Then release the button.
- 3. To lift your chair**, press the up button (S1) and hold until you reach your desired height. Then release the button.
- 4. To return to a seated position from lift**, press the down button (S2) and hold until the chair returns to the floor, seated position. Then release the button.
- 5. To lift your chair from a reclined position**, press the up button (S1) and hold. The chair will keep going past the seated position and start to lift. Release the button when the chair reaches the desired height.
- 6. To recline your chair from the lift position**, press the down button (S2) and hold. The chair will keep going past the seated position and start to recline. Release the button when the chair reaches the desired reclined position.
- 7. Getting into your lift chair.** While standing next to your chair, press the chair up button (S1) and raise the chair to a height you can sit against. This height will vary from person to person depending on the individual's needs. Raise the chair high enough so you do not have to flop down into the chair. **Flopping down into the chair will damage it and could cause personal injury.** Press the chair down button (S2). As the chair continues downward, adjust yourself into a position that is comfortable to you. Continue pressing the button until chair is in the seated position (chair firmly on the floor), then release the button.
- 8. Getting out of your chair**, press the chair up button (S1) and raise the chair to a height that enables you to easily stand. Release the button. Return the chair to the seated position (chair firmly on the floor) after getting out of the chair so children, pets or obstructions do not get under the chair.

Note: Your UltraComfort power lift chair can be stopped at any time during lift or recline by releasing the selected button, offering you a wide range of comfortable positions. Select the position that best suits your needs. The motor is set to shut off when it reaches its fully extended or fully retracted position if the selected button is not released. If you should go past your desired position, select the button to bring you back to your desired position. You can repeat the process as often as you like.

USB Disclaimer: The supplied USB charging port is only intended to be used with appropriate smart devices such as phones or tablets. Please refer to device manufacturer's charging requirements before using. UltraComfort excludes liability for any damages or data loss regarding the charging device on a computer or other electronic device connected to the USB port.


1AMP USB
Charging Port

L.E.D.
light

S1

S2


PATENTED
Smart Tek
inside

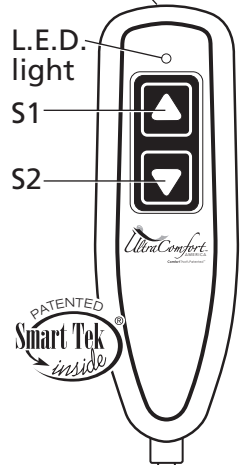

CHAIR OPERATION

Models UC550-M26, UC559-M26

- 1. To recline the back of the chair,** press button (S1) and hold until the back reaches the desired position. Then release the button.
- 2. To bring the back of the chair to an upright position,** press button (S2) and hold until the back reaches the desired position. Then release the button.
- 3. To raise the foot rest,** press button (S3) and hold until the foot rest reaches the desired position. Then release the button.
- 4. To lower the foot rest and lift the chair,** press button (S4) and hold until the footrest closes, then release the button. If you want to lift the chair, continue holding button until chair reaches the desired height. Then release the button.
- 5. To lower the chair from a raised position,** press button (S5) and hold until the chair reaches the floor. Then release the button. If you want to raise the footrest, continue holding the button until the footrest reaches the desired position. Then release the button. Button (S4) will also lower the chair from a raised position.
- 6. To lift the chair from a seated position** (footrest closed), press button (S6). Continue holding button until chair reaches the desired height. Then release the button.
- 7. To lift the chair with the back open** (back down, footrest closed), press button (S4). Continue holding the button until the chair reaches the desired height. Then release the button.
- 8. To close and lift the chair from a reclined position** (footrest up, back down), press button (S6). If you want to bring the chair to a seated position, hold the button until the footrest closes, then release it. Press button (S2) to bring the back the rest of the way up to seated position. If you want to close and lift the chair, hold down the button until the chair stops. All 3 motors will automatically shut off when the back closes all the way and the chair reaches its highest position.
- 9. Getting into your lift chair.** While standing next to your chair, press button (S4 or S6) and raise the chair to a height you can sit against. This height will vary from person to person depending on the individual's needs. Raise the chair high enough so you do not have to flop down into the chair.

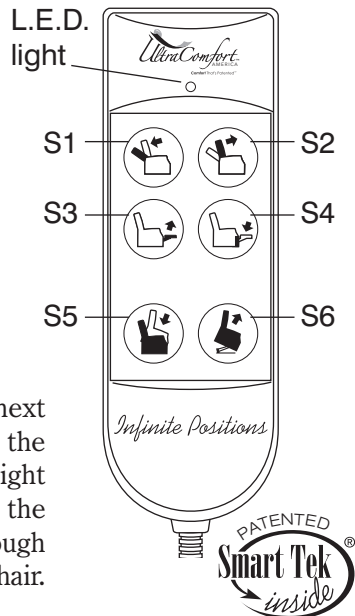

Flopping down into the chair will damage it and could cause personal injury. Press button (S3 or S5). As the chair continues downward, adjust yourself into a position that is comfortable to you. Continue holding button until chair is in the seated position (chair firmly on the floor), then release the button. Continue operating the chair with buttons S1 thru S6 until you reach your desired position.

10. Getting out of your chair. Press button (S4 or S6) and raise the chair to a height that enables you to easily stand. Release the button. Return the chair to the seated position (chair firmly on the floor) after getting out of the chair so children, pets or obstructions do not get under the chair. Repeat no. 9 when you want to sit down.

Note: The motors are set to shut off when they reach their fully extended or fully retracted position if the selected button is not released. If you should go past your desired position, select the button to bring you back to your desired position. You can repeat the process as often as you like.



Total Control
Used to experience our patented positioning technology

Position Control
Programmable buttons to customize your comfort

Custom Control
Arrow buttons are used to find your ideal back and footrest positions and also for position programming

AUTODRIVE HAND CONTROL OPERATION

- 1. Getting into your lift chair.** While standing next to your chair, press button (S7) and raise the chair to a height you can sit against. This height will vary from person to person depending on the individual's needs. Raise the chair high enough so you do not have to flop down into the chair. **Flopping down into the chair will damage it and could cause personal injury.** Press button (S5). As the chair continues downward, adjust yourself into a position that is comfortable to you. Continue holding button until chair is in the seated position (chair firmly on the floor), then release the button. You can also press buttons (S6, S8, S9, S10) and the chair will move to those positions.
- 2. Getting out of your chair.** Press button (S7) and raise the chair to a height that enables you to easily stand. Release the button. Return the chair to the seated position (S5) (chair firmly on the floor) after getting out of the chair, so children, pets or obstructions do not get under the chair. Repeat no. 1 when you want to sit down.


Micro Adjustments

- 1. To recline the back of the chair,** press button (S1) and hold until the back reaches the desired position. Then release the button.
- 2. To bring the back of the chair to an upright position,** press button (S2) and hold until the back reaches the desired position. Then release the button.
- 3. To raise the foot rest and seat,** press button (S3) and hold until the foot rest reaches the desired position. Then release the button.
- 4. To lower the foot rest and seat,** press button (S4) and hold until the footrest closes. Then release the button.

Note: The motors are set to shut off when they reach their fully extended or fully retracted position if the selected button is not released. If you should go past your desired position, select the button to bring you back to your desired position. You can repeat the process as often as you like.

Programming the Memory Buttons S5, S6, S8 and S10

Press one of the preprogrammed buttons (S5, S6, S8, S10) and hold until the chair stops. Press buttons (S1 thru S4) to micro adjust those positions to suit your needs. Use the following procedure to reprogram (S5, S6, S8, S10) buttons after you have the chair in your desired position.

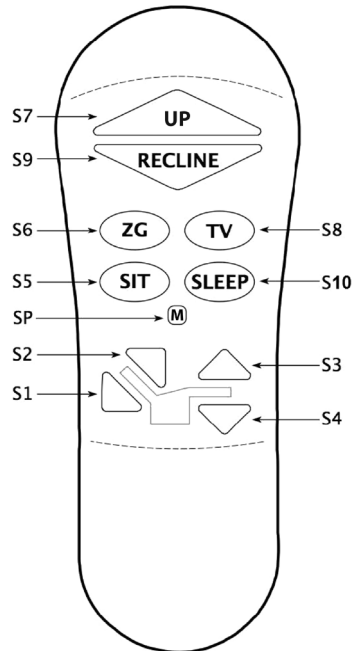

1. Press and hold program button (SP). The back light goes off while pressed.
2. In about 3 seconds the backlight will start flashing (one flash)
3. Within 5 seconds, press one of the memory buttons (S5, S6, S8, S10) to save the position to the button pressed. The back light will return to non-flashing position indicating that the new position is stored.

If a programmable memory button is not pressed within 5 seconds, or a non-programmable button is pressed, then the programming sequence is aborted and you will need to repeat steps 1 to 3 above. You may reprogram the memory buttons as often as you like.

Factory Reset

Resets all the programmed buttons (S5, S6, S8 and S10) back to the factory settings.

1. Press and hold program button (SP). The backlight goes off while button is pressed.
2. In about 4 seconds the backlight will start flashing.
3. Within 5 seconds, press button (S1). The backlight will change to non-flashing.
4. Within another 5 seconds, press button (S4). Factory reset is done.

If button (S1) is not pressed within 5 seconds, or a different button is pressed, then the programming sequence is aborted and you'll need to repeat steps 1 to 4 above.

Hand control disable

Hand control disable, disables all functions on the hand control

1. To disable, press and hold the program button (SP) for 4 seconds until the backlight starts flashing. Then press button (S7).
2. To enable, press and hold the program button (SP) for 4 seconds until the backlight starts flashing. Then press button (S7).

Function Lock

Should you ever want to limit the button function of your hand control, you can use the function lock feature to disable all button function except for buttons (S7 and S9).

1. To lock function, press and hold the program button (SP) for 4 seconds until the backlight starts flashing.
2. Press button (S1), press button (S2), press button (S3), press button (S6).
3. To unlock function, press and hold the program button (SP) for 4 seconds until the backlight starts flashing.
4. Press button (S1), press button (S2), press button (S3), press button (S6).

Chair Operation Model UC480 BHV

The PWR button needs to be turned on to activate all the functions of the hand control.

To turn the power on to the hand control, press the PWR button at the top of the hand control. The PWR button will glow green. To turn off the power to the hand control, press the PWR button again. PWR button will glow blue. Activating the hand control starts a 20 minute timer. To restart the timer after it shuts off, press the PWR button again. Lights will dim to conserve energy after about 10 seconds of non use.

Power Lock out

To lock the hand control, press and hold the PWR button for 4 seconds. The button will glow red.

To unlock the hand control, press and hold the PWR button for 4 seconds. The button will glow blue. After unlocking the hand control, you will need to press the PWR button to turn on the power to the hand control.

To recline your chair or lower it from the lift position, press the down arrow and hold until you reach your desired position. Then release the button.
To return to a seated position from recline or to lift your chair to a standing position, press the up arrow and hold until you reach your desired position. Then release the button.


Model UC480 BHV heat and massage operation

Press the MSG button to turn on the vibrators. Press the MSG button again to turn off the vibrators.

Press the Heat button to turn the heating pad on. Press the heat button again to turn off the heating pad.

The heat and massage work independent of each other and can be used separately or together and are factory set to turn off in 20 minutes. If you desire an additional 20 minutes you will need to press the buttons again to reactivate the heat and/or massage functions. Heat and vibration turn off when the up and down arrows are activated. Heat and vibration resume when the up and down arrows are released.

Note: Your power lift chair can be stopped at any time during lift or recline by releasing the selected button. The motor is set to shut off when it reaches its fully extended or fully retracted position if the selected button is not released. If you should go past your desired position, select the button to bring you back to your desired position. You can repeat the process as often as you like.

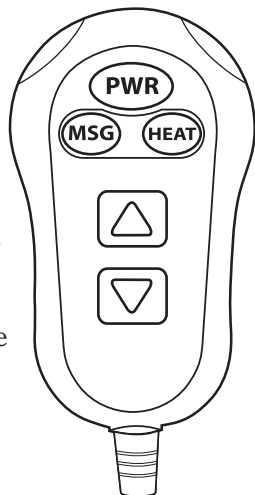

Should you ever encounter a problem with the operation of your lift chair, such as mechanical or electrical problems, discontinue all use of the product upon discovery of the problem. Contact the provider from whom you purchased your chair. They will be able to resolve problems and take care of service if needed.

NOTICE

To reduce the risk of potential product damage:

Follow these instruction to help ensure product quality over the life of the system. Failure to follow these instructions may invalidate the warranty.

Do not attempt to disconnect the power cord by pulling the power cord.

Always hold the power plug and pull it out.

Only use replacement parts specified by your provider. Unauthorized substitutions may result in fire, electrical shock or injury and will void the warranty. Only use the power supply that accompanied this system.

If their is a malfunction of any kind, please contact your provider for further instructions, and for returning it for examination, repair or replacement.

⚠ WARNING

- ⚠ To reduce the risk of fire or electrical shock, do not expose this product or any components of this product to rain or moisture.
- ⚠ Do not spill liquid on the system. Spilled liquid may damage the system.
- ⚠ Do not reach for any portion of the system that has fallen into water or any other liquid. Unplug immediately. Contact provider for service.
- ⚠ Use the power cord provided only. Do not overload wall outlets. The power outlet used for this product should be readily accessible for fast disconnection in case of an emergency.
- ⚠ Unplug the power supply should you find it necessary to clean or move the product. We strongly recommend using only a clean dry cloth.
- ⚠ You should unplug the system when it will not be used for a long period of time.
- ⚠ Keep all cords away from heated surfaces and pinch points.
- ⚠ Any changes or modifications to your system, or use of unauthorized accessories to your system not expressly approved by the manufacturer may be dangerous and voids any applicable warranties.
- ⚠ Close supervision is recommended when this product is used near children, invalids, or disabled persons. Always check the skin in contact with the heated area of this product.
- ⚠ Do not use outdoors. **FOR INDOOR USE ONLY.**

BATTERY BACK UP SYSTEM

Your UltraComfort power lift chair comes with a battery back up system so the chair can return to the seated position in the event of a household power failure. The batteries have enough energy to bring the chair to a seated position from the recline position. *The batteries do not have enough energy nor are they intended to operate the chair for a prolonged period of time in the absence of standard household voltage (120 volt, 60 Hz).* **Batteries should be replaced after one use.** Batteries should be changed yearly. (See pages 8 and 9.)

(Power Indication)

Your power lift chair is equipped with an indicator light on the hand control (StellarComfort only). This light illuminates green when any button on the hand control is pressed indicating power to the hand control.

Your power lift chair also comes with a power indicator light on the transformer and on the power cord connecting the motor to the transformer. The lights illuminate green constantly indicating power from the outlet to the transformer and the motor. The lights have been provided to help you identify electrical disconnects, should they occur, and to help your provider to diagnose electrical problems, should they occur.

Should the chair not operate and the lights are out, check all electrical connections to the chair. Make sure everything is plugged in properly and there is power to the outlet into which the chair is plugged (see installation on page 8 and 9 for electrical connections). **Should you ever encounter a problem with the operation of your power lift chair and the lights are on, such as mechanical problems or one light is out, discontinue all use of the product upon discovery of the problem.** Contact the provider from whom you purchased your chair. They will be able to resolve problems and take care of proper factory service if needed. You will need to be prepared to answer some basic questions about your power lift chair, such as model and serial number. The model and serial numbers are located on the rear of the lift frame. Your provider will ask you questions about your power indicator lights, such as which one is out. Prior to calling your provider, check all the electrical connections to the chair. Make sure everything is plugged in properly and there is power to the outlet into which the chair is plugged (see page 8 for electrical connections).

STELLARCOMFORT POSITIONS

Models UC520, UC550-JPT, UC550-M, UC550-L, UC550-M26, UC556-M, UC559-SMA, UC559-MED, UC559-LAR, UC559-M26

Your UltraComfort power lift chair has been equipped with StellarComfort. StellarComfort has been designed specifically for you. Offering you a wide range of body positioning technology, so you can choose the position that best suits your needs. StellarComfort accomplishes this by providing four primary motions: three recline, one lift and combinations of those motions, allowing you to settle into your favorite comfort position. Shown below are some of the StellarComfort positions. There are many to choose from. You may stop your UltraComfort power lift chair at one of the positions shown or anywhere in between just by releasing the buttons on the hand control when the chair reaches the position you desire.

CONSUMER LIFETIME AND SEVEN YEAR LIMITED WARRANTY

UltraComfort, through its authorized distributor/service organization, warrants this chair to the original consumer only, against defects in material or workmanship as follows:

Lifetime Warranty

UltraComfort warrants for the lifetime of the chair by the original consumer to repair or replace at their option the Lift Mechanism, Recline Mechanism and Chair Frame. UC544-SMA, UC544-MED, and UC544-LAR chair frames are limited to 5 years. Labor to replace these parts is subsidized on all models as follows: The UC214, UC332M, UC332L are limited to one year subsidized labor on above parts. The UC480, UC520, UC536-MLA, UC542-SMA, UC542M, UC542L, UC542-ME6, UC544-SMA, UC544-MED, UC544-LAR, UC549-SMA, UC549-MED, UC549-LAR, UC549-M26, UC550-JPT, UC550-M, UC550-L, UC550-M26, UC556M, UC559-SMA UC559-MED, UC559-LAR, UC559-M26, UC790, UC798-MLA are subsidized for a period of three years on above parts.

Three Year Parts Warranty

UltraComfort warrants the electrical parts of this chair against defect to original consumer only as follows: The UC214, UC332M, UC332L is limited to two year electrical parts warranty. UC480SME, UC520, UC536-MLA, UC542-SMA, UC542M, UC542L, UC542ME6, UC544-SMA, UC544-MED, UC544-LAR, UC549-SMA, UC549-MED, UC549-LAR, UC549-M26, UC550-JPT, UC550-M, UC550-L, UC550-M26, UC556M, UC559-SMA, UC559-MED, UC559-LAR, UC559-M26, UC772, UC790, UC798-MLA are limited to three year electrical warranty. Labor to replace these parts is subsidized for one year on all models. Replacement of the hand control and/or the transformer is expected to be done by the consumer.

continued

CONSUMER LIFETIME AND SEVEN YEAR LIMITED WARRANTY

continued

Limited Warranty Fourth Through Seventh Year

During the period beginning with the expiration of the full three year parts warranty offered above and continuing through the seventh year after date of original purchase by a consumer, UltraComfort will warrant the electrical parts on the following pro-rated basis. Year 4 earns a 35% discount off suggested retail price. Year 5 earns a 25% discount. Year 6 earns a 15% discount and year 7 a 5% discount. By way of example, a part costing \$100.00 that needs replacement in year 4 would cost \$65.00.

The UC214 is excluded from this portion of the warranty.

Terms Applicable to Warranties

UltraComfort will, at its option, repair and/or replace the defective parts and/or components provided that the warranty parts have not been subjected to misuse, abuse or improper service by the consumer; have not been damaged in transit or handling, and/or have not been altered or repaired by unauthorized representatives. This warranty does not include costs for unnecessary service calls, i.e., service calls solely for the purpose of educating the consumer about the product, or finding an unsatisfactory power connection, etc. Exceeding the weight capacity of the chair (published in each owner's manual) will void all warranties. Fabric shading cannot be guaranteed for replacement parts or replacement chairs.

Model UC214	325 lbs.
Model UC542-SMA, UC549-SMA, UC550-JPT, UC559-SMA	300 lbs.
Model UC544-SMA, UC544-MED, UC544-LAR	350 lbs.
Model UC332M, UC332L, UC480, UC520, UC536-MLA, UC542M, UC542L, UC549-MED, UC549-LAR, UC550-M, UC550-L, UC556M, UC559-MED, UC559-LAR, UC772, UC790, UC798-MLA	375 lbs.
Model UC542ME6	500 lbs.
Model UC549-M26, UC559-M26, UC550-M26	500 lbs.

UltraComfort will pay standard shipping rates on all warranted parts for the first year. The consumer will be responsible to pay cost for transportation, handling and packaging of the parts or components beyond the first year. UltraComfort shall not be responsible or obligated to pay for any costs or expenses for examination of any Warranted parts. All repairs and labor rates must be approved by UltraComfort before service is provided. Any repairs by unapproved or unauthorized representatives will void all warranties.

In the event that the product does not conform to this warranty at any time while this warranty is in effect, UltraComfort will correct the defect in conformity with the Consumer Limited Lifetime and Seven Year Warranty. THE LIMITED LIFETIME AND SEVEN YEAR WARRANTY SET FORTH ABOVE IS THE SOLE AND ENTIRE WARRANTY PERTAINING TO THE PRODUCT AND IS IN LIEU OF AND EXCLUDES ALL OTHER WARRANTIES OF ANY NATURE WHATSOEVER, WHETHER EXPRESS, LIMITED, IMPLIED, OR ARISING BY OPERATION OF LAW INCLUDING, BUT NOT LIMITED TO ANY IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. THIS WARRANTY DOES NOT COVER OR PROVIDE FOR THE REIMBURSEMENT OR PAYMENT OF INCIDENTAL OR CONSEQUENTIAL DAMAGES. Some states do not allow this exclusion or limitation of incidental or consequential damages so the above limitation or exclusion may not apply to you.

UltraComfort shall not be liable for any incidental or consequential damages or for any other damage or loss not contained in this warranty. This warranty gives you specific legal rights, and you may also have other rights which may vary from state to state.

CARE AND MAINTENANCE

Your lift chair will require routine maintenance checks. You can perform some of these checks, while others may require assistance from a provider. By following these maintenance checks, you can make sure your lift chair gives you years of trouble-free operation. If you have any questions or concerns regarding your lift chair's care and maintenance, contact your provider.

- ✓ Before cleaning your lift chair, consult your provider for the proper types of cleaner and cleaning methods.
- ✓ Keep all electrical components free from moisture and liquid spills. **Intended for indoor use only.**
- ✓ Periodically inspect all wiring harnesses to make sure they are not cut, damaged, frayed, or disconnected. If damage of any type is present, unplug the lift chair and contact your provider.
- ✓ Periodically check all fasteners for tightness, wear, and corrosion. If any of these are present, unplug the lift chair and contact your provider.



Warning: Be careful of pinch points and sharp metal parts (wear gloves).

- ✓ Replace the batteries after one use. Replace the batteries yearly, even if they were not used. **Batteries are intended for one use only!**



UltraComfort[™]
AMERICA



PLACE
STAMP
HERE

UltraComfort
401 BRIDGE ST
OLD FORGE PA 18518-2323





ULTRACOMFORT PRODUCT REGISTRATION

Register Now To Lock In Your Warranty and Receive Important Product Updates

SAVE TIME - REGISTER ONLINE: WWW.ULTRACOMFORT.COM

SERIAL NUMBER

Most Lift Chairs Have On Controller Wire or Back Rail of the Metal Frame

PRODUCT MODEL NUMBER

Most Lift Chairs Start with UC (UC550, UC484, etc).

PURCHASE DATE

FIRST NAME

LAST NAME

ADDRESS

ADDRESS LINE 2

CITY

STATE / PROVINCE ZIP / POSTAL CODE

E-MAIL

PROVIDER NAME

PROVIDER'S ADDRESS

ADDRESS LINE 2

CITY

STATE / PROVINCE

ZIP / POSTAL CODE

COUNTRY

Product comments:

Rate Provider Experience (Circle One - 1 Bad, 5 Great)

1 2 3 4 5

FOR YOUR RECORDS

Please fill in your Power Lift & Recline Chair information below. This information will be useful in the event that you should ever need to contact UltraComfort, Inc. concerning your lift chair.

Date of Purchase _____

Model _____

Serial Number _____

Your UltraComfort representative or provider

Name _____

Company _____

Address _____



CAUTION



Do not allow children to play on this mechanized furniture or operate the mechanism.

Leg rest folds down on closing so that a child could possibly be injured.

Always leave chair in a sitting position. Keep hands and feet clear of mechanism. Only the occupant should operate it.





CAUTION



READ BEFORE OPERATING
FAILURE TO DO SO MAY VOID WARRANTY

Chair must be placed to allow free movement between:

BACK of CHAIR and WALL
FOOTREST and any object in FRONT of CHAIR

Keep children, pets and foreign objects from back of chair or underneath chair and from in front of footrest while operating chair.

Do not allow children to play on this mechanized furniture or operate the mechanism.

Leg rest folds down on closing so that a child could possibly be injured.

Always leave chair in a sitting position. Keep hands and feet clear of mechanism. Only the occupant should operate it.

Failure to do so may cause injury or damage to chair.



1-866-793-5100