



AutoDrive™ 3.0

CUSTOMIZED COMFORT

Enjoy three types of comfort control:

Total, Position, and Custom.

1. **Ergonomic hand control design** – The shape, as well as the finger groove on the back, makes it easier to hold.
2. **Button display and positioning** – Simple button layout takes the guesswork out of operating a StellarComfort chair.
3. **Programmable positioning buttons** – All four position control buttons are programmable for customizable comfort.
4. **Lock functions** – Partial and total lock functions limit or restrict hand control usage for safety and convenience.



Total Control

Only two buttons are needed to experience our patented positioning technology

Position Control

Use these four programmable positioning buttons to customize your comfort

Custom Control

Use the arrows to move the backrest and footrest independently in order to find your ideal positions



UltraComfort™
AMERICA
Positioning you for life.



AutoDrive™ 3.0

CUSTOMIZED COMFORT

AutoDrive™ your body into its favorite positions!



Lift



Sit



TV Watching



Zero-gravity



Sleep



Trendelenburg

Total Control



The **Recline** button raises your legs and feet and has the capability to take you to the zero gravity position.

The **Lift** button raises the chair to an ideal standing position, saving your joints from unneeded stress.

Partial lock option disables all buttons except "lift" and "recline". See below for partial lock instruction.

Position Control



Sit brings you to a neutral seated position with the back upright and footrest closed.



TV keeps the back upright, but raises the seat box and footrest for relaxation of your lower back and legs.



ZG or "zero-gravity" is a level recline position that supports your legs and evenly distributes your weight providing back relief and pressure-free relaxation.



Sleep takes you to a lay-flat recline position allowing you to stretch out for a peaceful rest.



P initiates any programming features and lock capabilities.

Custom Control



Four arrow buttons move the chair back and footrest independently allowing for infinite positioning and optimal comfort.

Achieve the Trendelenburg position by holding the "down" arrow button that operates the back while reclined in the zero gravity position. This can promote circulation and provide a satisfying stretch to your lower back. (Consult your physician before trying this position.)

Full lock option available to disable entire hand control. See below for full lock instructions.

Partial Lock / Unlock Sequence



Full Lock / Unlock Sequence



Position Control / Reset Sequence

