



LINE2design, Inc. 2-WHEEL STAIR CHAIR INSTRUCTIONS Item # 70005-O

Please read these general instructions for the use and operation of the LINE2design, Inc. Emergency Evacuation 2-Wheel Stair Chair. This Stair Chair was designed for use by EMS and emergency professionals. The LINE2design, Inc. 2-Wheel Stair Chair requires **two** EMS Technicians for successful and safe transport of patients.

The instructions provided are not all-inclusive. Safe and proper use of the Stair Chair is solely at the discretion of the user. Safety information is included as a service to the user. All other safety measures taken by the user should be within and under consideration of applicable regulations and local procedures. Training on the proper use of the Stair Chair must be provided before using this product in an actual situation. The 2-Wheel Stair Chair should always be used with two people to transport patients.

Please keep these instructions for future reference. Store them with the Stair Chair for reference and/or new users.

PREPARATION FOR USE

To unfold the chair, place one hand on the front of the seat and one hand on the back of the chair and push apart until it locks into the open position.

At the upper back of the seat are two non-locking handles that pull up for transporting patients and fold down for storage.

At the front and bottom of the chair are two extending handles with push buttons on top to extend or return the handles to the normal position. The extended handles have two different positions. The lock is underneath the chair seat, pull up and adjust horizontally, release to change horizontal position.

USE WITH PATIENT

To place a patient in the Stair Chair:

1. Unlatch all three safety straps from the seat back, seat of the chair and around the leg area.
2. Position the patient in the chair.
3. Tighten safety straps securely, and buckle all three straps. The safety straps should be secure around the patient.
4. Once the patient is secure, have a second EMS Technician double check all safety straps.
5. The Stair Chair is then ready for lifting.
6. To transport the patient, one EMS Technician should lift the front handles and the other EMS Technician should lift the handles from the chair back.
7. The patient can then be transported to the desired area.

STORAGE

To store the chair after use, fold the chair, engage lever under the seat, release lever and fold seat up.

CLEANING

Wipe down with clean cloth, use mild soap and water. **NO BLEACH**

LINE2design, Inc.
1500 Crafton Avenue, Building 128
Mentone, CA 92359
909-798-1829
www.LINE2design.com
www.LINE2EMS.com